Nettoyage

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Cleaning

Option 1. Clean ContactEZ Dental Strips in an Automatic Washer.

Option 2. Clean ContactEZ Dental Strips by hand.

Step 1. Holding the ContactEZ Dental Strips under a faucet, run regular tap water over each strip, and scrub the stainless steel surface using a brush.

Step 2. Clean the ContactEZ Dental Strips using General Cleaning Solution in an Ultrasonic Tank for 10 - 20 minutes.

Step 3. Rinse the ContactEZ Dental Strips under a faucet with regular running tap water.

Additional mention : refer to the next page.

Sterilization

Step 1. Sterilize the ContactEZ Dental Strips in a standard steam heat autoclave (not dry heat) at 30 psi max for 7 minutes, with a maximum temperature of 273°F/134°C. ContactEZ recommends using distilled water in the autoclave.

Step 2. Allow the ContactEZ Dental Strips to cool down for a period of 5 minutes once the sterilization process is complete.

Step 3. Remove the ContactEZ Dental Strips from the autoclave by gloved hand. The strips are now ready to use.

Disposal

When the diamond powder has worn off of the stainless steel strip, ContactEZ Dental Strips may be disposed of with normal waste products.

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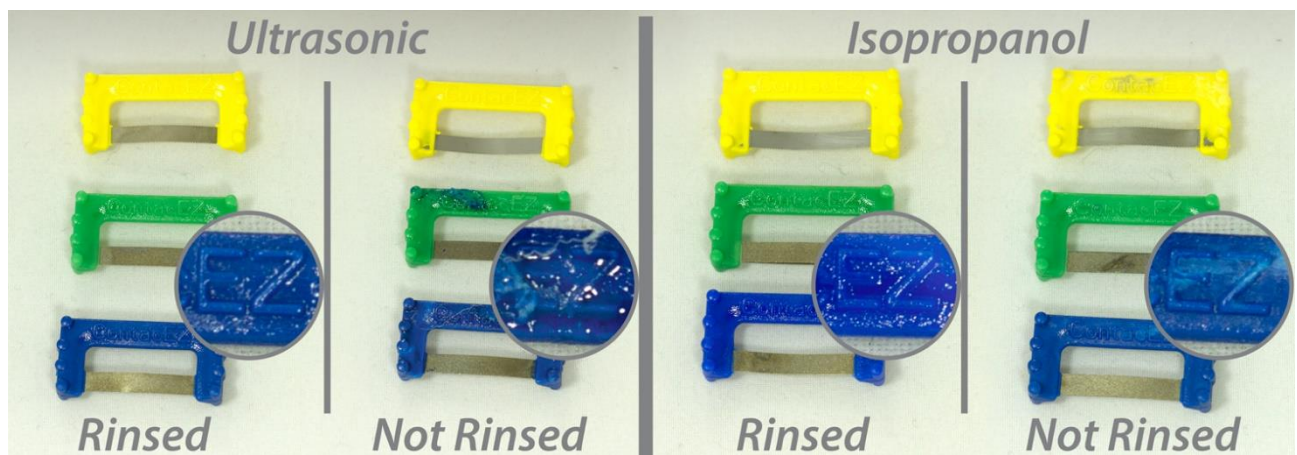
Exemples de dommage :

ADDITIONAL INFORMATION

MANUAL CLEANING FOR STRIPS BEFORE STERILIZATION

The strips should not be cleaned with a chemical disinfectant solution before sterilization.

During sterilization, residual chemical disinfectant including solution as chlorine bleach can damage plastic bond occurring deformation and/or breakage.

In case of using disinfectant : Rinse thoroughly the strips with water

Examples of damage :