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Orthodontic instruments have been designed to provide long and faithful service. They are medical devices that come into contact with mucous membranes and skin. They are intended to be used by healthcare professionals and/or technicians with appropriate skills to ensure their proper handling and their sterility in each use.

In order to maintain the orthodontic instruments in good quality and proper function, it is important to take into account the following recommendations :

PRECAUTIONS FOR USE AND STORAGE :

- **Protection of personnel and third parties :**

Contaminated instruments must be disinfected as soon as possible to ensure maximum safety for persons using them. When handling contaminated instruments, it is necessary to wear protective gloves that comply with the requirements of **Regulation (EU) 2016 / 425 on Personal Protective Equipment (PPE)**.

- **Before each first use :**

New orthodontic instruments must be thoroughly cleaned and sterilized according to validated processes.

Note : Instruments must be lubricated before each sterilization.

- **In routine before each use:**

- Examine each instrument carefully to ensure that it is not broken, cracked or damaged, especially tips of jaws and axis.
- Check the condition of the locking and moving parts.

- **Improper handling and maintenance, as well as improper use, can lead to premature wear of orthodontic instruments.**

The following recommendations are to be respected :

- Use only distilled, osmosis or filtered water when diluting cleaning solution and when rinsing.
- Lubricate regularly the instrument joints with silicone lubricant (for flexibility of joints and protection of instrument).
- Check visually for abnormal signs or damage before sterilization and discard immediately damaged instruments, if necessary.
- Check the mobility of jointed instruments and correct movement of locking mechanisms.
- Always leave jointed instruments in the open position during cleaning and rinsing.
- Dry instruments thoroughly, especially those with steel inserts.

- **Storage conditions :**

At room temperature with low humidity to avoid condensation.

CONTRAINDICATIONS :

- Do not use damaged orthodontic instruments.
- Do not have unqualified personnel or service repair orthodontic instruments.
- Do not use disinfectant solution with binding protein effect, or NaCl solution, brushes or steel wool and/or abrasive drying cloths/papers.
- Do not use tap water (which can cause rust and wear).
- Do not use lubricants with additives.
- Do not soak instruments in disinfectant solution for too long.
- Do not sterilize by heat instruments with plastic tips.
- Do not clean sharp instruments or instruments with welded parts in the ultrasonic bath.

TREATMENT : CLEANING - DRYING - STERILIZATION

- **Cleaning (systematically in an ultrasonic bath) :**

- Clean for 10 minutes with a disinfectant solution containing rust inhibitor.
- Rinse thoroughly with distilled, osmosis or filtered water.
- Leave instruments completely open during cleaning and rinsing.

- **Drying :**

- Let them drain for 5 minutes.
- Dry the instruments very carefully with compressed air blast and/or non-abrasive paper/cloth.

- **Sterilization :**

Steam sterilization :

- Respect maximum load allowed by equipment's manufacturer during sterilization cycle.
- Ensure that the sterilization time and temperature parameters defined by the manufacturer are respected.
- Use sterilizers that comply with the **EN NF 13060 +A1 Small steam sterilizers**.
- Pack devices in appropriate bags for steam sterilizers.
- Load the instruments in the open position on trays without having any contact between them.
- Perform sterilization in accordance with **ISO 17665-1: 2006 Sterilization of health care products**.
- Depressurize the equipment and allow the instruments to cool.
- Remove the instruments and make sure they are dry before storing them.

Cold sterilization

Note: this sterilization method is only for instruments with plastic tips :

1. Soak the instruments in disinfectant solution:
 - Use non-acidic solution containing rust inhibitor when using glutaraldehyde solution.
 - Use distilled, osmosis or filtered water for dilution.
 - Avoid contact with quaternary ammonium compounds and iodophores.
2. Rinse pliers in the open position with distilled or filtered water.
3. Let them drain for 5 minutes.
4. Dry the instruments very carefully with compressed air blast, and/or non-abrasive paper/cloth.

IMPORTANT:

Any responsibility for products that have been modified or used for purposes other than those intended or in an inappropriate manner is excluded.

All of above-mentioned instructions have been validated as appropriate for processing a reusable medical device.

Any serious incident arising from the use of the medical device should be notified to the manufacturer and to the regulatory authority of the member state where the patient/user is established.

In compliance with the applicable statutory provisions and, above all, for the protection of our employees, we require a signed "Proof of decontamination" for each return. Please ensure that this "Proof of decontamination" is filled in and enclosed with each product return (complaint / repair / other reason for return) and package the product such that no risk of injury exists for our personnel in incoming goods inspection while unpacking.

Article no.: Article designation:

If possible provide information on

Delivery note N°: of [date] Batch N°:

Reason for return:.....

We hereby confirm that

- The medical device(s) enclosed with the letter has/have not come into contact with blood, tissue or other bodily substances/fluids and hygienic safety can be confirmed by signing.
- The medical device(s) enclosed with the letter has/have come into contact with blood, tissue or other bodily substances/fluids and have been disinfected, cleaned and sterilized according to the applicable hygiene requirements for medical devices and the manufacturer's specifications, which we confirm by signing.

Specifications on disinfection, cleaning and sterilization:

- Disinfection and cleaning have been performed manually
 - Disinfection and cleaning have been performed mechanically
 - Steam sterilization (3 min. at 134°C / 15 min. at 121 °C)
 - Other method (please specify)
- The medical device(s) enclosed with the letter could not be decontaminated!

Reason:.....

Stamp from the institution

Date

Signature (and full name)

IMPORTANT : Should we not receive this document or a comparable confirmation, we reserve the right to return the goods to the sender at their own expense.

SURVEY ON MAINTENANCE METHODS OF ORTHODONTIC INSTRUMENTS

The informations below are specific to procedures that were implementing in your office. They will also help us to give you an appropriate answer to the issue met. Thank you in advance for your cooperation,

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1 – CLEANING :	<input type="checkbox"/> YES	<input type="checkbox"/> NO	
<input type="checkbox"/> Manual			
<input type="checkbox"/> Automatic	Material : -----		
Time of cleaning: -----			
PRODUCT(S) : -----			
WATER :	<input type="checkbox"/> tap water	<input type="checkbox"/> demineralised water	<input type="checkbox"/> Other -----
RINSING :	<input type="checkbox"/> YES	<input type="checkbox"/> NO	
If yes, which kind of water ?: -----			
DRYING	YES <input type="checkbox"/>	NO <input type="checkbox"/>	
Tissue	<input type="checkbox"/>		
Compressed air	<input type="checkbox"/>		
others	<input type="checkbox"/> -----		

2 – DECONTAMINATION :	<input type="checkbox"/> YES	<input type="checkbox"/> NO	
<input type="checkbox"/> Manual			
<input type="checkbox"/> Automatic	Material : -----		
Time of decontamination : -----			
PRODUCT(S) : -----			
WATER :	<input type="checkbox"/> tap water	<input type="checkbox"/> demineralised water	<input type="checkbox"/> Other-----
RINSING :	<input type="checkbox"/> YES	<input type="checkbox"/> NO	
If yes, which kind of water ?: -----			
DRYING	YES <input type="checkbox"/>	NO <input type="checkbox"/>	
Tissue	<input type="checkbox"/>		
Compressed air	<input type="checkbox"/>		
others	<input type="checkbox"/> -----		

3 – STERILIZATION :	<input type="checkbox"/> YES	<input type="checkbox"/> NO
<input type="checkbox"/> Cold sterilization – product : -----		
<input type="checkbox"/> Autoclave	Time of cycle : ----- / T°maximum : -----	
Protection : <input type="checkbox"/> YES – which way (bags, oiling...) : -----		
<input type="checkbox"/> NO		

4 – LUBRIFICATION :	<input type="checkbox"/> YES	<input type="checkbox"/> NO
When ? ----- / Frequency ? -----		
Product(s) : -----		

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