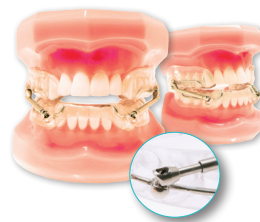
INDICATIONS :

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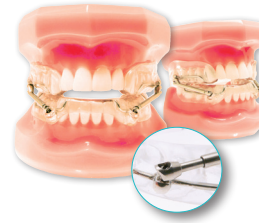
INDICATIONS :

This appliance is an invasive articulation device of an orthodontic set intended to cause a displacement of a mandible in relation to a maxilla. It is indicated in the treatment of a class II skeletal shift by retro-mandibular and recommended as an anti-snoring mandibular propulsion device.

← Liberty
Bielle →

This device is intended to be used on any public. It is a single-use device, manufactured on medical prescription and intended for use on a single patient.

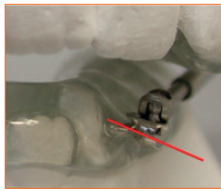
PRECAUTIONS FOR USE :

- The self-locking clamping screw once loosened must be replaced with a new one to ensure a new locking if necessary.
- The device can be adapted by considering the shortening of the stem
- Decontamination/sterilization rules must be applied in accordance with regulations to avoid any risk of cross-contamination.


The Liberty Bielle™ was designed with manufacturing comfort in mind, however, a few basic rules should be observed depending on the type of substrate used :

LIBERTY BIELLE™ WITH ELEVATION :


The support is 5mm. It accepts a wire from 1.1 mm to 1.2 mm only in Elgiloy®. The maxillary support must be positioned distal and cervical to the first molar. The mandibular support must be positioned mesially and at the cervical level of the first premolar. It is imperative to respect the planned inclination of the supports.

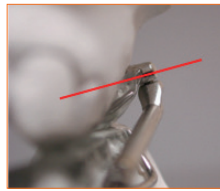The mandibular supports must be tilted downwards (about 15°)

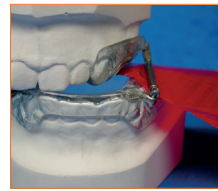The jaw supports must be tilted upwards (about 15°)

The resin in the vestibular area (often at the level of the premolars) should not interfere with lateral movements. Take the time to remove the excess resin with marker paper. It is possible to open «windows» in the vestibular area of the 1st and 2nd mandibular premolars, to lighten the appliance.

LIBERTY BIELLE™ WITHOUT ELEVATION :

The support is 4.5 mm and the wire is 0.9 mm in Elgiloy® only. The maxillary support must be positioned distal to the crown of the first molar. The mandibular support must be positioned mesially at the level of the crown of the first premolar. The supports are parallel to the plane of occlusion. Without elevation, the inclination of the supports is less important. In any case, the support on the flat side (the side that holds the ball screw) must be completely clear. If you opt for the pivot with pre-welded wire, it is important to bend the wire at a distance of 2 to 3 mm from the weld or the wire will break. If these rules are respected the Liberty Bielle™ will give you complete satisfaction.

MAINTENANCE OF THE UNIT :

- The Liberty Bielle™ should be maintained daily and removed for eating.
- Brush the device with a toothbrush (without toothpaste).
- The device can also be immersed in a fluoride-free dental cleaning solution for approximately one hour.

CONTRAINDICATIONS :

- Not to be used for a class II shift without retro-mandibularity.
- Do not use in case of allergy to Nickel.
- Do not perform any work on the connecting rod head (risk of breakage).
- Do not clean with toothpaste (risk of corrosion).
- Do not bring the device into contact with fluorine-based products.

IMPORTANT

Any liability for products that have been modified or used for purposes other than those intended or in an inappropriate manner is excluded.

Any product-related adverse reaction should be reported to the manufacturer and any serious incident should be reported without delay to the competent authority of the Member State in which the patients and practitioners are based.

SOLUTIONS TO COMMON PROBLEMS

PROBLEM

Screw breakage.

Finding: :

Uncleared supports.

Solution :

Clear supports of resin residue.

PROBLEM

No laterality.

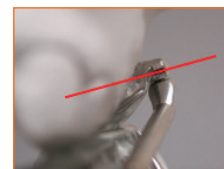
Finding: :

Incorrect inclination of the supports.

Solution :

Mandibular supports facing downward.

Maxillary supports pointing upwards.

PROBLEM

No laterality.

Finding: :

Poor positioning of supports.

Solution :

Maxillary support: position distal to the crown of the first molar.

Mandibular support: position mesially at the level of the crown of the first premolar.

PROBLEM

No laterality.

Finding: :

Resin too thick vestibularly at the level of the premolars.

Solution :

Adjust the resin laterally with marker paper.

PROBLEM

The screws are unscrewed.

Finding: :

Screw not fully screwed in.

Solution :

You absolutely have to screw it in all the way with the special Liberty Bielle™ Screwdriver.

PROBLEM

The screw no longer holds the connecting rod.

Finding: :

Incorrect handling of the connecting rod in the mouth.

Solution :

The connecting rod must always be rotated on a horizontal axis, never on a vertical plane. Please inform the patient. Replace with a new connecting rod and screw.

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