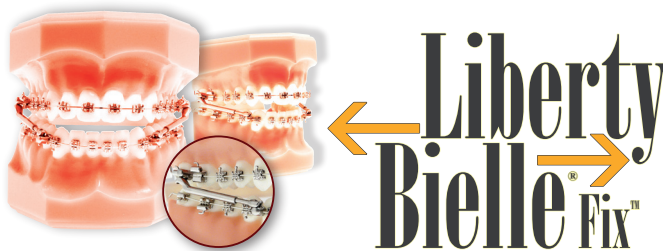
INDICATIONS :

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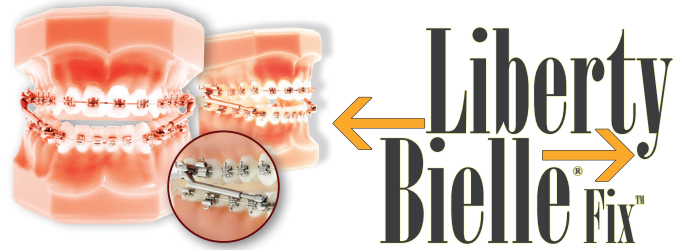
INDICATIONS :

This appliance is an invasive articulation device of an orthodontic set intended to cause a displacement of a mandible in relation to a maxilla. It is indicated in the treatment of a class II skeletal shift by retro-mandibular and is recommended as an anti-snoring mandibular propulsion device.

This device is intended to be used on any public. It is a single-use device, manufactured on medical prescription and intended for use on a single patient.

PRECAUTIONS FOR USE :

- The self-locking clamping screw once loosened must be replaced with a new one to ensure a new locking if necessary.
- The device can be adapted by considering the shortening of the stem
- Decontamination/sterilization rules must be applied in accordance with regulations to avoid any risk of cross-contamination.
- After adjusting the device and before inserting it into the mouth, we recommend cleaning it with ultrasound.
- Note: For routine cleaning, the use of a low fluoride toothpaste is recommended.

The Liberty Bielle® Fix™ was designed with placement comfort in mind. The Liberty Bielle® Fix™ includes three lengths: 32mm for placement between the 1st and 2nd molars and between the canines and 1st premolar; 22mm and 20mm for placement between the second premolar and 1st molar and between the canines and 1st premolar. Select the most appropriate Liberty Bielle® Fix™ and adjust if necessary. To do this:

1

Screw the upper bracket onto the arch between the first and second molars or between the first molar and second premolar using the hex key

2

Place the upper arm on the support by positioning it at 90°, as soon as the tube is oriented at less than 85° the two parts are secured.

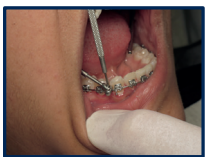
3

If necessary, mark the length of the tube up to the mesio-distal space between the canine and lower first premolar.

4

If necessary, cut the tube to length

5

If necessary, cut the lower connecting rod to length and screw the bracket onto the arch between the canine and the first premolar.

Advice

The length of the lower connecting rod should not exceed the distal end of the upper connecting rod by more than 2 mm in the closed position, to avoid discomfort to the patient.

Can be used on arches from .018 x .025 to .022 x .028. Due to the forces generated, the Liberty Bielle® Fix™ can be attached to NiTi arches.

CONTRAINDICATIONS:

- Not to be used for class II shift without retro-mandibularity.
- Do not use in case of nickel allergy.
- Do not perform any intervention on the connecting rod head (risk of breakage).

IMPORTANT

Liability for products that have been modified or used outside their intended purpose or in an inappropriate manner is excluded.

Any product-related adverse reaction should be reported to the manufacturer and any serious incident should be reported without delay to the competent authority of the Member State in which the patients and practitioners are established.

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