## INDICATIONS

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## INDICATIONS

Multi Family orthodontic appliances are removable medical devices for functional education. They guide dental growth in children and are indicated in the treatment of dental malocclusions. They also help to correct bad breathing, swallowing, chewing and posture habits. These appliances are intended for use on children aged from 4 to 13 or older by orthodontists with the professional skills necessary to use them safely and correctly.

## PRECAUTIONS FOR USE

- **These devices are delivered NON-STERILE. A cleaning is recommended before each first use.**
- **The device is designed to be used on a single patient and then discarded after treatment.**

**Note :** If the device has been used during a trial in mouth on another patient, exclusively at dental clinic and under control of the dentist, it may exceptionally be reused provided it has been inspected, cleaned and sterilized according to the processes recommended below.

**Health care professionals are responsible for providing the necessary information to patients to use properly these medical devices according to the following guidelines :**

- Always check the device before placing it in the mouth. In case of abnormal signs or damage stop wearing it and contact healthcare professional.
- Rinse the device daily before putting it in the mouth and after removing it from the mouth.
- Use a toothbrush to clean it. The device can be disinfected by immersion in boiling water for at least 1 minute. The storage box can be cleaned in a dishwasher.
- Do not use toothpaste as the abrasive particles in toothpaste can scratch and damage the device.
- Respect the frequency and duration of wear of the medical device recommended by the practitioner.
- Always keep the medical device in its storage box when not in use.

**Each professional is required to check regularly the medical device for signs of wear or damage. They must also inform their patients on what to do if they detect any abnormal signs or damage to the medical device.**

## CONTRAINDICATIONS

- **Do not use toothpaste or abrasive solutions/brushes for cleaning to avoid scratches and early wear of the device.**
- **Do not use solutions containing chlorine, phenol or amines or/and solutions that are not compatible with the device.**
- **Do not use detergents containing aldehydes.**

## TREATMENT : Cleaning - Disinfection - Sterilization - Storage

- Follow the instructions provided by the manufacturer of the disinfection, cleaning and sterilization equipment and used solutions (detergents/sterilizers).
- Before each step, check the device for abnormal signs of damage.

## CLEANING

- Use enzymatic detergents with a neutral pH for the cleaning of medical devices made of silicone and not classified according to the Regulation (EC) N°1272/2008 on classification, labelling and packaging of substances and mixtures.
- Use an Ultrasound Bath (optional) if needed:
  - Soak the device in the ultrasonic bath with disinfectant solutions suitable for silicone for at least 15 minutes at 30°C.
  - Scrub carefully with a soft, non-abrasive brush if there are visible impurities.
  - Rinse with room temperature water for at least one minute.

## CLEANING AND DISINFECTION

### Automatic Cleaning (Washer-Disinfector) :

- Place the device in a vertical position: positioning of the holes to avoid the formation of air bubbles.
- Program the machine to rinse and disinfect the holes.
- **Note:** The temperature of the washer-disinfector program can be freely selected as the silicone can withstand a temperature of 134°C.
- After cleaning, visually check that the device is clean and repeat the operation if necessary.

### Manual cleaning :

- Scrub with a soft brush using a neutral pH enzymatic detergent as indicated for at least one minute.
- Clean and rinse the medical device and its holes thoroughly: special attention must be paid to holes in the device.
- Check the cleanliness of the device and repeat the operation if necessary.

## STERILIZATION

- Wrap the medical devices in appropriate pouches for steam sterilizers. Materials used for wrapping should be manufactured in accordance with **ISO 11607-1: 2019 Packaging for terminally sterilized medical devices - Part 1: Requirements for materials, sterile barrier systems and packaging systems**.
- Arrange the devices in trays intended for sterilization without contact between them.
- Pack the trays and place them in the sterilizer according to the manufacturer's instructions without exceeding the maximum load.
- Proceed to steam sterilization, according to the standard **NF EN 13060+A1 : 2018 relating to small steam sterilizers**.
- The sterilization procedure should be carried out in accordance with **ISO 17665-1: 2006 "Sterilization of health care products - Moist heat - Part 1: Requirements for the development, validation and routine control of a sterilization process for medical devices"**.

**Recommendations :** recommended temperature of **134°C / 273°F for 3 minutes**, followed by a drying time of **20 minutes** minimum.

- Check the cleanliness of the device and repeat the operation if necessary.
- Wrap the medical devices carefully to avoid contact with moisture and ensure sterility.

## STORAGE

Store packaged and sterilized products in clean environment, away from moisture and direct light.

### ATTENTION:

**Any liability for products that have been modified or used outside their intended purpose or in an inappropriate manner is excluded. All above-mentioned instructions have been validated as appropriate for the treatment of a medical device for the purpose of its reuse.**

**Any adverse event related to medical devices should be reported to the manufacturer and any serious incident should be reported without delay to the competent authority of the Member State in which the patients and professional users are established.**

## SIZE SELECTION WITH XPSIZER

**Image 1:**

- Place the high mark between the left maxillary lateral incisor and the canine.
- Smooth the sizer between the right maxillary lateral incisor and the canine (mesial surface of the canine).
- Sizes 10, 20, 30, etc. are shown with larger marks, while sizes 15, 25, 35, etc. are shown with smaller marks. (In this photo, the measurement is equal to «20»).

**Image 2 :**

- Choose an appropriate size without having an overlap or diastema.
- Place the device in the patient's mouth.
- Make sure that the teeth are well placed in the slots.

## FIT: The fit is the most important step in size selection

**Image 1:** The canine rests at the bottom of its slot and thus the device does not exert any mesio-distal force on the canine. If there is no overlap or risk of overlap, the size shown in the picture is correct. However, if there is overlap or risk of overlap, a larger size should be selected to allow for expansion of the arch perimeter.

**Image 2 :** One size larger than **image 1**. The canine is guided by the device downwards from the canine slot and then subjected to a force that pushes it distally. This size is suitable when there is overlap or risk of overlap and an extension of the arch perimeter is required.

**Image 3:** Three sizes larger than in **image 1**. The device is too large. The canine is guided until it reaches a wall between two locations and the device does not guide the teeth correctly. Select a smaller size.

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