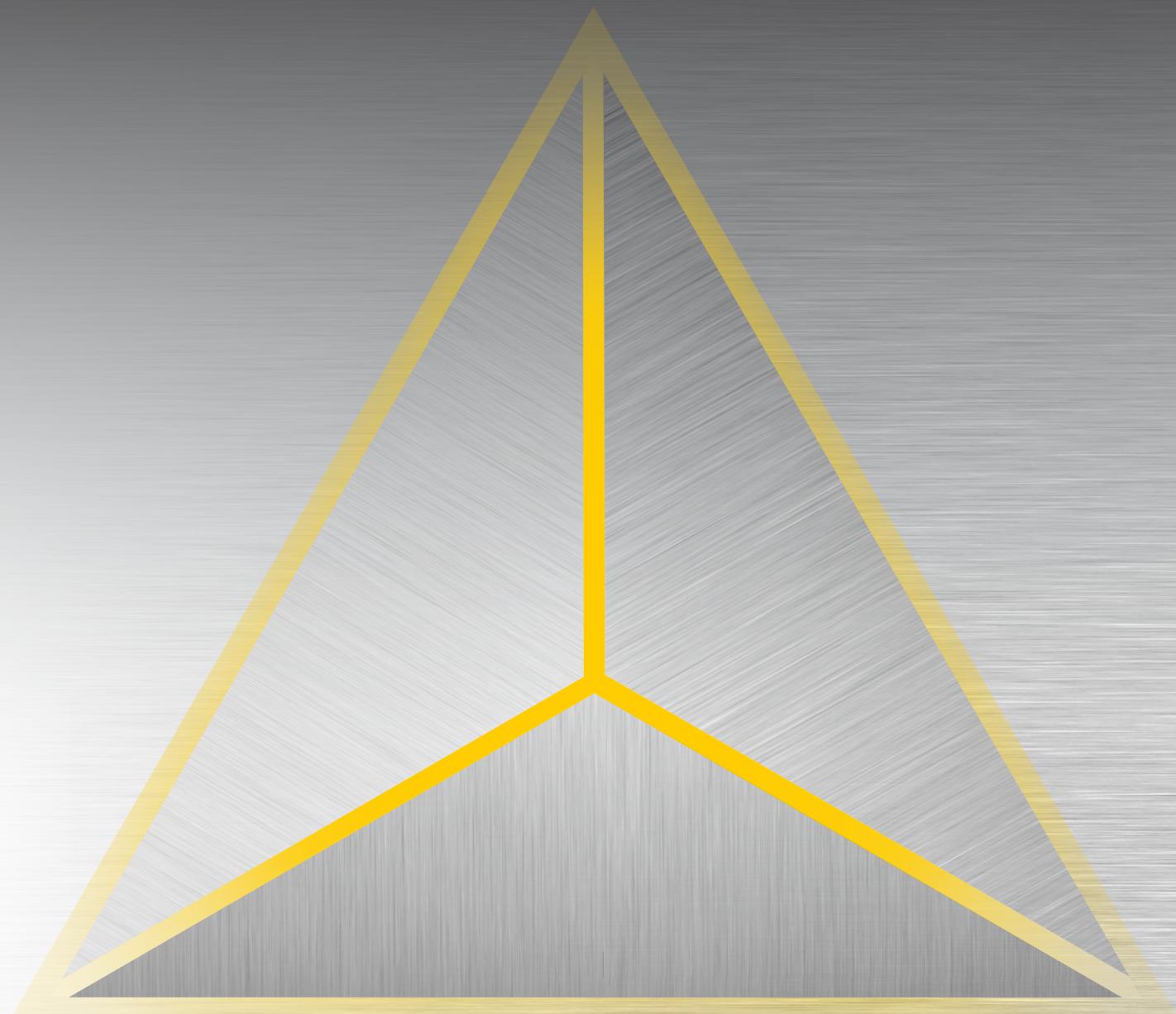
S-A-M

PRODUCT CATALOG 2022

VDDI
Dental Solutions.
German Manufacturers.

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The innovations of SAM® go back to Heinz Mack, dentist in Munich. The reason was courses on functional teaching. An articulator that was very stable in the centric was needed for skull-fitting model assembly with adjustable guides. This is how the specifications for the modular device with the project name SAM® (= Schul Artikulator München) came about, which a few years later was further developed into a fully adjustable articulator system. Very well known is the AXIOGRAPH® with the electronic additional device AXIOTRON® and the contactless measuring AXIOQUICK® Recorder. But many more products in the dental market go back to the development work of SAM®. The continuity of the company is guaranteed by the second generation. Florian Mack took over the management in 2012 and opened a new chapter in the company's history towards the future with the succession and the consulting activities of the company's founder.

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A UNIQUE DIGITAL ARTICULATOR SOFTWARE

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This unique software offers the possibility to better understand the connections between articulator as patient simulator and the dynamic movements and positions of the temporomandibular joint. First and foremost, this digital articulator software was

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developed for teaching, researching, science and practice. With sample files and the change of the articulator parameters or the model position different scenarios can be simulated. The effects on the teeth can thus be displayed quickly and logically.

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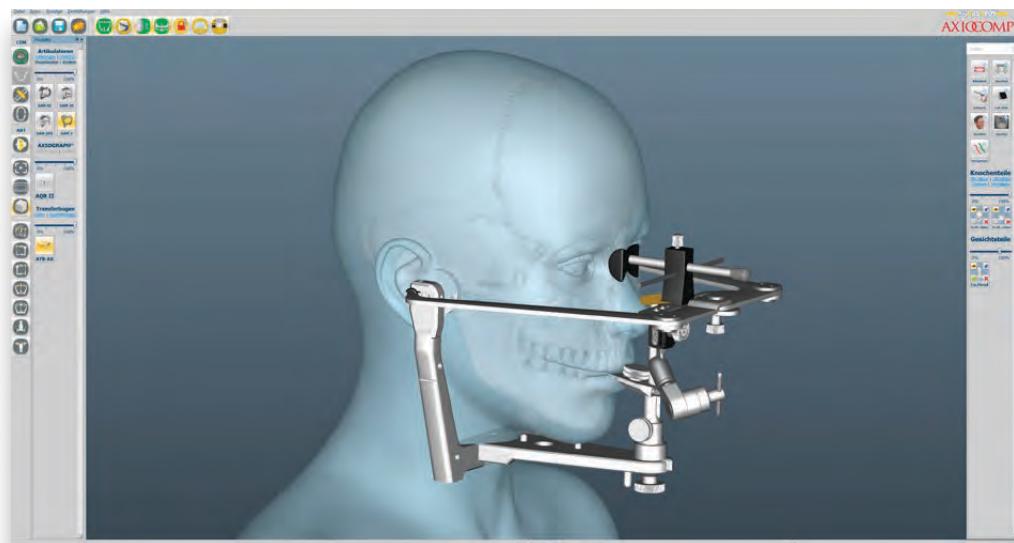
We provide you with a free basic version for download. All important requirements are included. So you can easily learn how to work with the Artikulator & Axiocomp.

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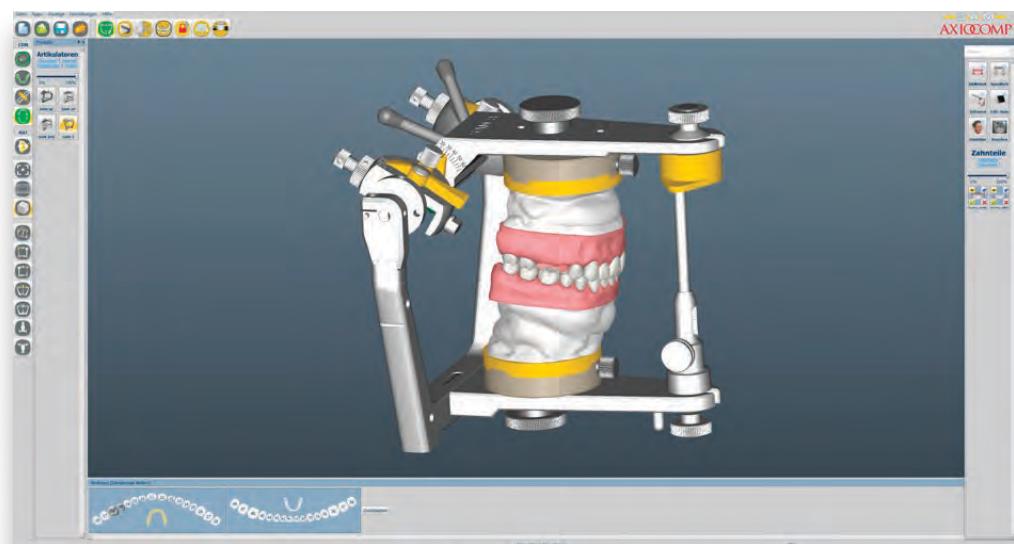
Overlaying digital data sets: For better visualization, the skull and face can be simulated with an articulator and transfer sheet overlay. The controllability of the transparency helps to understand the relationships and the alignments.

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Different SAM articulators and models can be selected and animated. A dynamic movement in the articulator can be illustrated, zoomed and viewed from any angle down to the smallest detail. Bennett and condylar trajectory inserts, the incisal table and other accessories can be displayed and its settings changed.

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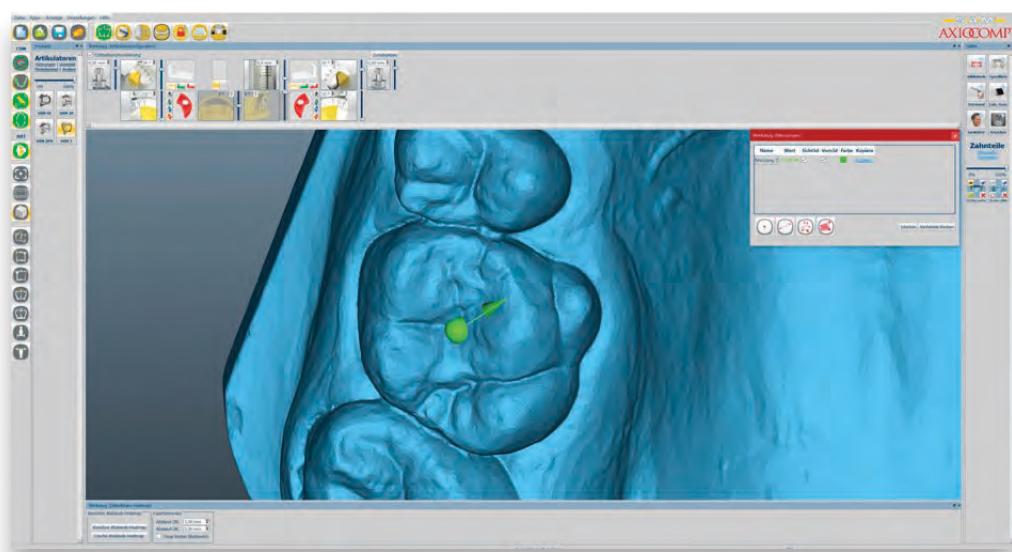
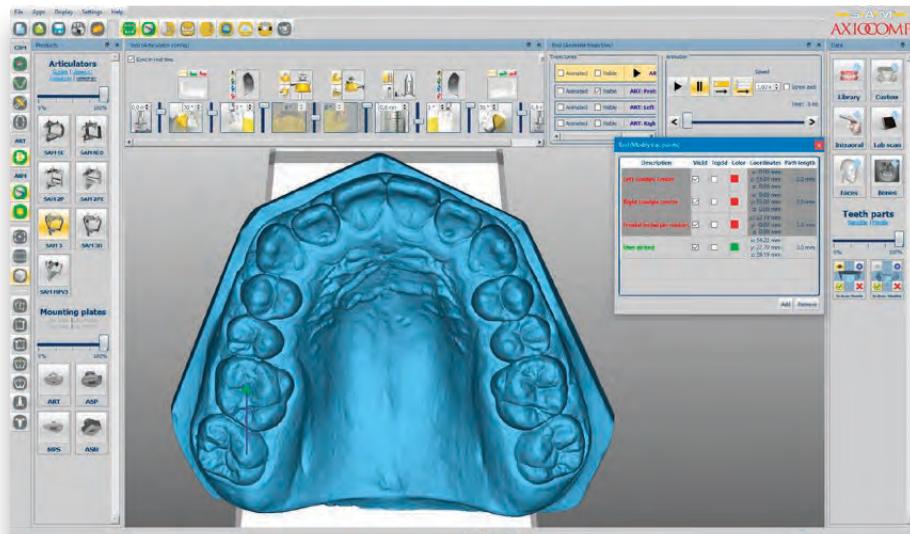
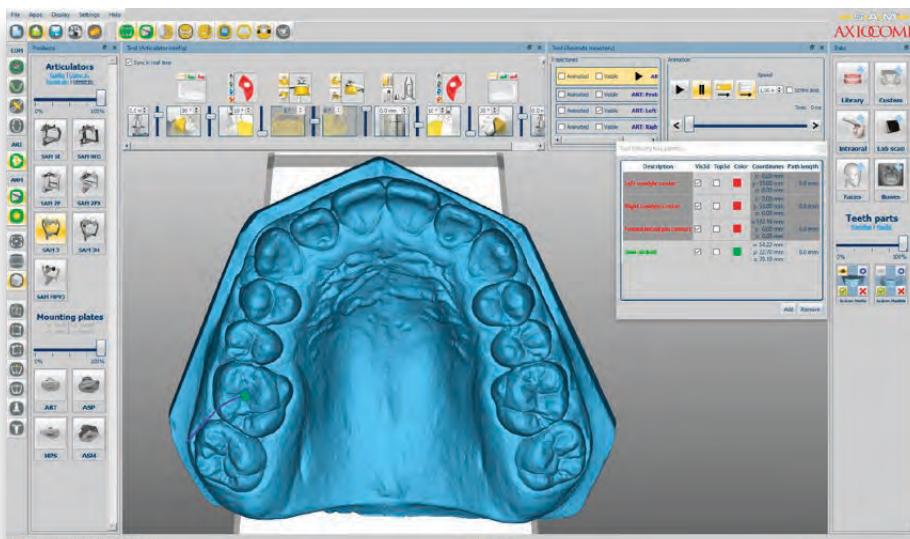
Points and routes can be marked freely on the teeth. The system informs about the 3D space coordinate as well as about all edge lengths and their angles to each other during a distance measurement. Additional applications allow this program to be used as a platform for the integration of external data sets, such as model scans, motion recording,

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By marking different contact points on the occlusion, the cusp movements are going to be revealed through the excursion movements. In this example, different paths with different articulator programming of the Bennetts are shown.

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Important: This software is not a medical device in the sense of §3 MPG.
It is not used for diagnosis, therapy or monitoring of diseases and injuries.

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The teaching software represents the simulation of the temporomandibular joint. Assembly errors, temporomandibular joint problems and bite errors can be reproduced.

AxioSim® is not a software in which patient data can be documented in order to identify incorrect bites and situations. AxioSim purely offers the possibility to familiarize oneself with the geometric framework conditions and to search for causes and remedies.

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The user is shown which change causes which effect, graphically and numerically. The following parameters can be changed: Axis error position (distance and direction), patient rotation radius, occlusal plane angle, occlusal plane inclination, anterior guidance angle, molar cusp inclination, occlusal length and the anterior thickness of the centric record. The display of various reference planes rounds off the setting options.

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We offer different license systems that allow different setting options.

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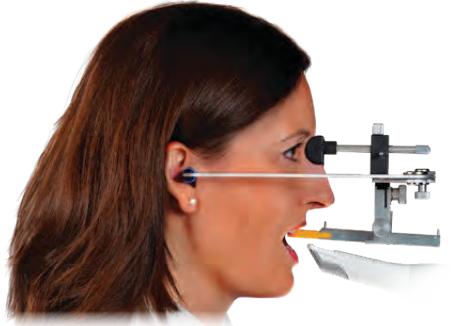
The digitizable bite fork carrier

Axioprisa allows the exact maxillary position of the patient to be recorded and digitally transferred using an intraoralscanner and analog transfer bow.

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Before Axioprisa, the patient models were in a virtual space, without an exact reference plane. Now, with the Axioprisa digital bite fork carrier and an oral scan, the upper jaw can be digitally assigned to the patient. The models are skull-related in the articulator thanks to Axioprisa and can be prepared for 3D printing in the AxioSnapMount software and sent to print providers such as Dreve, Team Ziereis, Anaxdent.

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*Export as STL-file for further use,
e.g. in Axiosnapmount or with a CAD software (3Shape, Exocad,...)*

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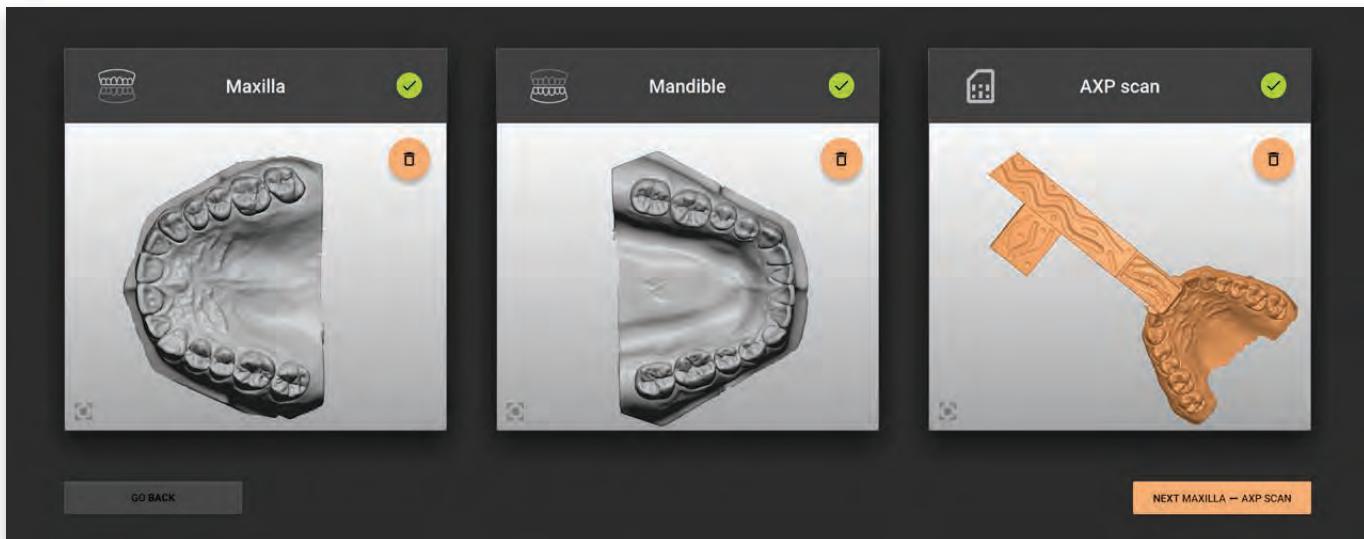
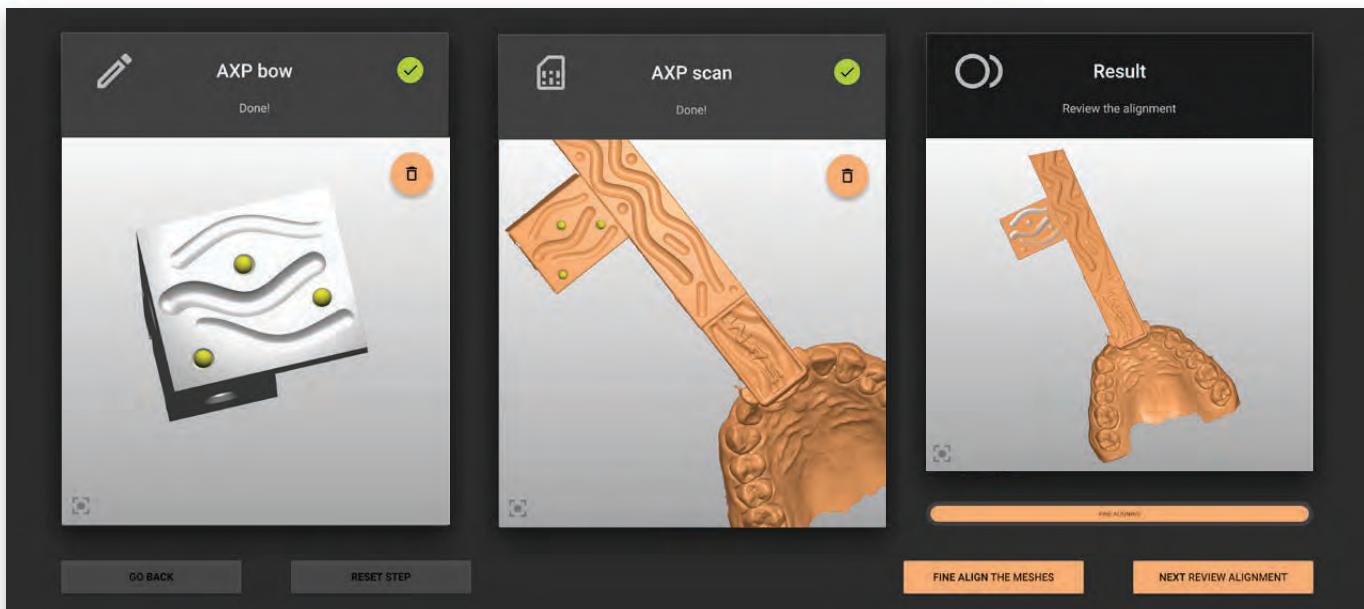
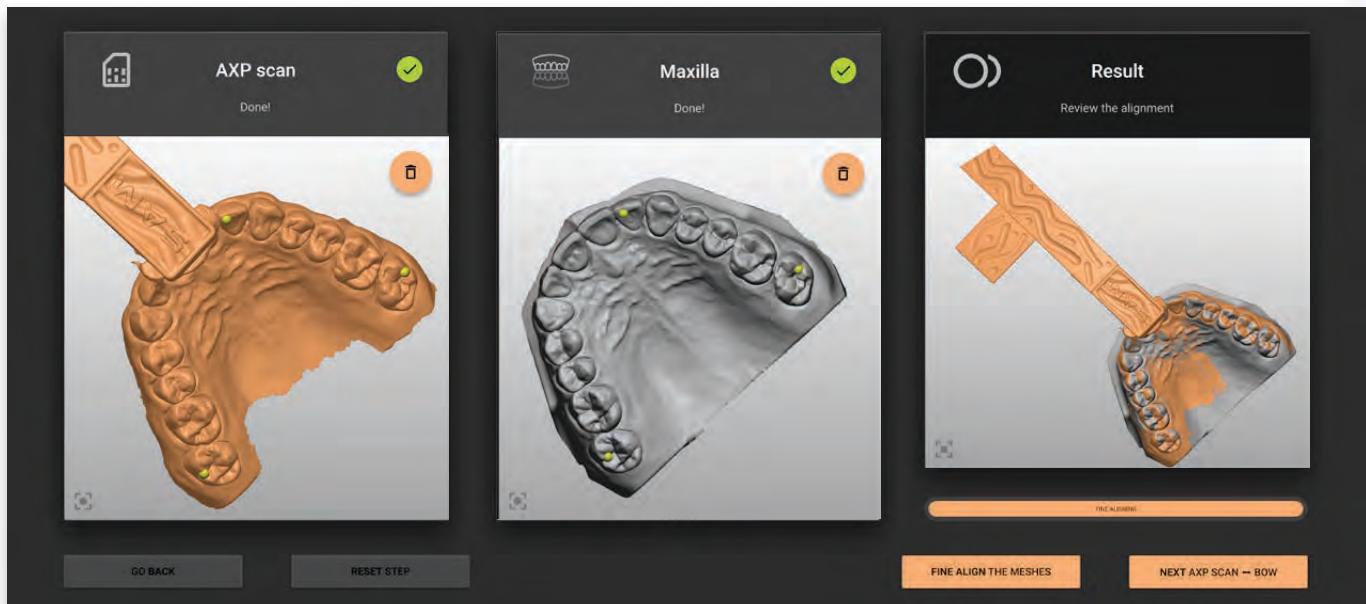
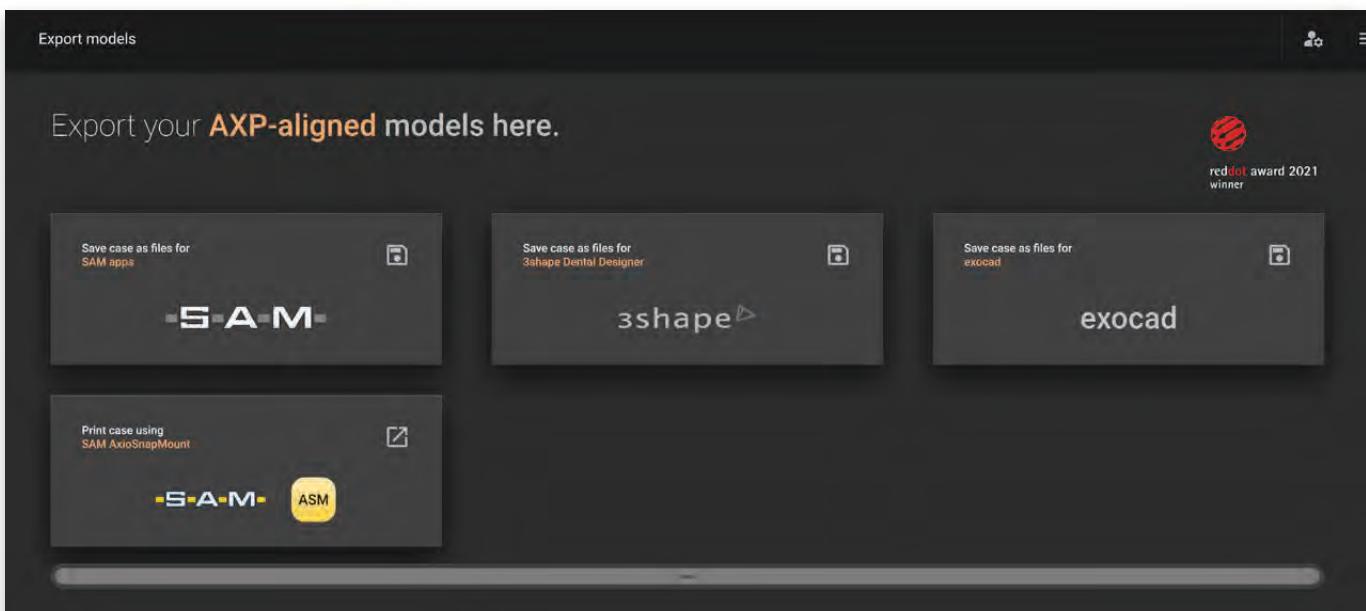
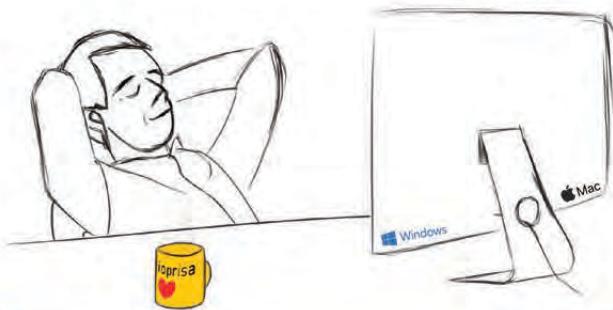
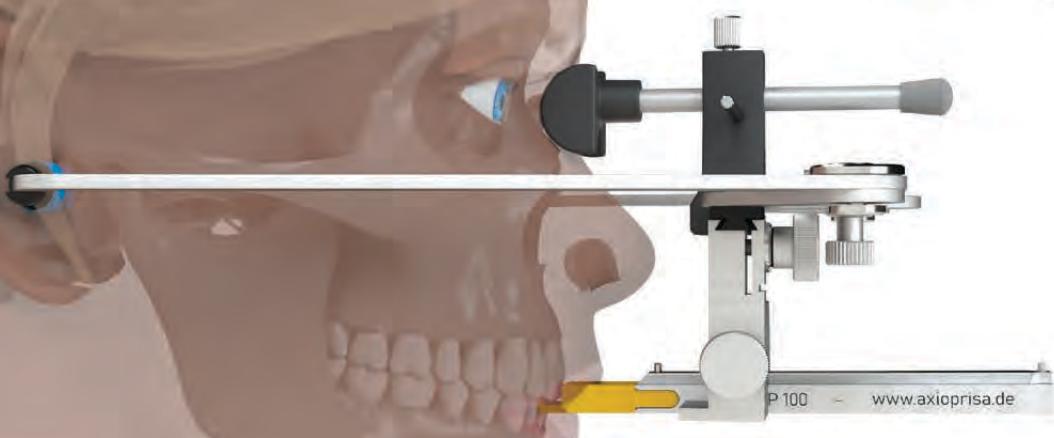
*Analog insertion of
3D models with matching
distance block and
mounting plate*

*or for quick plastering
of models with reduced
expansion and economic
material consumption*

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*digital model assembly in
the software*

*output of the STL file for
further use in other CAD
softwares*

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AxioSnapMount arose from the problem that until now you always had to articulate printed models with plaster or use a mounting clamp.

Now digital models can be mounted in the articulator in a reproducible and anatomically correct manner.

Using the specially developed computer program, the distances between the articulator and the model are calculated, the optimum construction elements and the digital STL models are automatically “married” with the model mounting plate attachment. For the first time, digital models can be printed “ready-to-use”, for use in a fully adjustable articulator.

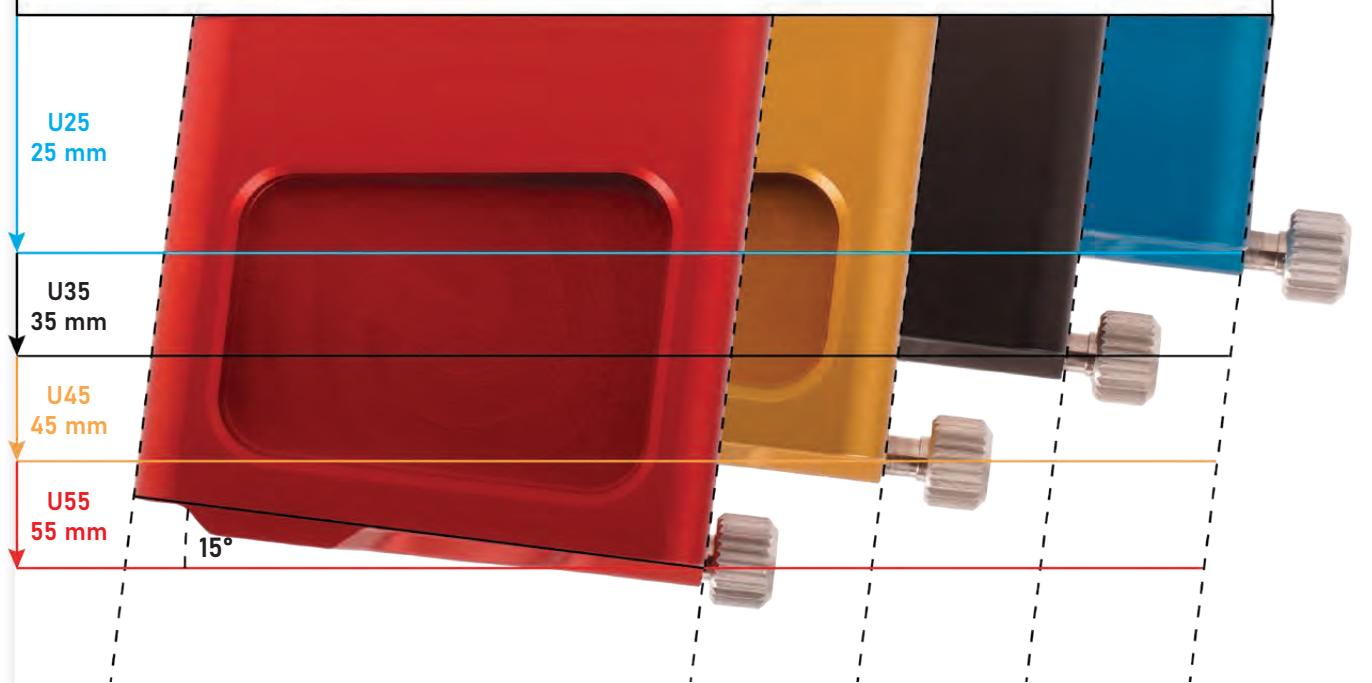
Different block sizes, as well as different fine-tuned mounting plates reduce the model height can thus save printing time, printing material and process labor time in the practice and laboratory.

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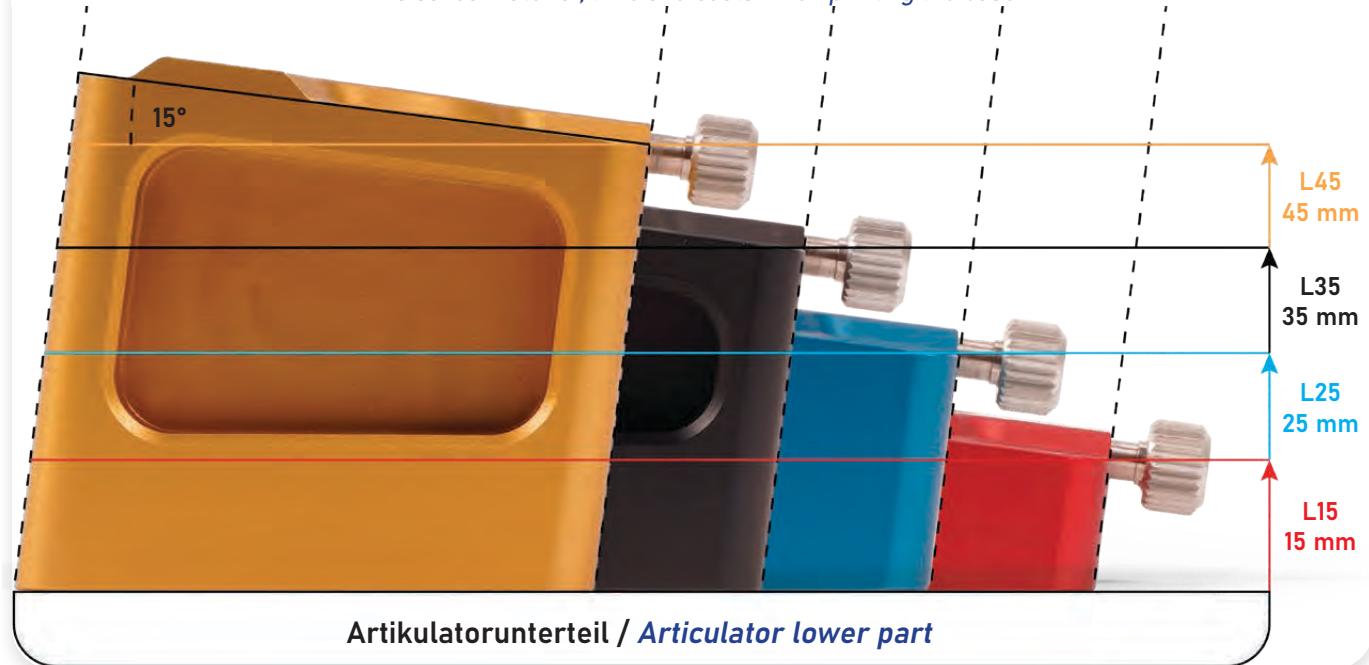
For each side 4 SAM distance blocks of different heights are available for upper and lower jaw models.

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*In combination with the mounting plates, the base height of the 3D models can be adjusted individually.
This saves material, time and costs when printing the base.*

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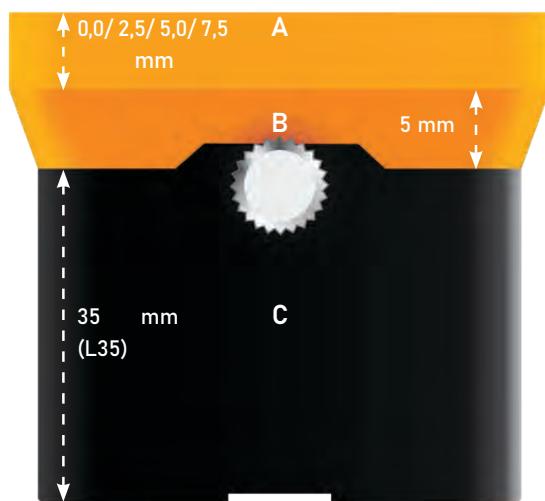
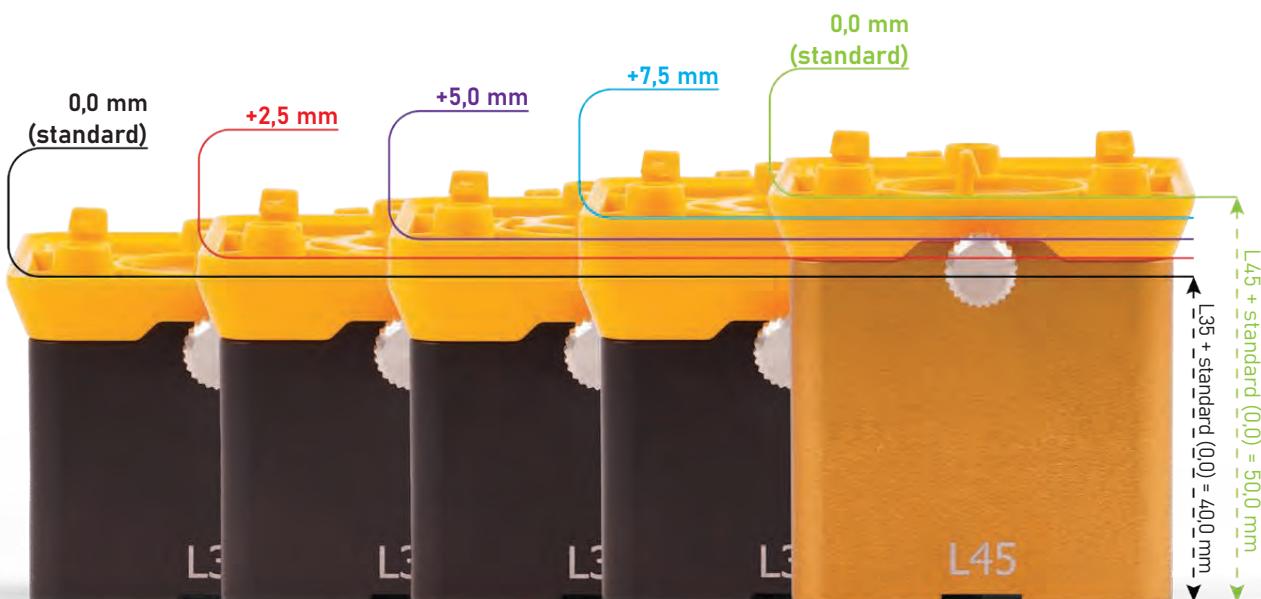
The distance bases each have a height difference of 10 mm. The four different mounting plates can be used to create additional intermediate heights.

These can be used for both the "Hybrid" and the "Analog mounting method".

- A Randleiste
Border height
- B Montageplattengrundhöhe
Mounting plate base height
- C Distanzsockel
Distance socket

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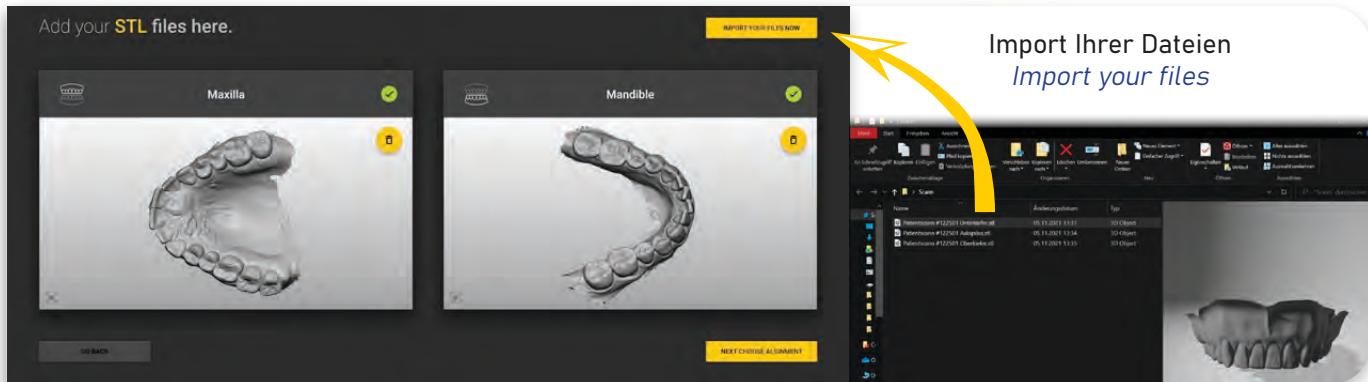
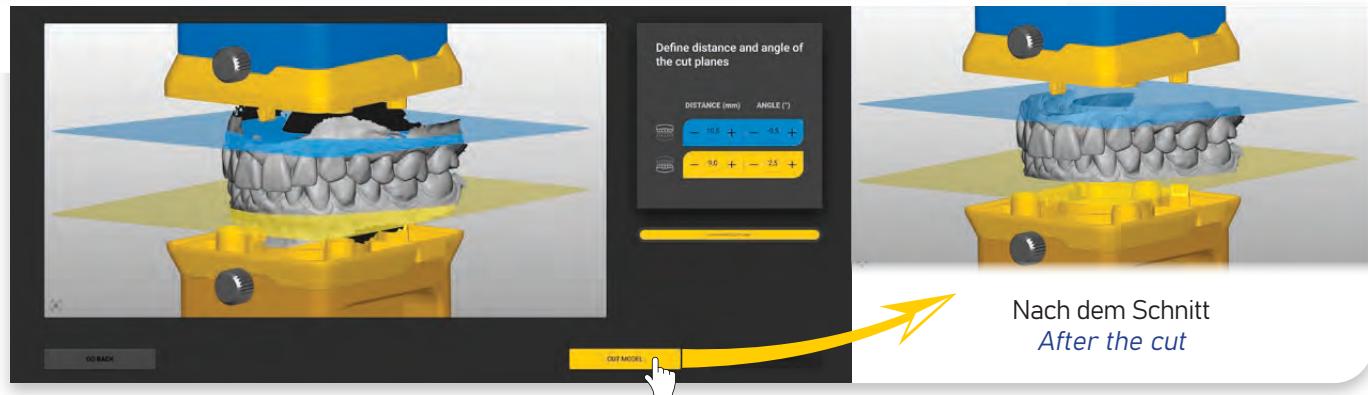
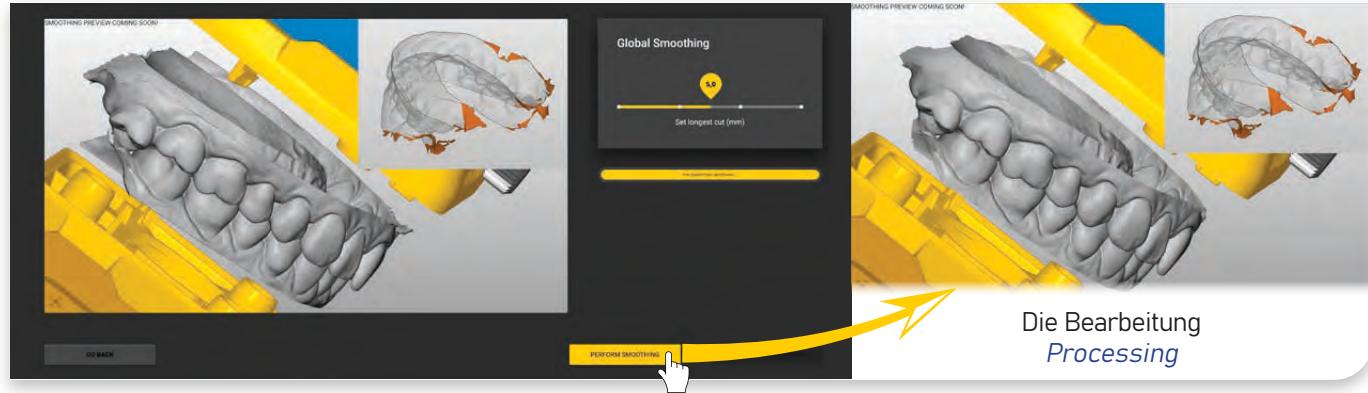
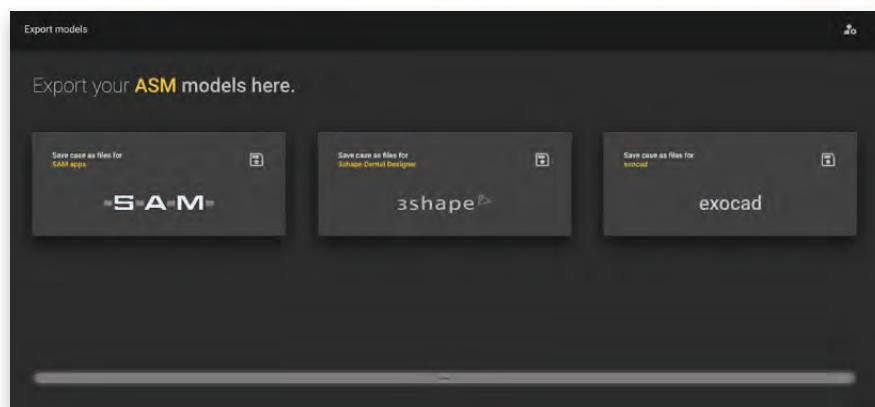
without software

Windows Kit with AXP software license for 24 months

Apple Kit with AXP software license for 24 months

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AXIOJIG TRANSFER FORK REFILL - XS
1 x bag with 10 disposable transfer forks

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SAM AXIOSNAPMOUNT PROFESSIONAL KIT

8X

Magnet N
REF ASP 580

REF ASM 120
100x

REF ASM 121
100x

REF ASM 122
100x

REF ASM 123
100x

Besteht aus:

4 x Oberkiefer-Distanzblöcke
je 1 x ASM 225 / ASM 235 / ASM 245 / ASM 255
4 x Unterkiefer-Distanzblöcke
je 1 x ASM 315 / ASM 325 / ASM 335 / ASM 345
1 x ASM Display (ASM 103)
8 x Magnete N (ASP 580) -(Je Block mit Magnet)
400 x Montageplatten (je 100x)

Includes:

*4 x Upper jaw distance blocks
1 each ASM 225 / ASM 235 / ASM 245 / ASM 255
4 x Lower jaw distance blocks
1x ASM 315 / ASM 325 / ASM 335 / ASM 345 each
1 x ASM Display (ASM 103)
8 x magnets N (ASP 580) -(Each block with magnet)
400 x mounting plates (100x each type)*

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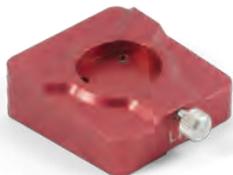
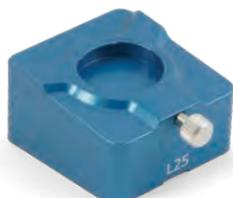
REF ASM 315

AXIOSNAPMOUNT L15
Distanzsocket XS - rot
Höhe: 15 mm

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ASM MOUNTING PLATE - STANDARD

20 Mounting plates yellow, for single use
Border height: standard

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ASM MOUNTING PLATE - PLUS 2,5

20 Mounting plates yellow, for single use
Border height: 2.5 mm

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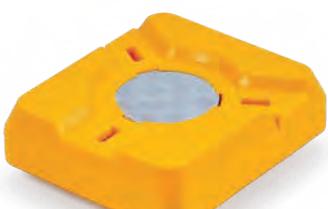
ASM MOUNTING PLATE - PLUS 5,0

20 Mounting plates yellow, for single use
Border height: 5.0 mm

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ASM MOUNTINGPLATE - PLUS 7,5

20 Mounting plates yellow, for single use
Border height: 7,5 mm

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ASM MOUNTING PLATE - COMBI PACK

20 Mounting plates, yellow, for single use
 Standard / 2.5 / 5.0 / 7.5

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1 x Gewindeschiebe, 2 x Fixierschrauben
 1 x Inbusschlüssel

CONVERSION SET - THREADED WASHER

1 x threaded washer, 2 x fixation screws
 1 x allen key

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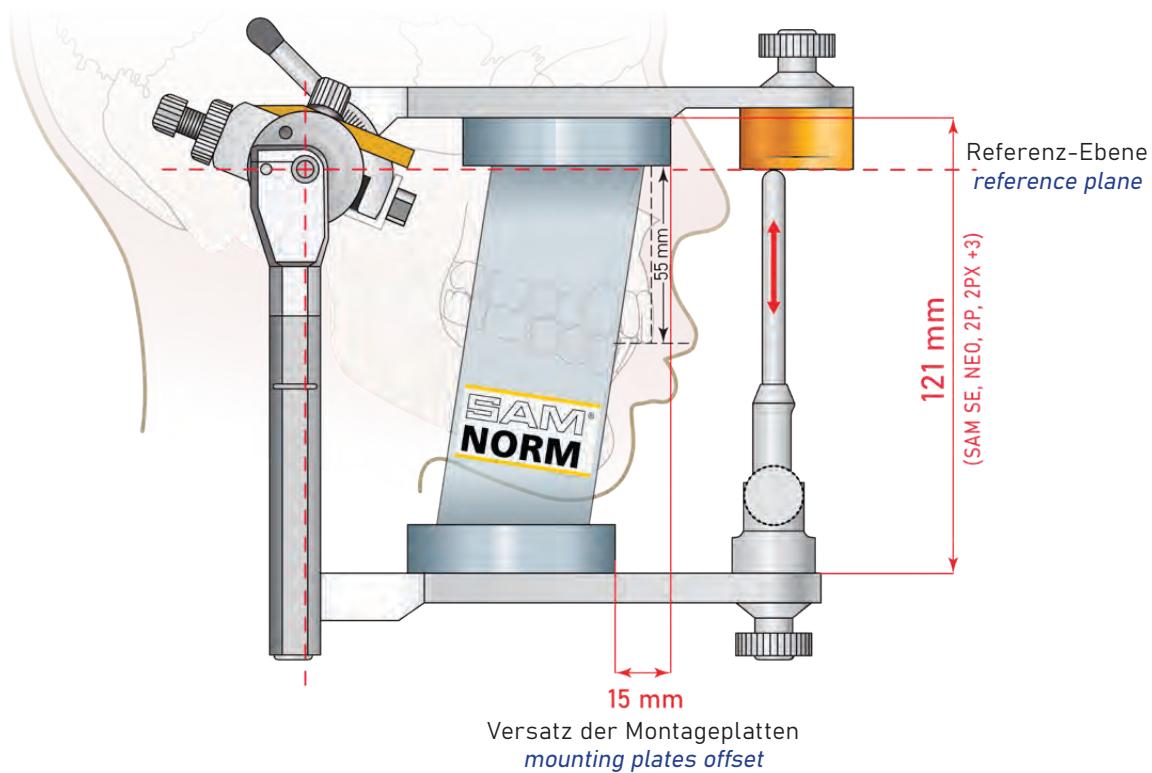
The SAM articulator system differs in many respects from the common and widely used articulators. Here are some of the unique selling points:

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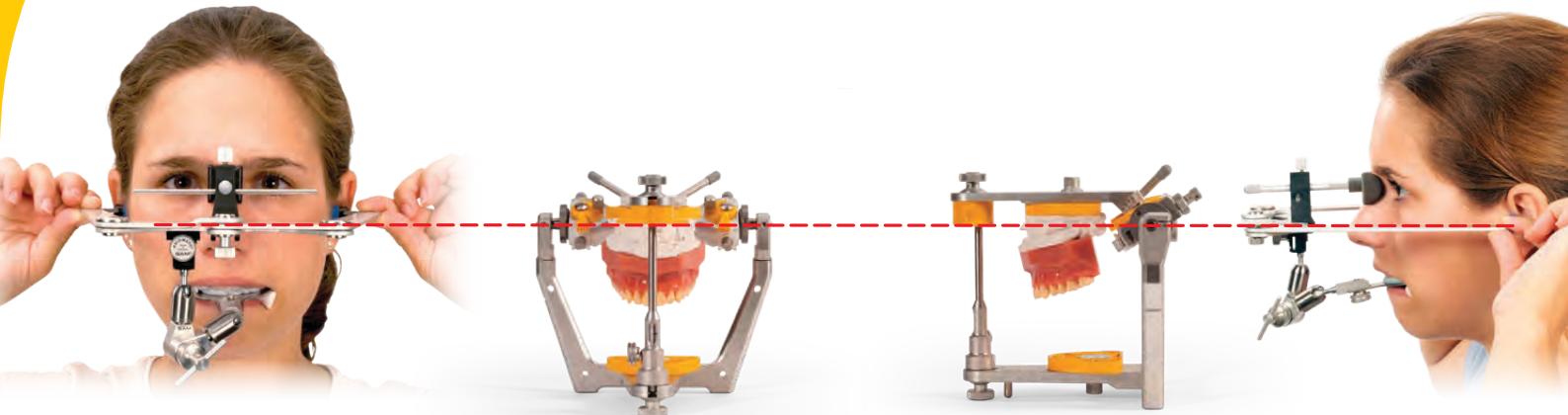
ANATOMICAL REFERENCE PLANE

The anatomical transfer of the patient situation into the articulator is one of the key points of the SAM system. The mandrels of the anatomical transfer arch, the hinge axis, the orbital point and the contact point of the incisal pin on the table are in line and parallel with the articulator upper

and lower part. A change in height in the vertical plane therefore does not cause any deflection of the incisal pin. Therefore, this error does not have to be compensated by a curved incisal pin or repositioning of the anterior guide plate.

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AXIS-ORBITAL PLANE

The used Reference Plane Porion orbital corresponds to the Frankfurt Horizontal and is correctly called Axis-Orbital Plane in the SAM system.

The SAM transfer sheet can be used to transfer the anatomically correct model position of the patient to an articulator.

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All articulators from SAM have the same height and are based on the same design principle.

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*All current articulators and SAM instruments are based on the construction height "P".
(P stands for prosthetic height = 121 mm between upper and lower part).*

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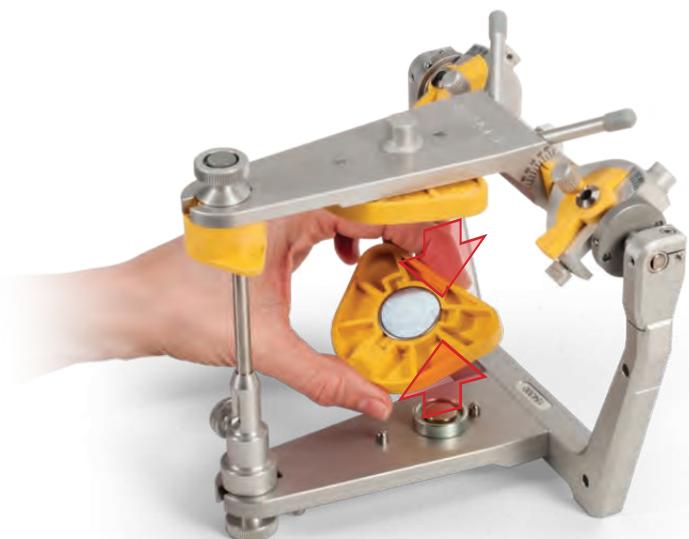
SAM articulators are Arcon articulators, the condylar balls are located, authentic to the temporomandibular joint on the articulator base, with a diameter of 10 mm.

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The distance between the center of the condylar ball and the arbor, for the transfer sheet, is standardized and measures 10 mm.

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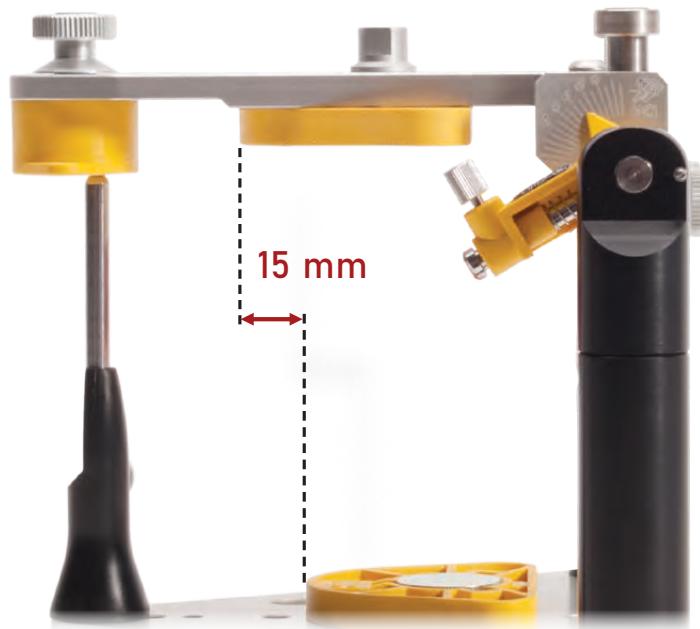
The identical position of the mounting mandrels enables exact mounting plate placement.

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MOUNTING PLATE OFFSET

Another special feature of the SAM system is the 15 mm offset of the mounting plates in the articulator, which takes into account the normal jaw relation. This distally offset mounting plate position facilitates the mounting of the mandibular model and less plaster is required. The model position appears more harmonious.

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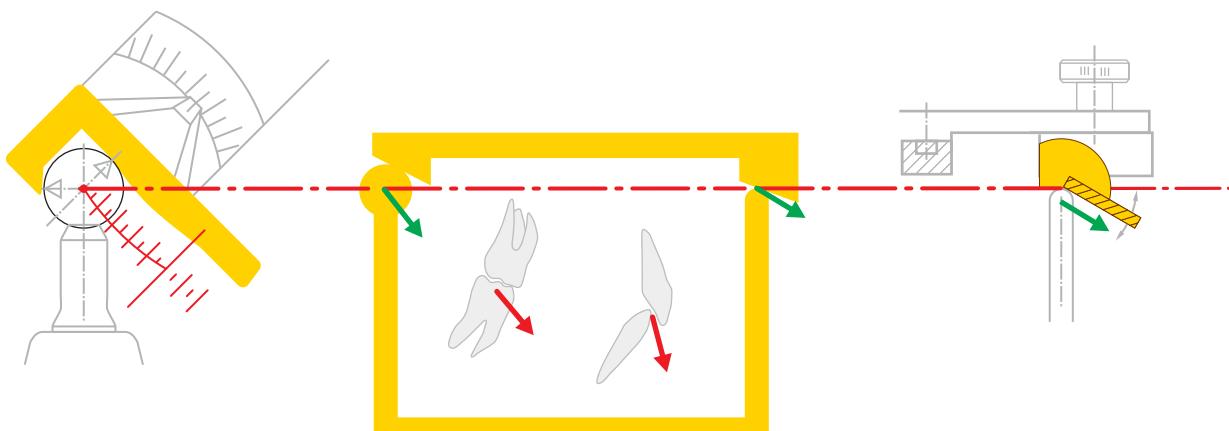
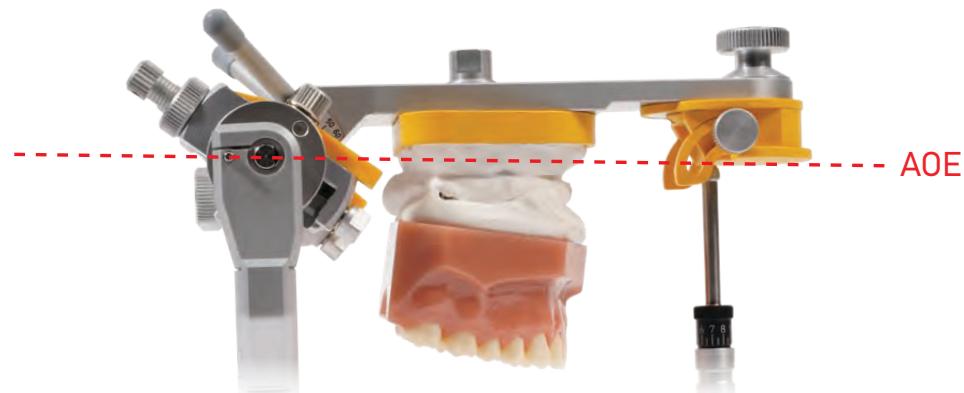
The red dot in the center of the lower member condylar element corresponds to a hinging axis rotation center of the mandibular condyles. Also known as "Hinge Axis".

Above the red dotted/dashed line is the axis-orbital plane (AOP) The upper anterior incisal table surface along with the posteri- or condylar element centers of rotation are on the same horizontal plane.

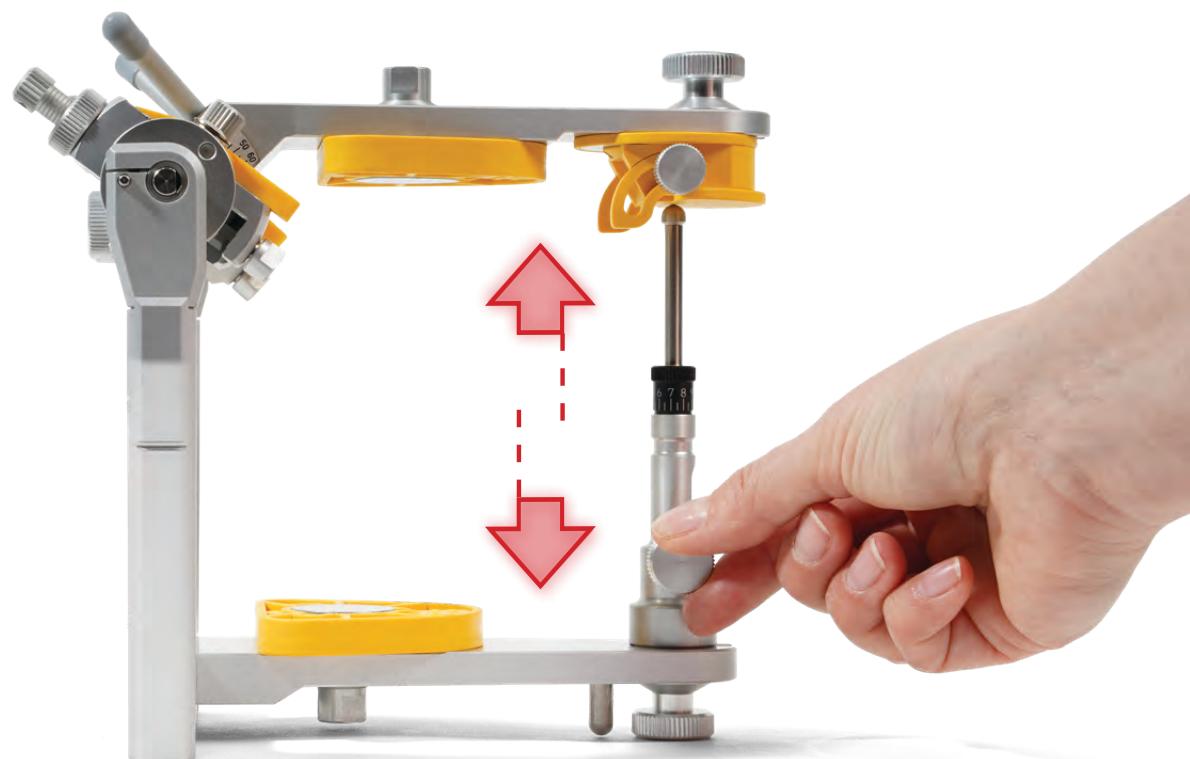
The lower incisal pin in contact with the upper incisal guidance table is used to simulate the relationship of upper and lower incisal edges of the patient's teeth. Thus promoting normal functional patterns.

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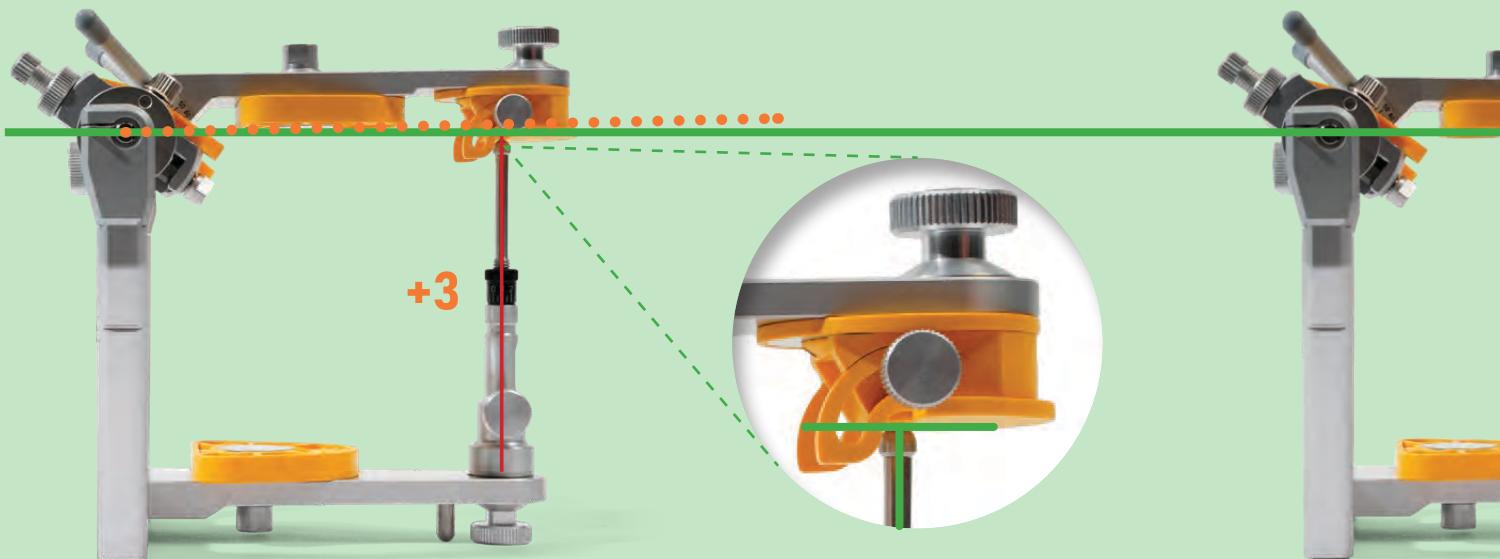
The Axis-Orbital Plane is established as a reference plane by the support surface of the incisal table and the mounting mandrel. Due to the incisal table, the condylar guide surface is located in the articulator upper part.

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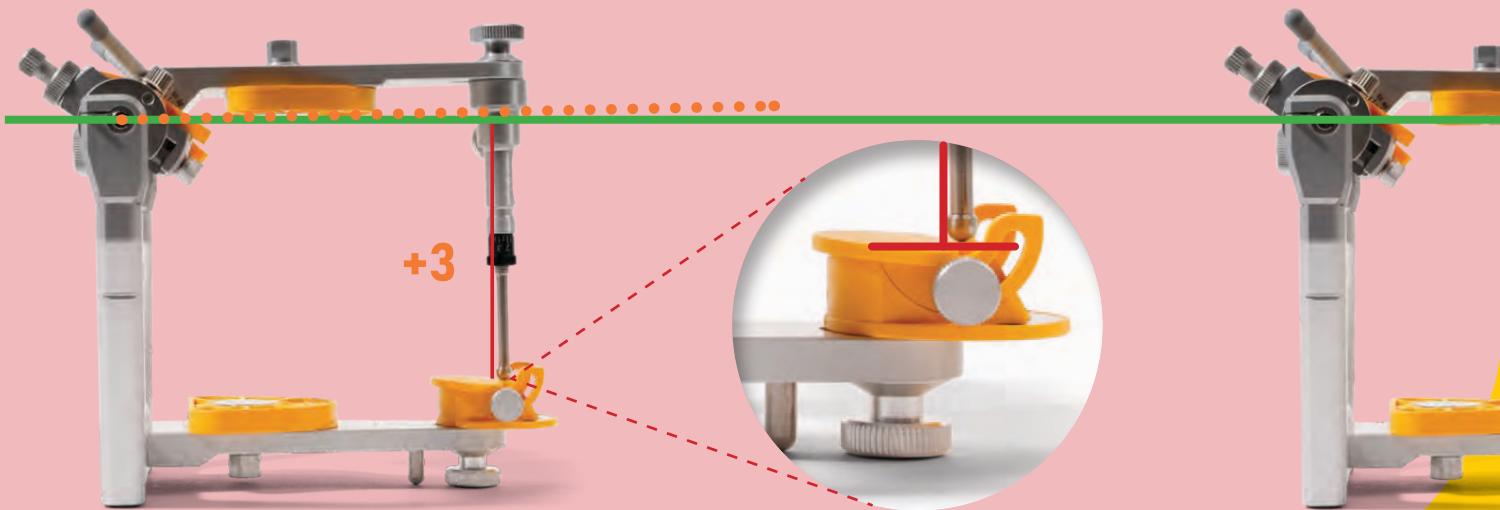
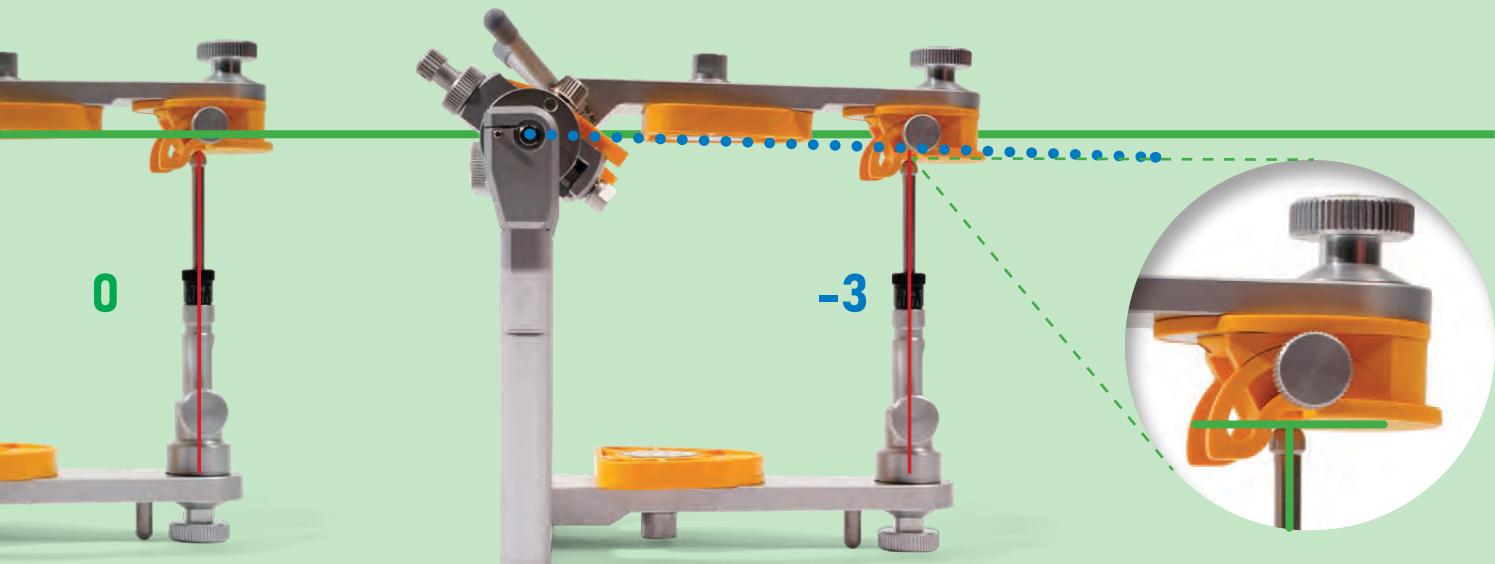
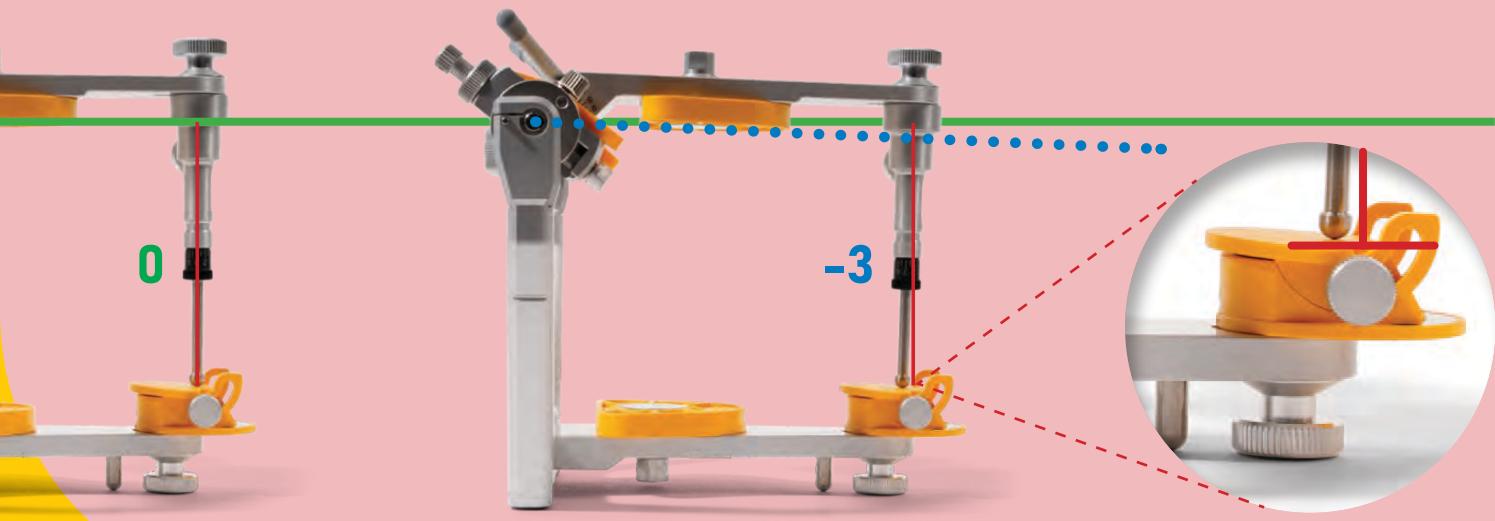
Precise

If the incisal pin is located in the lower part of the articulator, as recommended, the pin position and its contact point on the incisal table remain the same during raising or lowering.

Inaccurate

The incisal pin is in the upper part of the articulator, the vertical dimension can be changed when the incisal pin is raised and lowered. During a dynamic movement, the pin position and its contact point shifts.

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THE MODULAR EXTENSION OF THE FULLY ADJUSTABLE ARTICULATORS

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Many additional instruments and accessories complement and round off the SAM system uniquely in all directions. No other manufacturer offers so many helpful customization options to help the everyday use for their user. For almost all areas, there are small or large, analogue or digital useful

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tools, which are always matched to the articulator standard and can be interchanged. For users of our SAM transfer bow, we offer a wide range of transfer stands for transferring the patient models, also in articulators of other dental manufacturers.

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Due to the sophisticated and normalized design, the SAM instruments are compatible with each other.

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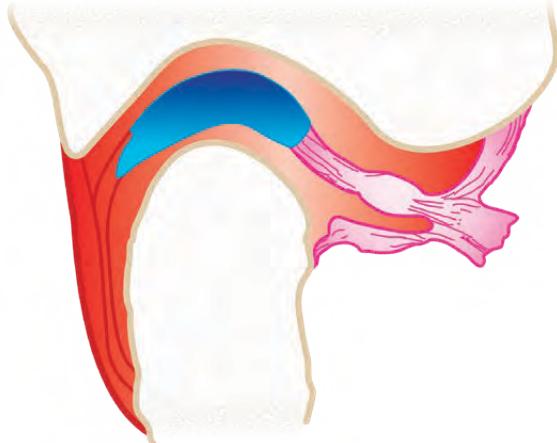
All articulators made by SAM are Arcon instruments.

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Using the Arcon principle of the SAM® articulator mandibular joint movements of the patient can authentically simulated. The condyle ball is located on the lower part of the articulator and moves in the condylar housing of the upper member.

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The SAM articulators and instruments can be equipped with the Axiosplit synchronization system for maximum precision, the robust MPS magnet system or the classic screw mounting plate, depending on the customer's requirements.

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These end-user trainings take place at the Center for Dental Education in Gauting near Munich. For international guests we offer courses in English. If you want another language, we can organize an interpreter for extra charge.

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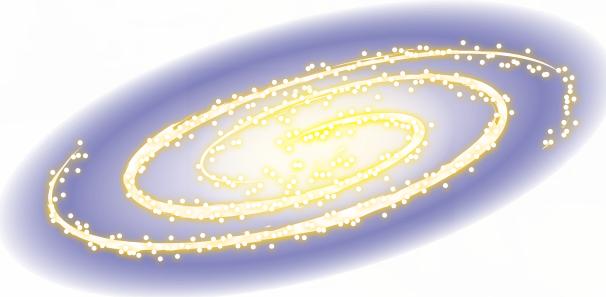
The user training courses can be held in your laboratory, your custom practice, or in suitable premises in your vicinity, in German and English. Arrange an appointment at an early stage!

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These user training courses take place in your laboratory, your custom practice or in suitable premises in your vicinity, within Europe. If you would like a more specialized training, we will be happy to book one of our "external specialists" who are very familiar with the SAM system and can provide you with detailed, case-related explanations.

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You can book user training for SAM products worldwide. We work together with renowned and SAM experienced external specialists. Are you interested in a customized course that is designed to meet your needs? Then visit an event with your colleagues. We arrange an individual training at

your desired location and answer all questions you want to know and learn about the SAM system. The course duration is 3.5 days. The user course is held in German and English. For other languages, an interpreter can be hired upon request. Experience a unique training.

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Are you interested in an international SAM course? Contact SAM directly or ask your local dealer.

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For all our rooms, the use of a music system, individual seating, a lectern and a flip chart are available on request for your event. Our 330 m² light-flooded cafeteria offers many comfort options.

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The Seminar room offers for the participants 24 Labor desks with Frasaco Phantom heads, technical motors, contra-angle Angel piece connections with air and water supply and dry suction.

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Our auditorium with concert acoustics and terrace-shaped auditorium offers the best view from all seats. With its own control room and perfect equipment, any type of event is possible.

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The fully visible operating room, which is separated from the lecture hall, provides the ideal setting for training and demonstrations in the field of dentistry.

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We are happy to take over the individual planning of the catering for you, from the classic buffet to the international flying buffet as well as coffee & cake.

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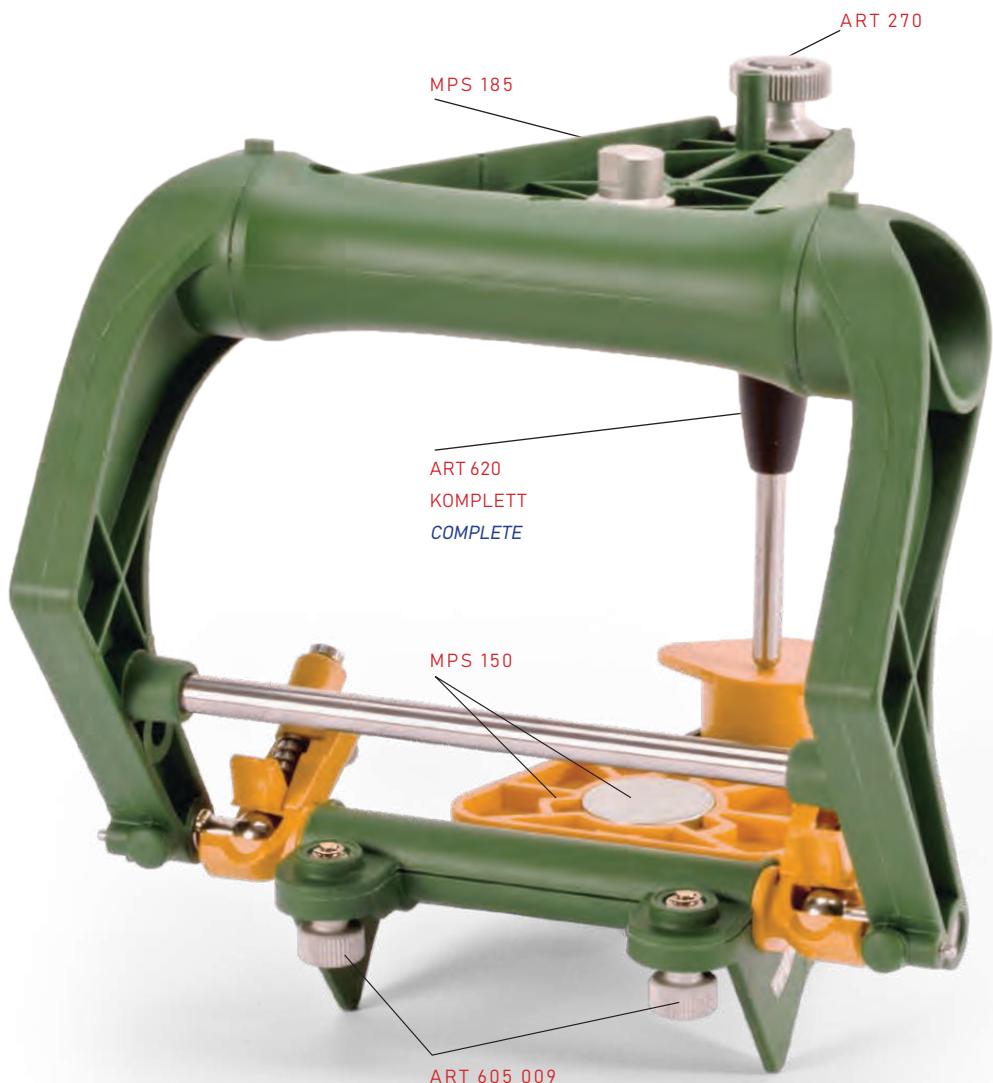
"THE LIGHT AVERAGE VALUE ARTICULATOR"

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for systems with screw type mounting plate
and suitable for AXIOSPLIT (page 134)

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SAM SE articulator with integrated curvature 1 and bennett guides with normal sideshift and a flat incisal table (ART 275).

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The SAM SE is valued as an inexpensive, lightweight and yet very stable articulator.

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All vital ergonomic features have been incorporated into the design of the SAM SE Articulator.

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The SAM SE has been designed using fiber glassreinforced space age plastics that are light, durable, and as precise as the aluminium versions. All vital ergonomic features have been incorporated into the design of the new SAM SE Articulator.

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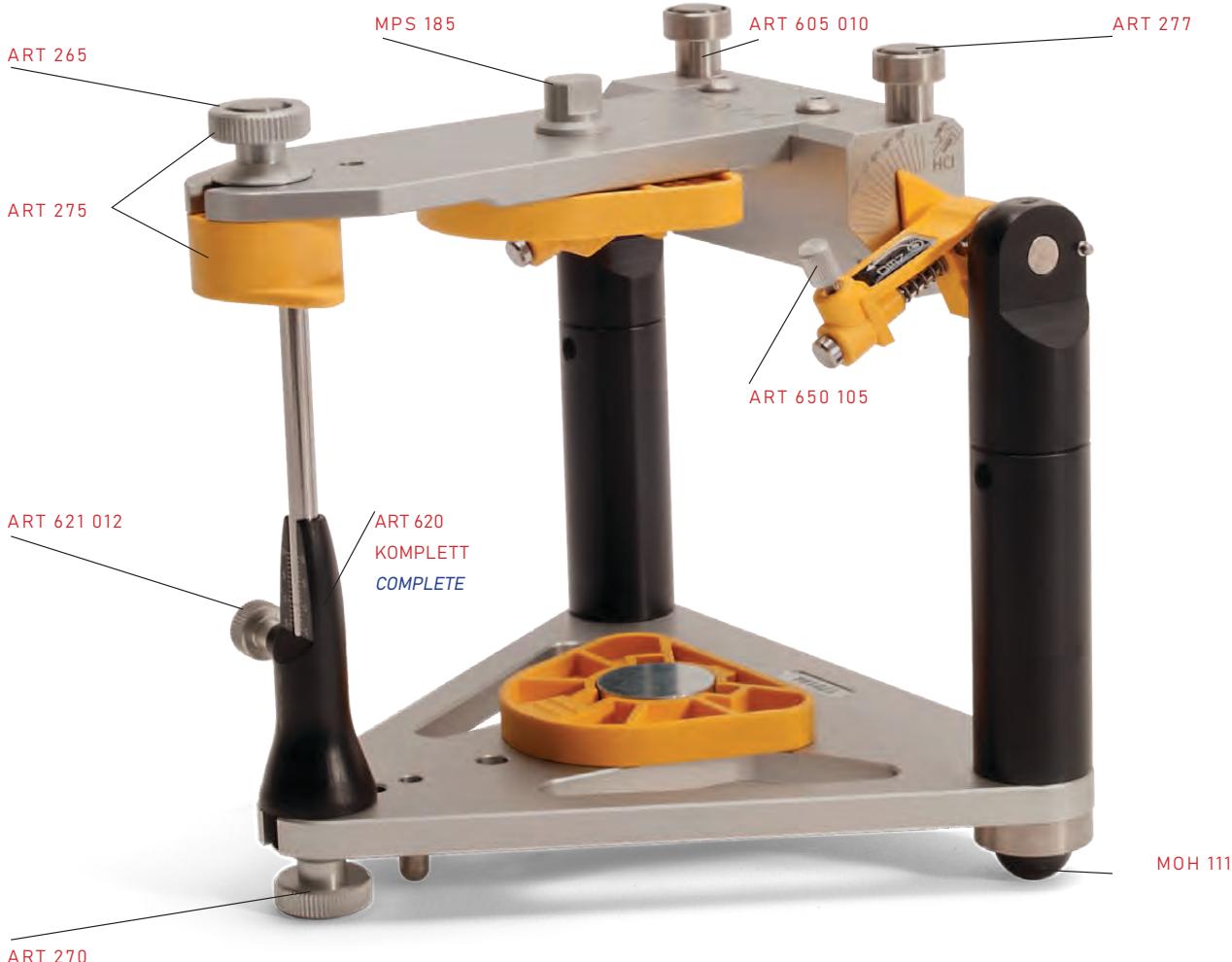
The SAM SE is suitable for everyday use in all disciplines of dentistry. Based on our experience in diagnostics and therapy, we know the minimum requirements underlying this device: a model positioning in line with the skull and an adjustable inclination of the condylar path with curvature 1, a Bennett guide that corresponds to the green SAM insert.

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The SAM standard allows the use of SAM additional devices. A wide range of accessories are available for the articulator.

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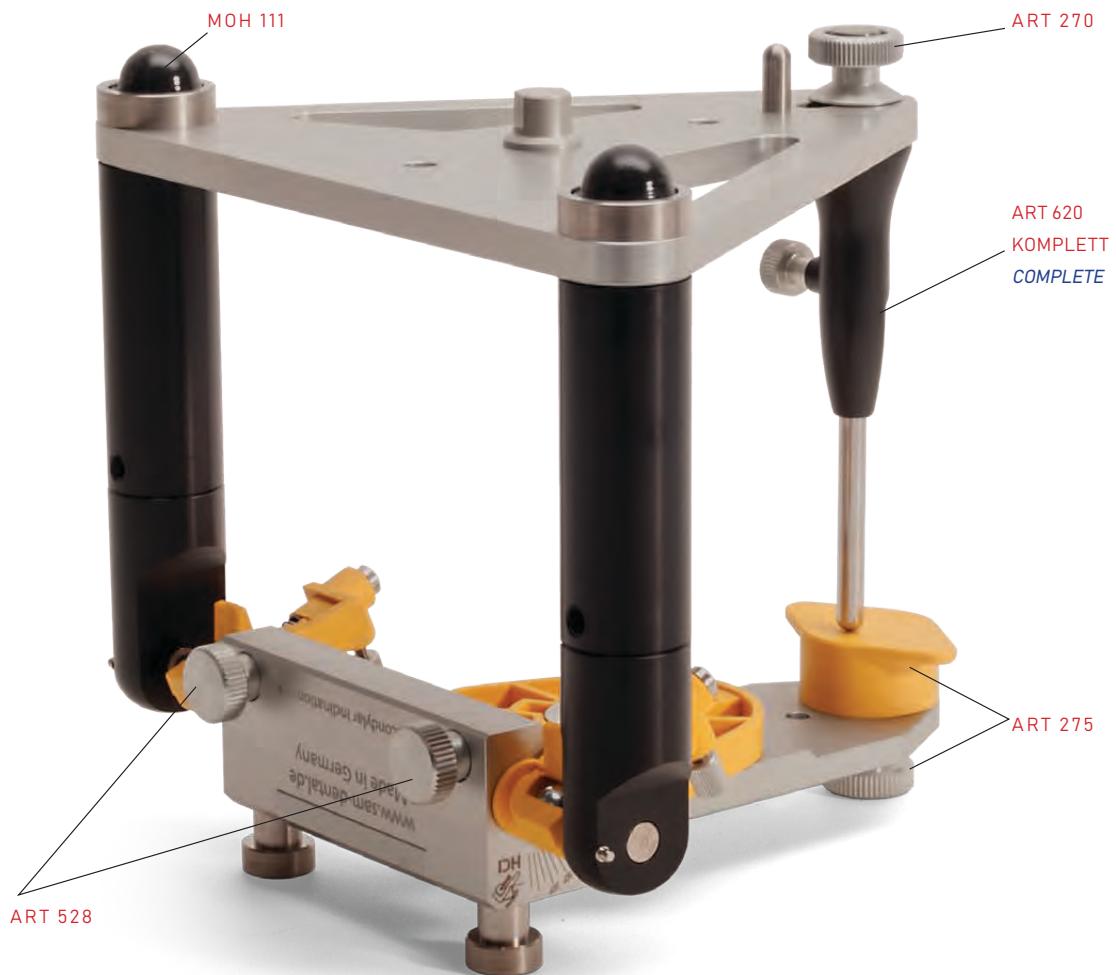
*for systems with screw type mounting plate
and suitable for AXIOSPLIT (page 134)*

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SAM NEO articulator with integrated curvature 1 and bennett guides with normal sideshift and a flat incisal table (ART 275)

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The SAM NEO is an inexpensive and very stable articulator. All the minimum requirements on which an articulator is based are combined in this unit.

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In the SAM NEO articulator, the articular path inclination (angle) can be set, and a statistical middle value based on an anatomical model has been used for the articular path curvature. In the SAM articulator, this corresponds to condylar path insert 1 (white, with low curvature).

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The SAM standard allows the use of SAM additional devices. A wide range of accessories is available for the SAM NEO articulator.

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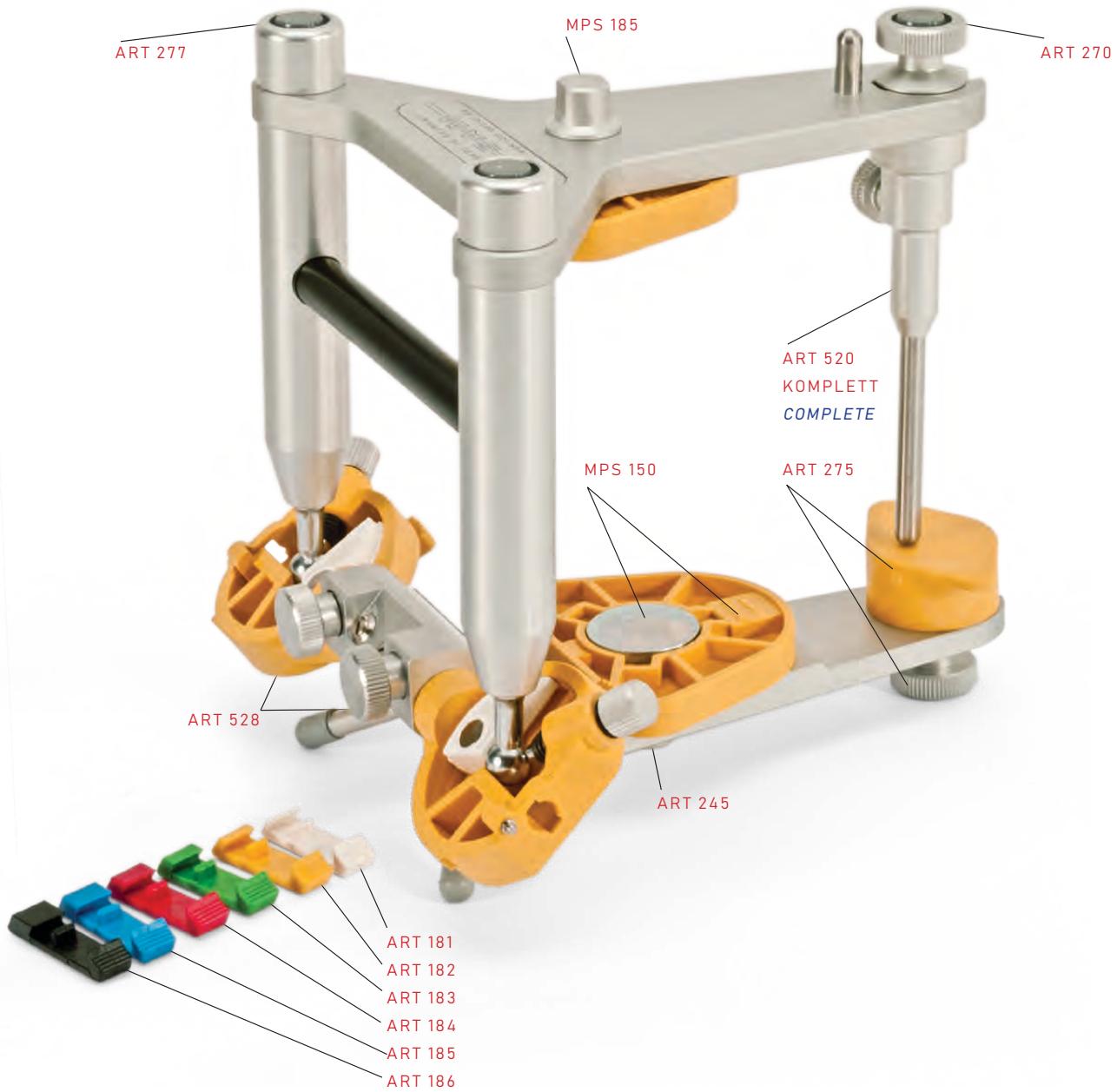
The support of the articulator upper part in the open state is very convenient. Tilting supports provide a better view of the dental work, support pin and tilting supports can be ordered optionally, for the articulator.

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*with condylar housing curvature 1 (ART 225), with one pair each of the bennett guides white (ART 150) and
green (ART 176), incisal table flat (ART 275)*

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The articulator developed for training and for wax-up has accompanied dentists and dental technicians in daily use for decades.

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A characteristic feature of the SAM 2P is the simple centric locking by means of a spring bolt.

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The well-designed functions, appealing appearance and high stability made this articulator world famous.

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Bennett angle and condylar track inclination are individually adjustable, and the condylar housings and Bennett guide inserts can be easily replaced.

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Of course, there is a wide range of accessories for the articulator for full adjustment and for all areas of dentistry & dental technology.

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CONDYLAR HOUSING CURVATURE 1

with integrated curvatures (shallow curvature);

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CONDYLAR HOUSING CURVATURE 2

with integrated curvatures (middle curvature);

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CONDYLAR HOUSING CURVATURE 3

with integrated curvatures (steep curvature);

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every pair includes:

centric lock thumb screw and bennett guides white (ART 150) for SAM 2C and SAM 2P

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CENTRIC LOCK THUMB SCREW

*centric locking thumb screw assembly for yellow condylar housing
for SAM® 2C + 2P*

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- grün (ART 176)
- blau (ART 177)
- rot (ART 178)

für SAM 2C + SAM 2P

BENNETT GUIDES SET I
includes one pair of bennett guides

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PROTRUSION INSERT SET
includes 2 pieces of each protrusion insert

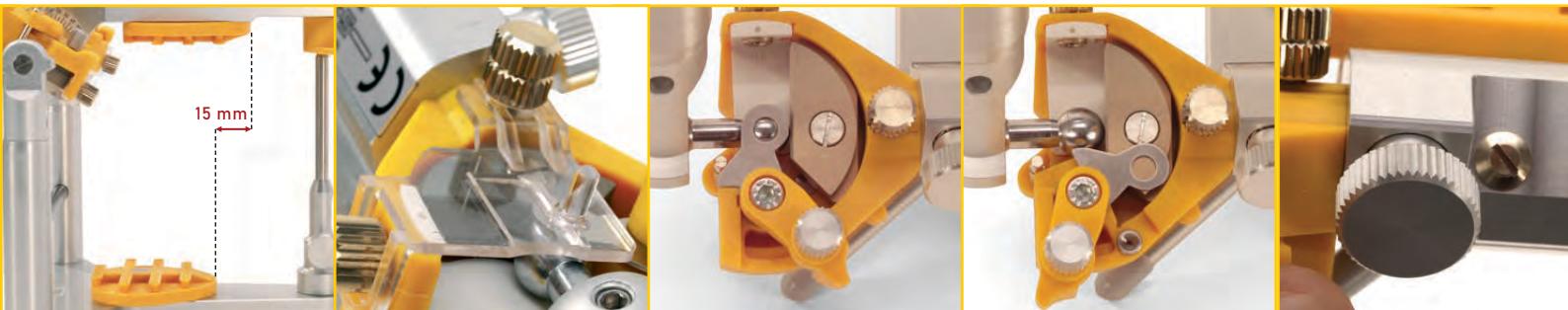
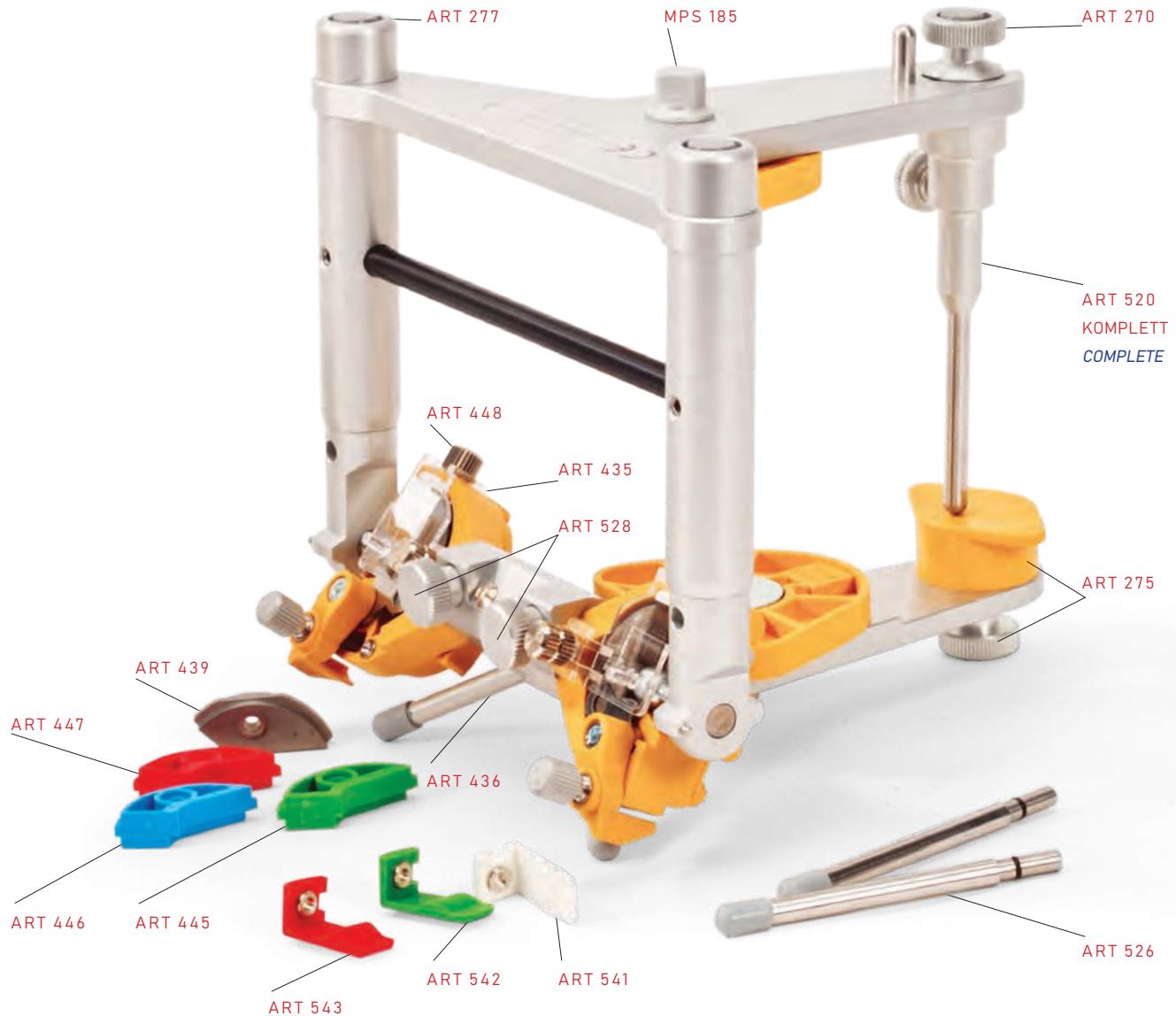
- | | | |
|----------|-----------|------|
| - white | (ART 181) | 1 mm |
| - yellow | (ART 182) | 2 mm |
| - green | (ART 183) | 3 mm |
| - red | (ART 184) | 4 mm |
| - blue | (ART 185) | 5 mm |
| - black | (ART 186) | 6 mm |

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SAM 2PX articulator with condylar inserts curvature 1 white (ART 541), with one pair each of the bennett guides straight, metal (ART 439) and green (ART 445), incisal table flat (ART 275), vertical support rod (ART 525), and tilt support rods (ART 526)

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The 2PX is a SAM articulator variant with many useful features. The Arcon articulator has a practical centric lock (leaf spring) with unrestricted opening rotation.

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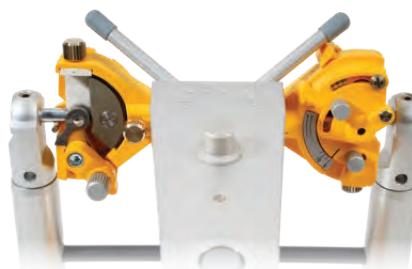
The articulator upper part can be supported in the open state. The tilting supports provide a better view of the dental work.

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The option to adjust the protrusion up to 5.5 mm is already built in.

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Due to the transparent condylar housing covers, the articulator upper and lower parts remain connected during movements, even when the centric lock is open.

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Bennett angle and condylar web inclination are individually adjustable, the condylar web curvatures can be easily replaced, as can the Bennett inserts.

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CONDYLAR HOUSING COVERS FOR SAM 2PX

(1 pair) with thumb screws (ART 436). Used to hold upper and lower member of the articulator together during eccentric movement

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(includes one pair of protrusion insert each)

- Bennett guide green (ART 445)
- Bennett guide red (ART 446)
- Bennett guide blue (ART 447)

(includes one pair of Bennett guides each)

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*Engraving
individual engraved to your specifications*

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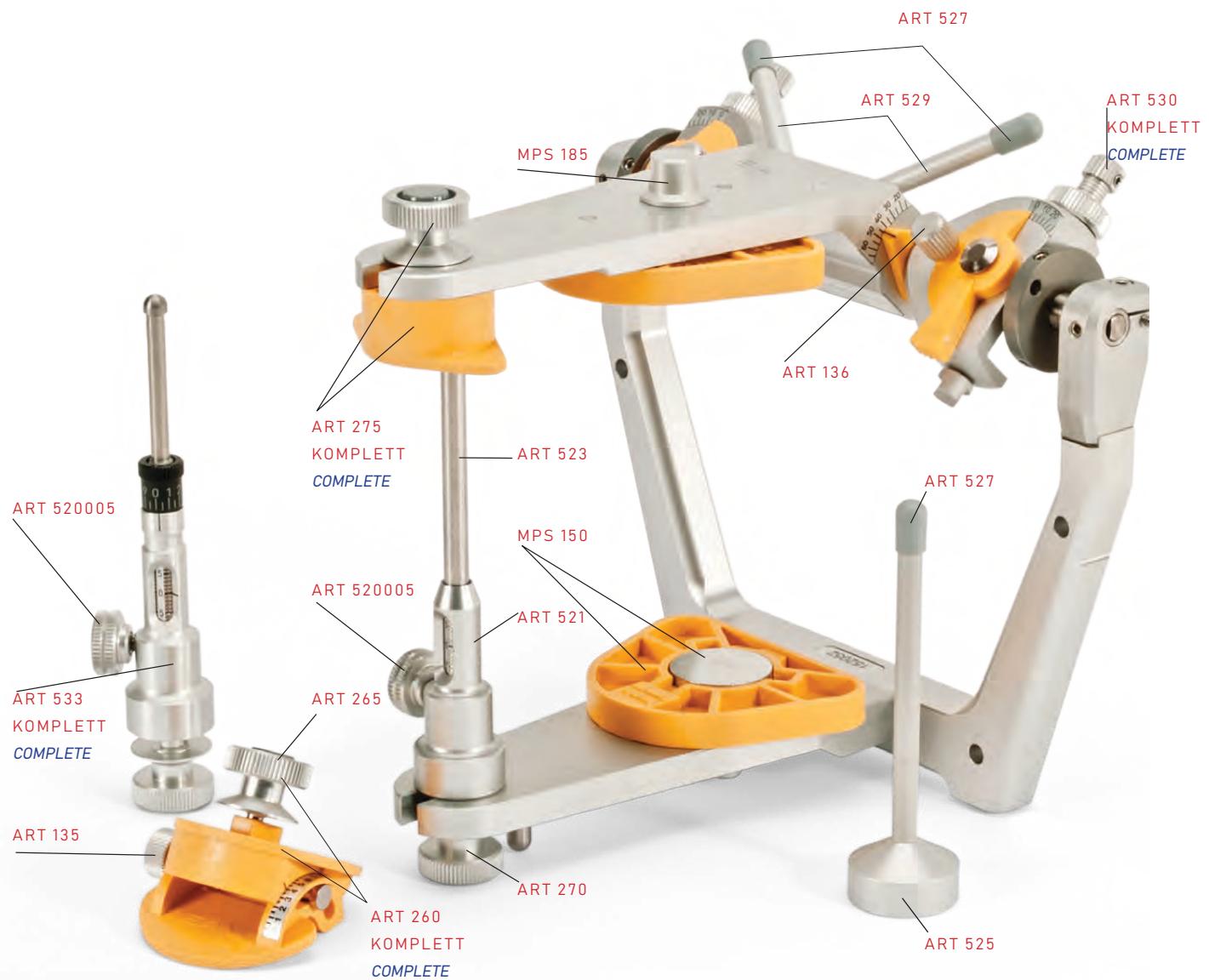
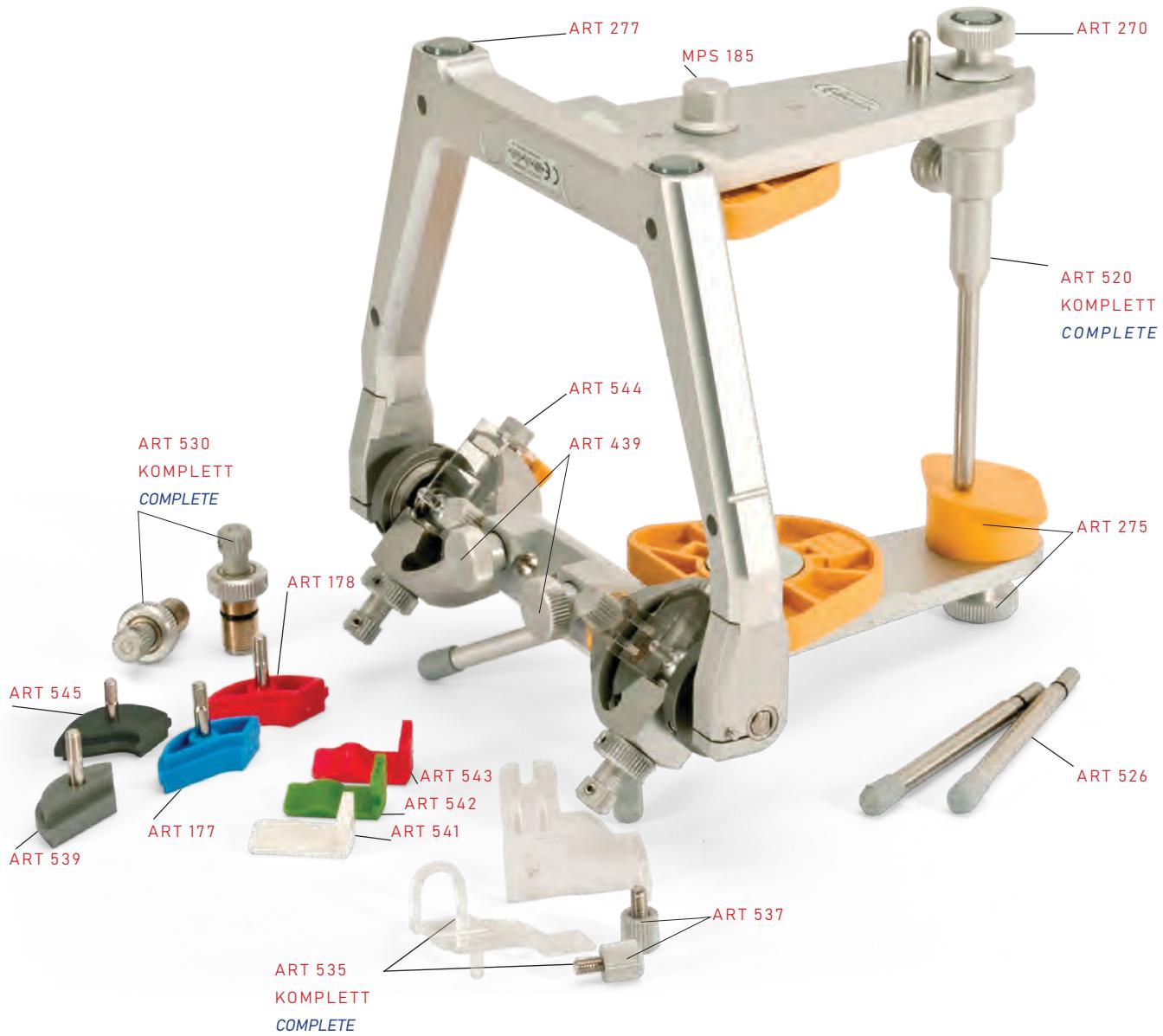
This service has a processing time of approx. 3-5 working days. The place for the engraving depends on the size of the device. Contact us and ask for the possibilities.

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The SAM 3 is considered as an articulator for high demands. It stands with the highest precision and functionality.

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Like all SAM articulators, it complies with the Arcon principle and the SAM standard. Due to the perfect centric locking (triple ball bearing conical locking), the SAM 3 achieves highest precision with unrestricted opening rotation.

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Its beautiful appearance and high material quality make working with this instrument so pleasant and efficient.

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The interchangeable Bennett guides with different sideshift can remain in the unit without sacrificing centric precision. The condylar track curvatures are easily interchangeable.

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Due to the transparent condylar housing covers, the articulator upper and lower parts remain connected during movements, even when the centric lock is open. Optionally, protrusion and retrusion are continuously adjustable.

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Of course, there is a wide range of accessories for the articulator for full adjustment.

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SAM® 3 Artikulator Professional**REF ART 550**

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*SAM 3 articulator with vertical support rod (ART 525), tilt support rods (ART 526), curvature set (ART 540),
Bennett guides set II (ART 547), incisal pin IIIPM micro adjustable (ART 533), incisal table,adjustable (ART 260), protrusion / retrusion screws (ART 530),*

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PROTRUSION / RETRUSION SCREWS,

(2 pieces) adjustable protrusion 0 - 6 mm, retrusion 0 - 2 mm, with yellow wrench (ART 510) and hex wrench for SAM 3

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CONDYLAR HOUSING COVERS FOR SAM 3

(1 pair) left and right sides with thumb screws (ART 537) used to hold upper and lower member of the articulator together during eccentric movements

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BENNETT AND CONDYLAR INSERT SET 3

includes condylar insert set (ART 540) = one pair of each curvature 1 white (ART 541), curvature 2 green (ART 542) and curvature 3 red (ART 543) and one yellow wrench (ART 510) and bennett guide set II (ART 547) = one pair of each - green metal (ART 545), blue (ART 177) and red (ART 178) and protrusive guide rings according to Prof. A. Gutowski (ART 581)

Artikulator Edition “Prof. Dr. A. Gutowski” Articulator Edition “Prof. Dr. A. Gutowski”

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SAM 3 articulator standard, AXIOSPLIT articulator kit N, condylar insert curvature 2 green, one pair of each of the metal bennett guides and green metal , protrusive guide rings, incisal table, adjustable and micro adjustable incisal pin IIIPM. A special compilation for the courses of Gutowski and for daily work!

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includes SAM 3 articulator standard, AXIOSPLIT articulator kit N, condylar trajectory inserts Curvatur 2 green, with one pair each of Bennett guides metal and green metal, pre-glide guide, incisal table adjustable, micro adjustable incisal pin IIIPM, incisal table - flat, AXIOQUICK transfer bow kit - AX with 6 + 2 hygiene protection caps, telescopic bite fork support, AXIOSPLIT mounting plates - rund (18 St), transparent transport box, hex keys 1.5mm, 2.0mm & 2.5mm. A special set for the J. Zvolanek courses and for daily practice!

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SAM 3 articulator with vertical support rod (ART 525), tilt support rods (ART 526), curvature set (ART 540), Bennett guides set II (ART 547), incisal pin IIIPM micro adjustable (ART 533), incisal table,adjustable (ART 260), protrusion / retrusion screws (ART 530),

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MOUNTING PLATES SCREW ADAPTER II
*used to attach models from the P-height in the 15 mm increased
SAM devices or articulators or SAM 2C AXIOSPLIT
mounted casts on a SAM 2P, SAM 2PX or SAM 3 articulator*

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AXIOSPLIT ADAPTER,
*used to attach models from the P-height in the 15 mm increased SAM
devices or articulators or SAM 2C AXIOSPLIT mounted casts on a SAM
2P, SAM 2PX or SAM 3 articulator - delivery form with magnet N*

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AXIOSPLIT REFERENCE CENTRIC INSTRUMENT II

Special feature: the magnet in the upper part of the articulator can be pulled out. This allows a quick check of the model assembly (split cast check).

Cast mounting device with quick removable thumb screw/magnet assembly for comparison with following articulators SAM 2P, SAM 2PX, and SAM 3, master check and master incisal pin are not included

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Laboratory upper and lower cast mounting instrument, eliminates gypsum contamination of articulators, same height as SAM SE, SAM NEO, SAM 2P, SAM 2PX, and SAM 3

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INCISAL TABLE, ADJUSTABLE

with changeable rotation and inclination angles plus numeric scale for all SAM articulators

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INCISAL TABLE, FLAT

yellow, acrylic monomer does not adhere to table surface, table height 16 mm for all SAM articulators

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INCISAL PIN IIIP, COMPLETE

with numeric scale, slightly offset for better viewing for SAM 2P, SAM 2PX, and SAM 3

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INCISAL PIN IIIP REVERSE, COMPLETE

with numeric scale (red) slightly offset for better viewing for SAM 2P, SAM 2PX, and SAM 3 for use of the incisal pin in the upper part

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VERTICAL SUPPORT ROD

extension rod for supporting the unfolded articulator upper part for all hinged SAM devices

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TILT SUPPORT RODS

(2 pieces) used to protect the condylar housings when the articulator is tilted backwards for SAM NEO, 2PX and SAM 3 and articulation devices

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INCISAL PIN IIIPM, MICRO ADJUSTABLE, COMPLETE

with micro scale for reading into 0,05 mm steps

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SPLITCAST VERIFICATION DEVICE

for the production of an individual plaster base for checking the articulator synchronisation, includes plastic base part with splitcast and rubber ring for all articulators and mounting plates

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CHECKING GAUGE

for articulators, from metal for SAM P-norm for screw type systems only

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Select the mounting plate system that you want to use in the articulator.

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Select the type of bite fork support you want to use with the transfer sheet.

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*Transfer fork support acrylic XL***REF** ATB 397Transferstand SE

*Transfer stand SE***REF** ART 598Transparenter Transportkoffer für
Artikulator

*Transparent transport pack for
articulator*

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REF ATB 398

Transferstand AX
Transfer stand AX

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Magnetic block for transfer fork support, mps

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Magnetic block for telescopic fork support, screw type

REF ATB 336

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Telescopic transfer fork support

REF ATB 303 & ATB 394

AXIOQUICK III Transferbogen & Nasionstütze AX - S

AXIOQUICK transfer bow III & Nasion relator AX - S

REF ATB 240

Hygieneschutzkappen (3x6 St.)

Hygienic earpiece caps (3x6 pc.)

REF ATB 398

Transferstand AX

Transfer stand AX

REF ART 180

Protrusionseinsätze Sortiment

Protrusion insert set

REF ART 170

Bennett-Führungen Sortiment I

Bennett guides set I

REF MOH 560

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Mounting stand with 30 flexible rods

REF ART 599

Stapelbox

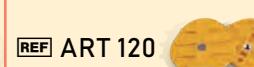
Carry case

Already built-in parts for the artikulators (ART 300 & ART 315M):

- incisal pin IIIPM (ART 533) - (Page 82)
- Incisal table - adjustable (ART 260) - (Page 81)
- Bennett guides straight (ART 150)

SAM 2PX KIT

	Schraubgewinde Typ / <i>Screw type</i>		MPS Version / <i>MPS type</i>	
	[REF] ART 401K	[REF] ART 402K	[REF] ART 416MK	[REF] ART 417MK
	[REF] ART 120 	[REF] ART 120 	[REF] MPS 150 	[REF] MPS 150 
[REF] ART 400				
[REF] ART 415M				
[REF] ATB 305				
[REF] ATB 395				
[REF] ATB 303				
[REF] ATB 240				
[REF] ATB 394				
[REF] ATB 334				
[REF] ATB 398				
[REF] ART 599				

PRODUKTÜBERSICHT / PRODUCT OVERVIEW

REF ART 400

SAM 2PX Artikulator

*SAM 2PX articulator***REF** ART 415M

SAM 2PX Artikulator

*SAM 2PX articulator***REF** ART 120

20+2 gelbe Schraubmontageplatten

*20+2 yellow screw mounting plates***REF** MPS 150

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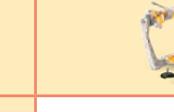
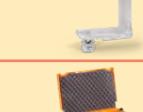
Already built-in parts for the artikulators (ART 400 & ART 415M):

- Incisal pin IIIPM (ART 533)
- Incisal table - adjustable (ART 260)
- white condylar inserts from (ART 440)
- Bennett guides straight

SAM 3 KIT

Schraubgewinde Typ / <i>Screw type</i>		MPS Version / <i>MPS type</i>	
[REF] ART 501K	[REF] ART 502K	[REF] ART 516MK	[REF] ART 517MK
[REF] ART 120 	[REF] ART 120 	[REF] MPS 150 	[REF] MPS 150 
[REF] ART 500 			
[REF] ART 515M			
[REF] ATB 305 			
[REF] ATB 395			
[REF] ATB 303 			
[REF] ATB 240 			
[REF] ATB 394 			
[REF] ATB 334 			
[REF] ATB 398 			
[REF] ART 599 			

PRODUKTÜBERSICHT / PRODUCT OVERVIEW

REF ART 500

SAM 3 Artikulator

SAM 3 articulator

REF ART 515M

SAM 3 Artikulator

SAM 3 articulator

REF ART 120

20+2 gelbe Schraubmontageplatten

20+2 yellow screw mounting plates

REF MPS 150

20+2 gelbe Magnetmontageplatten

20+2 yellow magnetic mounting plates

REF ATB 305

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Transfer fork assembly I-NT complete

REF ATB 395

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Transfer fork assembly AX complete

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Already built-in parts for the artikulators (ART 550 & ART 565M):

- Incisal pin IIIPM (ART 533)
- Incisal table - adjustable (ART 260)
- white condylar inserts from (ART 541)
- Bennett guides 3 - straight - metal (ART 539)

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OCCLUSAL PLANE INDICATOR 3D

transparent calotte, vertically and sagittally adjustable, tiltable and swiveling, with adjustable angle scale (0° - 45°), also includes: asymmetric incisal pin IIPA (ART 361), for better top view during prosthetic work, suitable for all SAM articulators.

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OCCLUSAL PLANE INDICATOR SET

includes flat (ART 366) and curved (ART 580) occlusion plane stages according to Spees & Wilson curve, vertically and sagittally adjustable with millimeter scale, inclination adjustable with angle scale 0° - 45° , as set-up aids in total prosthetics for model mounting of phantom models without transfer bow, for all SAM articulators

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OCCLUSAL PLANE INDICATOR SET JK

includes flat occlusion-plane measuring tables (ART 366), curved (ART 580) according to Spee & Wilson curve and a flat table (ART 366JK) for the KOIS bite insert, vertical and sagittal adjustable with millimeter scale, inclination adjustable with angle scale 0° - 45°, as Setting up aids in total prosthetics, for model assembly of phantom models without transfer bow, for all SAM articulators

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Schaufelgröße: L40 x B60 x H20

ANTERIOR TEETH EVALUATOR IDEA

for recording upper anterior teeth impression related to axis orbital plane for all SAM articulators

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AXIOSPLINT KIT GS

according to Prof. Dr. A. Gutowski and ZTM J. Stegmaier - transparent curved template for tooth set-up, with one-toggle joint clamping fixation and incisal pin GS, for SAM Axiosplit system; for fast and easy production of occlusal splints of light-curing composites and semi-permanent splints

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*See an example use of the
Axiosplint as video.*

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STUART PLANE INDICATOR BY DT CLAUDIO NANNINI

incisor setup aid to be used for determine the correct angle of the incisors according with the hinge axis, depending on the skeletal class includes guidance block with lateral movement possibility ($\pm 7,5$ mm) and three adjustable sagital pins (80 - 120mm) with different angled incisal adjustable needles (± 10 mm)

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AXIOQUICK® TRANSFER BOW KIT III – NT

Includes AXIOQUICK III transfer bow (ATB 303), transfer fork assembly I-NT (ATB 305), nasion relator AX (ATB 394), and six hygienic earpiece caps.

The perfect and cost-effective yet safe and torsion-free connection of bite fork and bite fork carrier.

AX

Transferbogen Kit III *Transfer bow kit III*

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AXIOQUICK® TRANSFER BOW KIT III - AX

Includes transfer bow AXIOQUICK III (ATB 303), bite fork assembly - AX AXIOMATIC (ATB 395), nasion support (ATB 394) and six hygienic protection caps.

A safe and torsion-free connection of bite fork and bite fork carrier is achieved by a maximum of technical perfection, e.g. by using a roller bearing.

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ORBITAL REFERENCE POINTER

used for facebow transfer with anterior reference point of your choice

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HYGIENIC EARPIECE CAPS

(200 pieces) blue disposable protective caps for the black ear olives on the Axioquick transfer bow and Axiograph III flag bow. Provides more comfort and facilitates positioning.

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AXIOQUICK ANATOMIC TRANSFER BOW III *without accessories*

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TRANSFER FORK I-NT, COMPLETE
with transfer fork NT and NT-clamp

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NASION RELATOR JSM

*universal nasion support according to ZT. Josef Schweiger
for tapping different reference points: FH, PH, AT or CP
and/or for alignment to the face midline*

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TRANSFER FORK NT

*two-piece, composed of transfer fork AX (ATB 396), used in transfer fork
assembly AX and transfer fork adapter (ATB 326), for use in AXIOQUICK
I-NT (ATB 305) and AXIOGRAPH*

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TRANSFER FORK ADAPTER NT

connects transfer fork AX (ATB 396) to transfer fork NT

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NASION RELATOR AX - S

with soft nasion support, interpupillary line leveling rod, for all SAM transfer bows

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TRANSFER FORK ASSEMBLY AX - AXIOMATIC

with a single secure universal clamping mechanism and removable transfer fork AX

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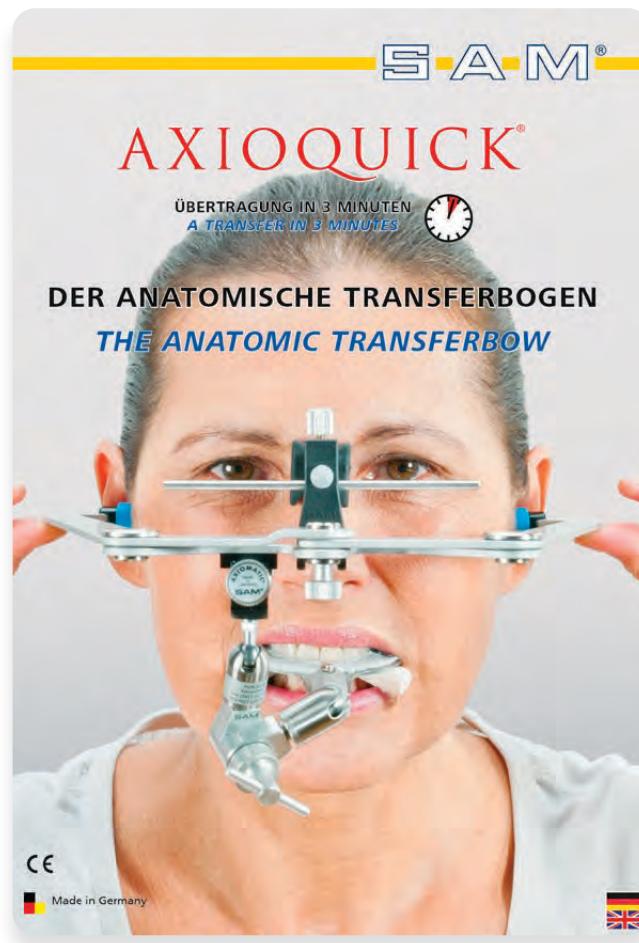
TRANSFER FORK AX

for SAM transfer fork assembly, removable for transportation and to reattach securely, sterilizable

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THE NEW JAW MOVEMENT REGISTRATION SYSTEM WITH OPTICAL SENSOR TECHNOLOGY.

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The AQR OPTIC is based on Zebris JMA Optic system and is a jaw motion registration system with optical sensor technology.

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It consists of:

- a handy and lightweight headset with cameras and processors
- small and lightweight mandibular sensor with magnetic holder
- two lower jaw attachment clutches (occlusal and paraocclusal)
- other accessories
- packed in high quality aluminum case

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Position and condyle movement can be detected with high accuracy. The field of application ranges from the creation of dental restorations from a functional point of view to the planning, documentation and control of stomatognathic rehabilitation.

The system can either be operated via a USB interface or wirelessly via WLAN.

Via a standardized XML export, real patient data or the setting values of the articulator can be transmitted to external systems.

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PARA-OCCCLUSAL ATTACHMENT FOR LOWER JAW SENSOR

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ALSO INCLUDES

USB cable, USB power supply, basic software (incl. articulator and XML data export), USB stick with 3 licenses + WLAN code, quick start guide, obligatory annual fee for the software maintenance contract, carrying case (SAM)

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3D ULTRASOUND MEASUREMENT FOR DIAGNOSIS AND THERAPY

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AXIOQUICK RECORDER II

anatomic transfer bow with ultrasonic receivers, nasion relator (ATB 394), lower recording bow with ultrasonic transmitters, porion ear pieces (one pair), neck band, velcro band, reference point marker, pre-determined hinge axis indicator, occlusal clutch (AQR 223), paraocclusal clutch (AXO 222), interface, power supply, foot control, USB cable, Laptop with pre-installed software (32bit) included (220V), supplied in a carrying case

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AXIOQUICK RECORDER II - AXO

It is an additional instrument for the mechanical axiograph. The AQR 100 electronic registration system with ultrasonic transmitters and receiver sensors can be integrated on the axiograph. On request, we will be pleased to build the ultrasonic sensor system onto your axiograph. Includes head sensor system with ultrasonic receivers, contra-angle handpiece for attachment to the axiograph, mandibular sensor system with ultrasonic transmitters, porion scanning pins (one pair), neck rubber, vertex tape, reference marker, hinge axis pin, occlusal tray (AQR 223), paraocclusal tray (AXO 222), interface case, power supply, foot switch, USB cable and Laptop with pre-installed software (32bit) included (220V).

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PARAOCCCLUSAL CLUTCH WITH Z-MAGNETIC ATTACHMENT

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OCCCLUSAL CLUTCH WITH Z-MAGNETIC ATTACHMENT

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AQR CLUTCH KIT

paraocclusal clutch, tray clutch N, Paraocclusal Clutch AXIODAPT, dentulous and edentulous with NT-clamp and Z-magnetic attachment and allen hex wrench

Interesse an einem Kurs?
Interested in a course?

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Two and a half day intensive practical course about temporomandibular joint measurement

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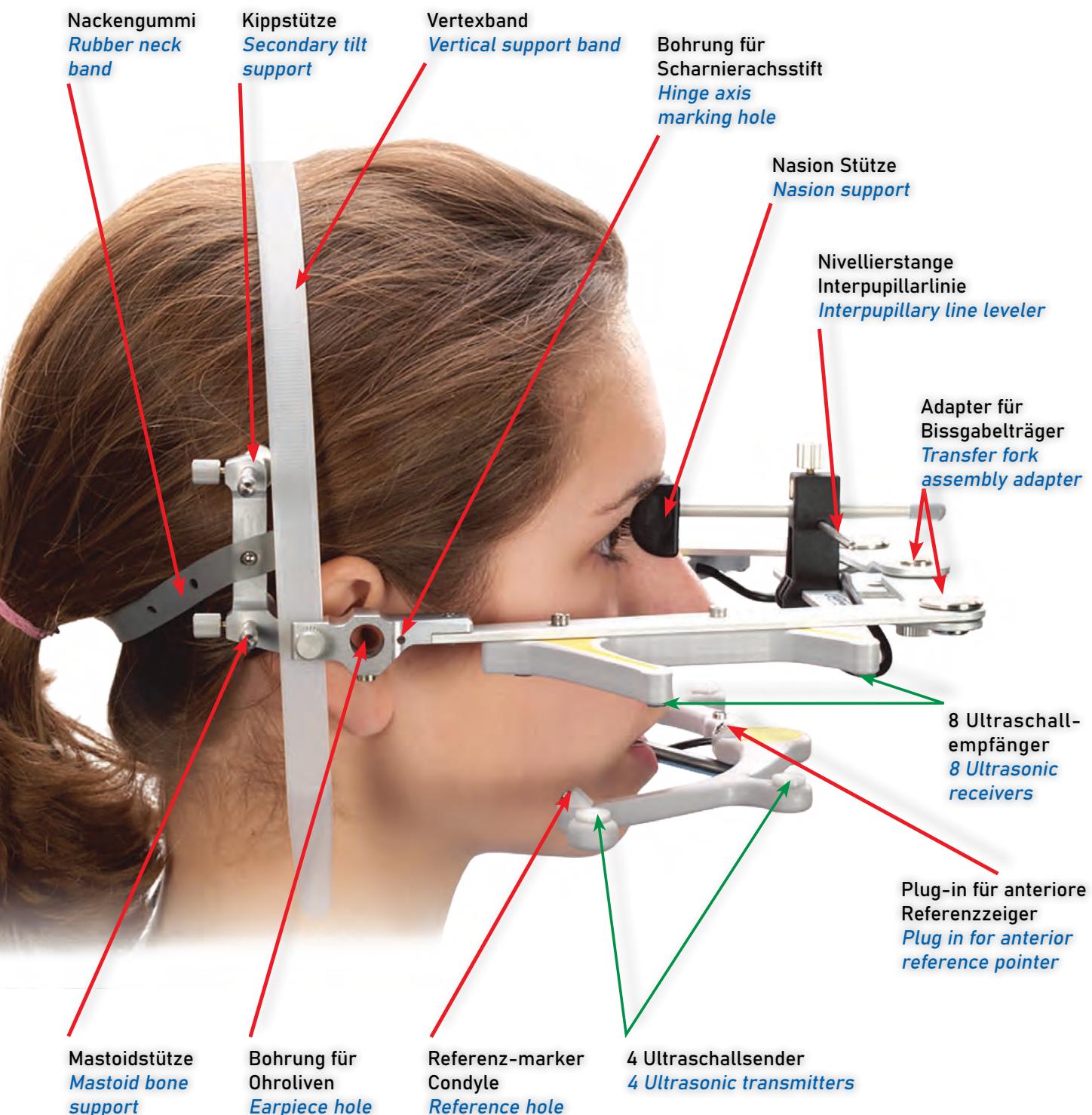
- instrumental temporomandibular joint diagnostics
- exact (individual) articulator programming
- axis localization
- exercise with a simulator
- mutual measuring (course participants)

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The AXIOQUICK Recorder is the combination of the anatomic facebow with a contactless ultrasonic recording device. Four ultrasonic transmitters and eight receivers detect the mandibular movements. The exact measurements of the condylar pathways and the anterior guidance are shown both graphically and numerically relative to a zero reference plane.

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The quick recording for articulator programming provides the data for extensive functional diagnostics, and in addition takes extensive measurements of the lower jaw positions and movements with synchronous optional EMG recording. Simultaneously the upper jaw model can be transferred to the articulator.

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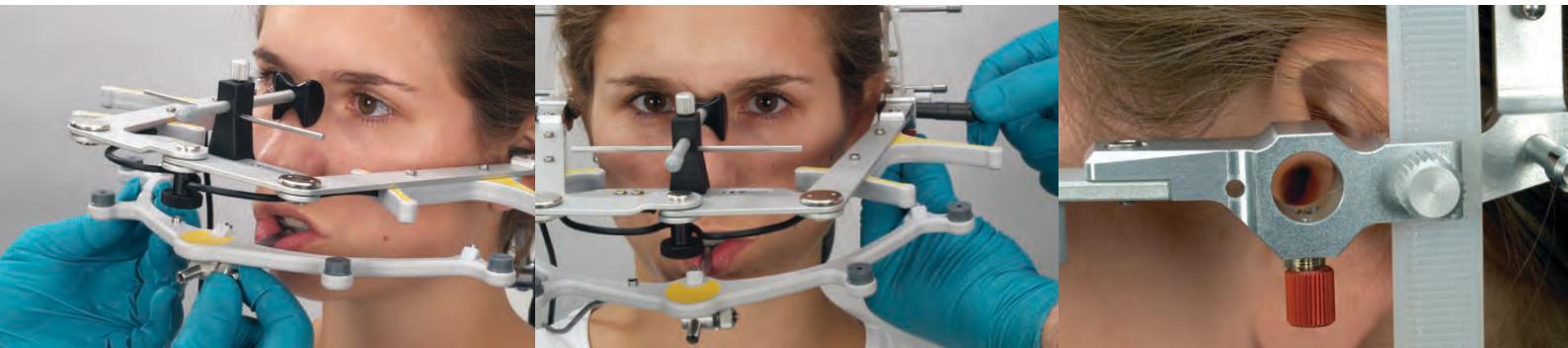
The Axiogram is the graphical Cartesian presentation of the measurements. There is a special diagnostic evaluation enabled by playing the recording as an animation. Angles and distances can be evaluated accurately.

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Porion-orbital plane (Frankfurt Horizontal plane/ axis-orbital plane) is the reference plane for all the measurements and the accurate positioning of the upper cast with reference to the skull. This ensures a correct and true reproduction of the condylar pathways and anterior guidance with reference to the recordings.

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3D presentations of the anterior guidance, the Bonwill-triangle and the DATA from the CT are possible.

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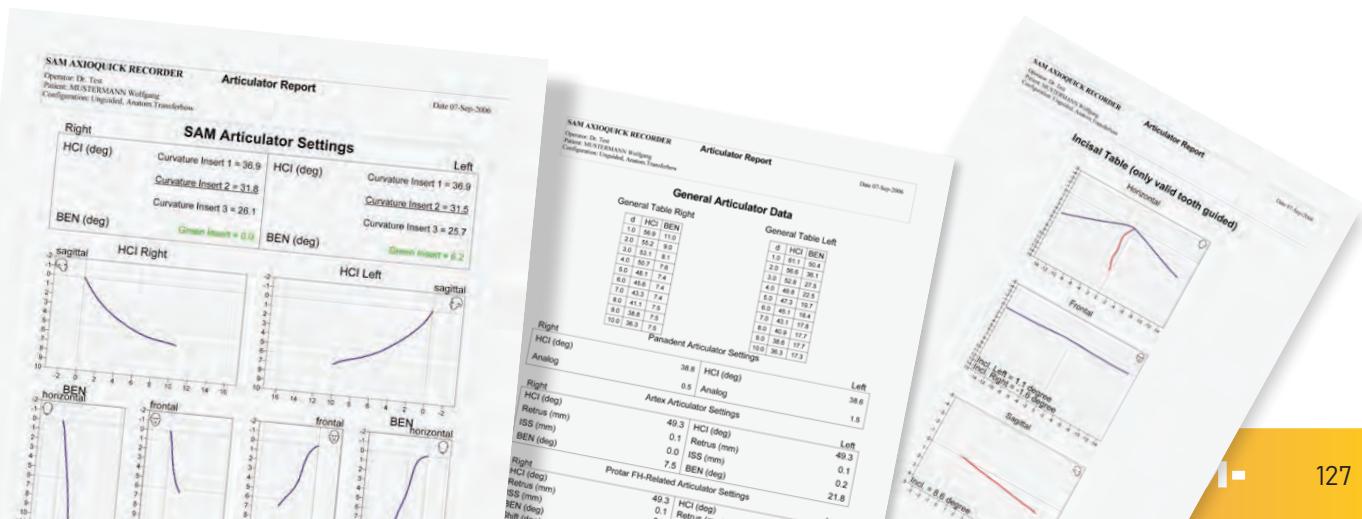
Ultrasonics operate on the principle of interval transit time recording of ultrasonic pulses. In a way similar to satellite navigation, the positions of teeth and the movement of the lower jaw can be calculated. The CPA program (condyle position analysis) relates the record-

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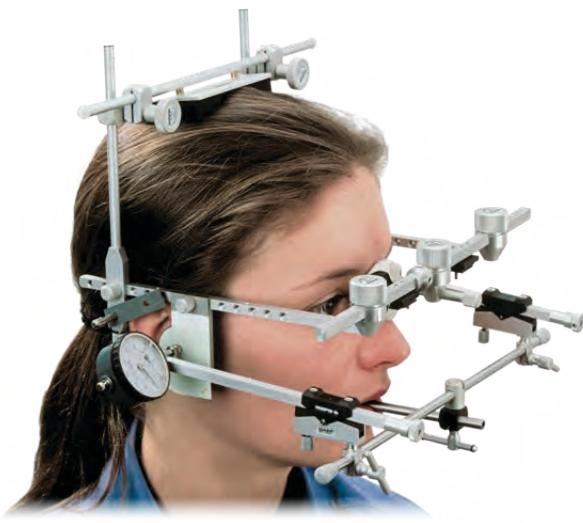
ing of the patient to a grid matrix with up to 10 reference points. The hinge axis is determined by manually guiding the patient's mandible with a 10 to 18 mm condyle rotation. This is done twice and you should see comparable values. Repeat as necessary.

SAM AXIOPH -DAS ORIGINAL GELENKBAHN-AUZEICHNUNGSISTRUMENT

SAM AXIOPH -THE ORIGINAL JOINT TRAJECTORY RECORDING INSTRUMENT

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AXIOPH® III KIT

used for functional analysis of the TMJ system, recording mandibular movement, anatomical and hinge axis facebow transfer. Also used with the ultrasonic AXIOPH® with hinge axis facebow mounting device, supplied in a carrying case

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*AXO 530 FOR SCREW TYPE SYSTEMS & AXO 545M FOR MPS

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RULER FOR AXIOPHAPH, STEEL, 200 mm
used to measure AXIOPHAPH interflag distance

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AXIOPHAPH RECORD SHEET
for recording AXIOPHAPH
(50 sheets)

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REGISTRATION GRID LABELS, GREEN
self-adhesive millimeter grids - á 4 pairs/page
(15 sheets)

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TRAY CLUTCH N
for AXIOPHAPH and AXIOQUICK Recorder

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PARAOCCCLUSAL CLUTCH

used to fabricate individualized paraocclusal custom clutch for AXIOGRAPH and AXIOQUICK Recorder

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PARAOCCCLUSAL CLUTCH, EDENTULOUS

for total prosthetics, incl. hex screwdriver, 2mm. Suitable as a holder, for removable dentures, when registering

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ARTICULATOR ADAPTER KIT

includes asymmetric incisal pin (ART 361) with transfer bow crossrod and clamp, for the production of paraocclusal clutch on articulator mounted casts in the lab for AXIODAPT

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MANDIBULAR CLAMP EDENTULOUS

for fixation of registration adapter for AXIODAPT edentulous

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TRANSFER FORK ADAPTER ASSEMBLY
for AXIOGRAPH III

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LUBRICANT LIQUID

contents 10ml

it makes easier to remove the labels from Axiograph and MPI

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OCCLUSAL INDICATOR WAX

green, used for intraoral occlusal contact checking, 300 µm

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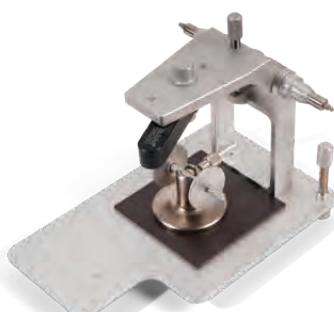
SHAN-AXLE MOUNTING DEVICE

with spindle operation for symmetrical extraction of the axes, exact transfer of the upper jaw model using the axiograph

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AXIOSPLIT starter kit N with normal magnets

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AXIOSPLIT starter kit S with strong magnets

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includes AXIOSPLIT® master check P with incisal pin (ASP 470), regular matrix holder (ASP 529 or ASP 530), matrix holder, two-piece assembly (ASP 539 or ASP 540), each with magnets with normal strength (ASP 580), security rings (ASP 435) and 20 oval mounting plates (ASP 420)

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for every other articulator with master check system, includes regular matrix holder (ASP 529 or ASP 530), matrix holder, two-piece assembly (ASP 539 or ASP 540), each with magnets and security rings (ASP 435)

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AXIOSPLIT MATRIX HOLDER N, ONE-PIECE
*screw type metal plate for articulator upper member
with normal magnet*

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AXIOSPLIT MATRIX HOLDER S, One piece
*screw type metal plate for articulator upper member
with strong magnet*

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AXIOSPLIT MATRIX HOLDER N, TWO PIECE ASSEMBLY
*for adhesive adjustment in the lower part of articulator
screw type metal plate supplied with normal magnet*

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AXIOSPLIT MATRIX HOLDER S, TWO PIECE ASSEMBLY
*for adhesive adjustment in the lower part of articulator
screw type metal plate supplied with strong magnet*

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AXIOSPLIT CAST POSITIONING TABLE S
adjustable cast holder includes strong magnet

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BENEFITS OF THE AXIOSPLIT SYSTEM

- *Synchronisation of articulators between practice and laboratory*
- *Precision control of the articulator*
- *Simple, precise and fast removal and re-installation of models*
- *prefabricated splitcast*
- *100% parallelism of upper and lower part by means of standard adjusting pin*
- *Simple and fast exchange between two different magnetic strengths*
- *Fast magnet removal without additional tools*
- *easy to reuse and recalibrate*

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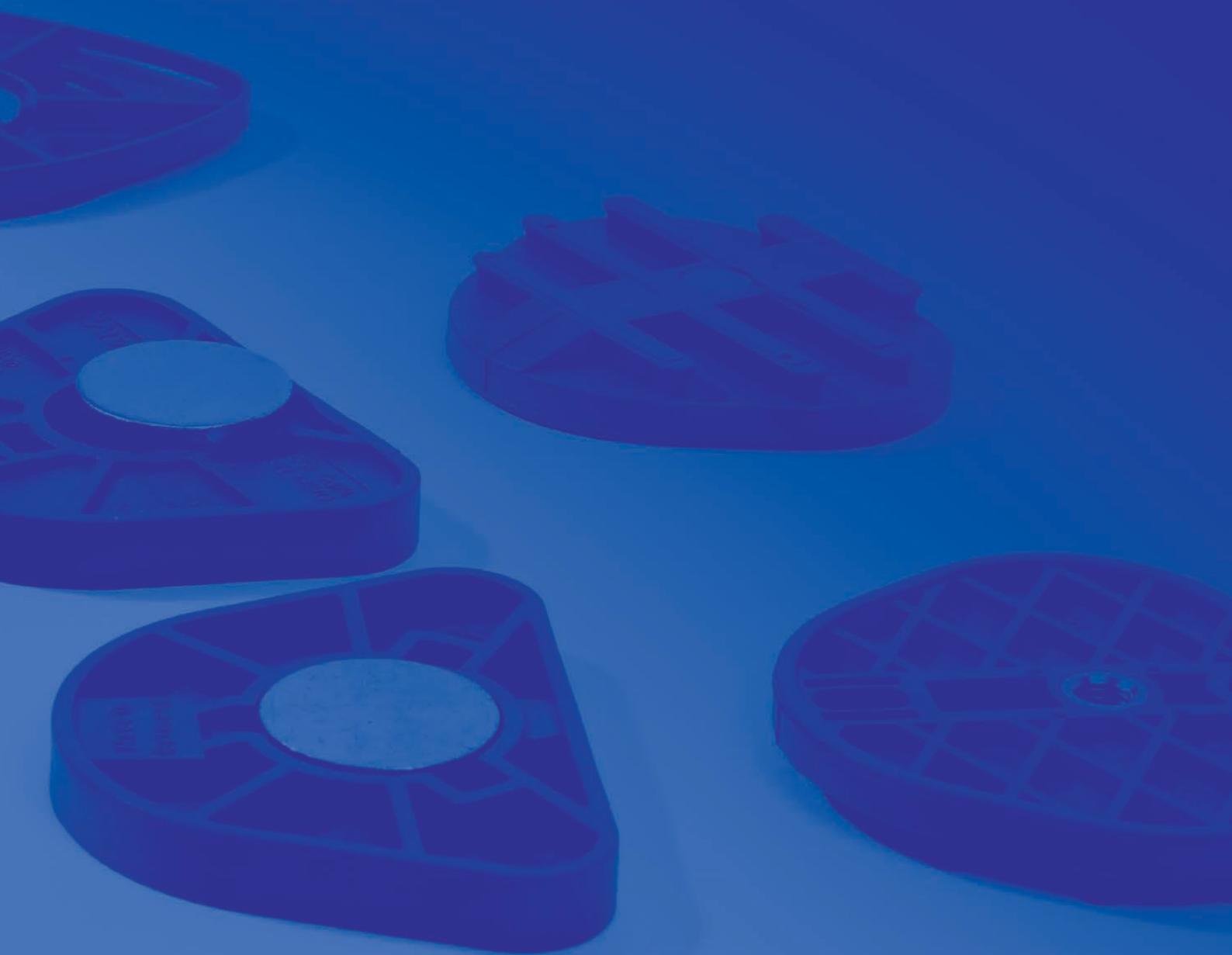
The synchronisation is possible in all articulators. With the MPS mounting plate system, only the magnetic pots have to be exchanged for the (ART 114) thumb screws.

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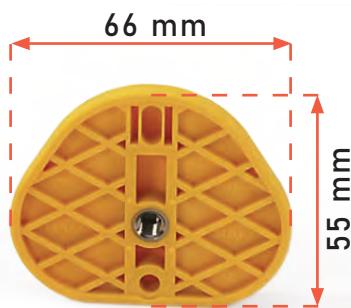
THE SAM MOUNTING PLATES WITH SCREW THREAD

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*Suitable not only for SAM articulators,
but also for:*

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1 PAIR OF LOCKING SCREWS

Includes hex wrench for mounting on the upper and lower part of the SAM or third-party articulators.

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**MOUNTING PLATES, SCREW TYPE - 66 x 55 mm
(20 pieces)**

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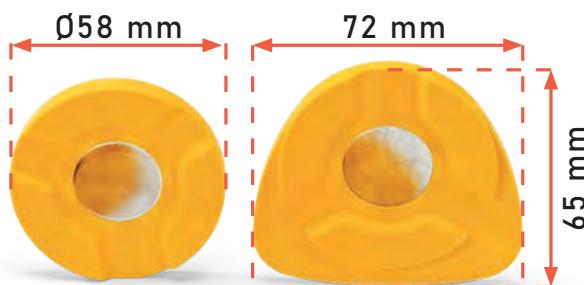
**MOUNTING PLATES THUMB SCREW BP - 66 x 55 mm
(100 pieces) bulk pack**

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Preformed secondary plates made of yellow plastic, for single use, with adhesive plate, for abrasion-free model.

REF ASP 420

**AXIOSPLIT MONTAGEPLATTEN OVAL - 72 x 65 mm
(20 Stück)**

**AXIOSPLIT MOUNTING PLATES OVAL - 72 x 65 mm
(20 pieces)**

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**AXIOSPLIT MOUNTING PLATES OVAL BP - 72 x 65 mm
(100 pieces) bulk pack**

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**mm Ø 58 - AXIOSPLIT MOUNTING PLATES ROUND
(20 pieces)**

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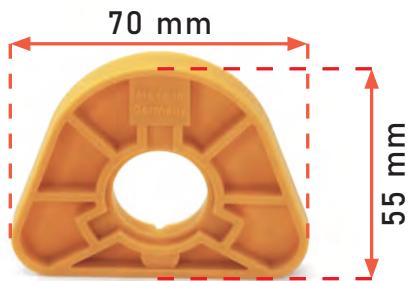
AXIOSPLIT DOUBLE MATRIX FORMER
for the production of secondary matrix holder made of gypsum with rubber rings and lubricant liquid

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mm 70 x 55 - MPS MOUNTING PLATES, YELLOW
(20 pieces) including metal adhesion plates

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mm 70 x 55 - MPS MOUNTING PLATES, YELLOW BP
(100 pieces) bulk pack, including metal adhesion plates

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MPS MOUNTING PLATE SYSTEM, STARTER KIT YELLOW - 70 x 55 mm
includes two magnetic holding cups (M 5x13mm) with nuts (closed) and yellow wrench (MPS 110), cup positioner (MPS 111), and 20 mounting plates yellow with adhesion plates (MPS 150) for SAM, WhipMix, Panadent, and Hanau 2Pin articulators

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MPS ADDITIONAL ARTICULATOR KIT, YELLOW - 70 x 55 mm

includes two magnetic holding cups (M 5x13mm) with nuts (closed) and yellow wrench, used to attach the MPS-system to additional articulators for SAM, WhipMix, Panadent, Denar, AD2, Hanau Ulti Mate und Hanau 2Pin articulators

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70 x 60 mm - MPS MOUNTING PLATES, GREY
(20 pieces) for non-SAM articulators,
including metal adhesion plates

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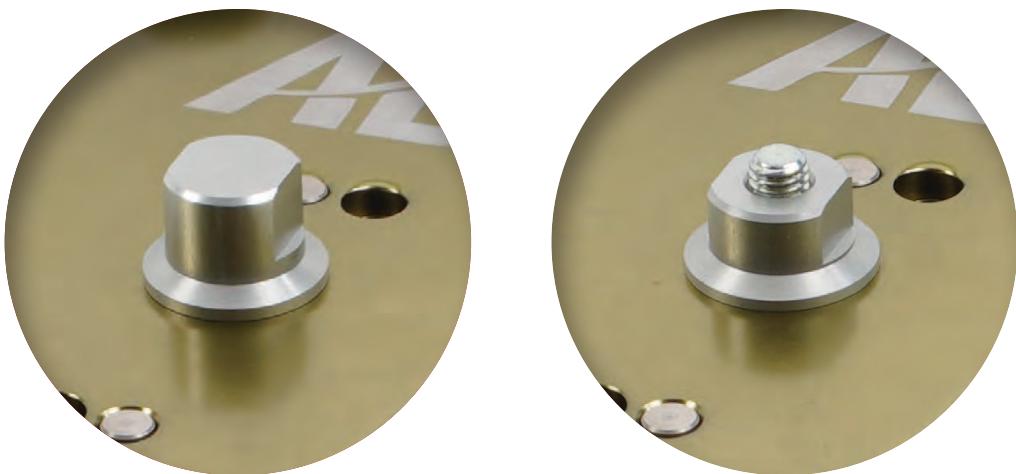
70 x 60 mm - MPS MOUNTING PLATES, GREY BP
(100 pieces) bulk pack for non-SAM articulators
including metal adhesion plates

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MPS STARTER KIT, GREY I

includes two magnetic holding cups (M 5x17mm) with one nut each (open and closed), yellow wrench, cup positioner (MPS 111), and 20 grey mounting plates with metal adhesion plates (MPS 250)

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MPS ADDITIONAL ARTICULATOR KIT GREY I

includes two magnetic holding cups (M 5x17mm) with one nut each (open and closed), yellow wrench, cup positioner (MPS 111), and two grey mounting plates with metal adhesion plates

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MPS STARTER KIT GK, GREY III

includes two magnetic holding cups (M 5x13mm) with nuts (closed) and yellow wrench (MPS 201K), cup positioner (MPS 111), and 20 grey mounting plates with metal adhesion plates (MPS 250)

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MPS ADDITIONAL ARTICULATOR KIT GREY III

includes two magnetic holding cups (M 5x13mm) with nuts (closed), yellow wrench, cup positioner (MPS 111), and two grey mounting plates with metal adhesion plates

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TRANSFER FORK SUPPORT, ACRYLIC XL

used to support transfer fork during upper cast mounting or to support the articulator during lower cast mounting (to compensate for the occlusion inclination)

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TRANSFER FORK SUPPORT, TELESCOPIC

used to support transfer fork during upper cast mounting procedures

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MAGNETIC BLOCK FOR TRANSFER FORK SUPPORT

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Magnet plate for holding the telescopic transfer fork support in the articulator.

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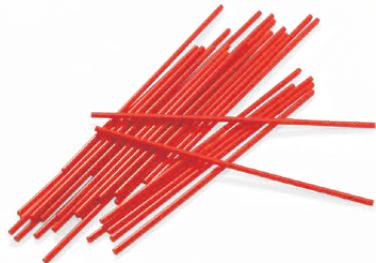
MOUNTING STAND

Aid for UK mounting, suitable for SAM 2P, SAM 2PX, SAM 3 , SAM NEO and reference centric instruments. Including model holder and 30 elastic rods.

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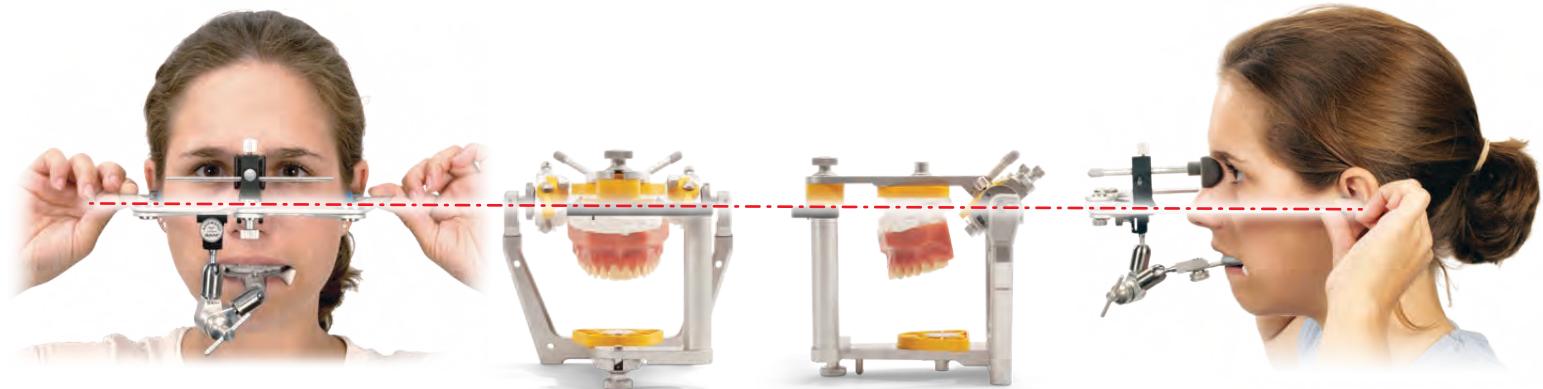
FLEXIBLE PLASTIC RODS

(30 pieces) rods, for single use only, fixes the lower cast model during assembly

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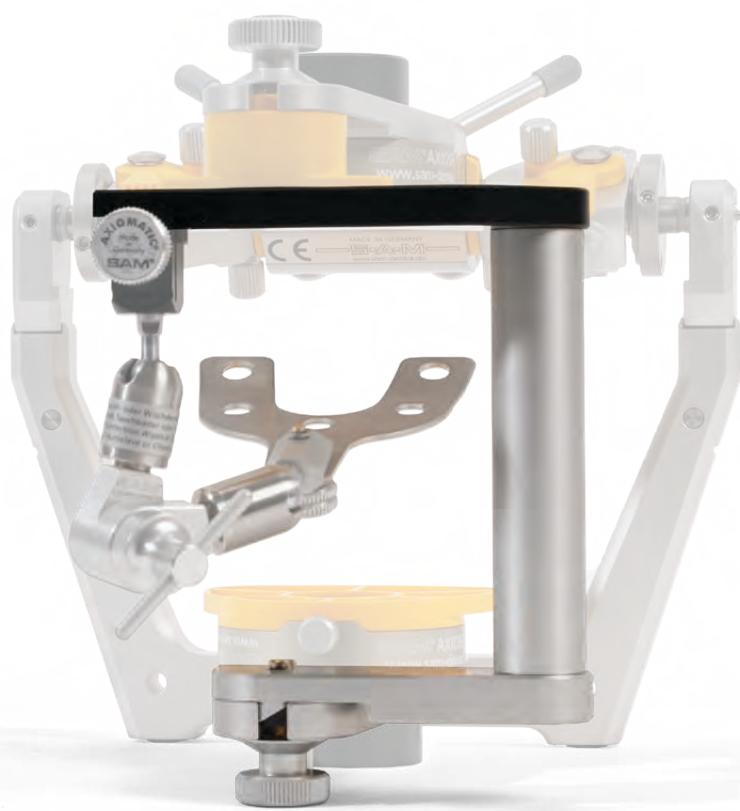
*Our target: transfer the patient situation into the articulator as accurately as possible.
Our solution: the SAM - transfer system*

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Anatomical symmetry in the articulator for individual model assembly. The Transfer bow remains in the treatment room, the bite fork carrier moves to the laboratory plaster table.

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TRANSFER STAND AX

for an easy and secure upper cast model mounting in the articulators SAM SE, SAM NEO, SAM 2P, SAM 2PX und SAM 3 (P-Norm) and in all reference centric instruments

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TRANSFER STAND SE

made out of plastic for an easy and secure upper cast model mounting

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All versions of the SAM transfer fork assemblies are compatible with the transfer stand AX. With the transfer stand the upper jaw position of the patient can be quickly and precisely transferred to the articulator.

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TRANSFERSTAND - D3
für Denar 300 Serie
for Denar 300 series

TRANSFERSTAND - WM 3
für WhipMix 3000 Serie
for WhipMix 3000 series

Transfer stands for non-SAM® articulators

With the SAM anatomical transfer bow and the special transfer stands, the upper jaw model can also be transferred to many common articulators that are not from SAM. Different construction heights and articulator adapters are taken into

account and calibrated to the axis-orbital height. Following the pattern of the AX transfer stand, all SAM bite fork carriers with a sliding guide (ATB 395, ATB 305, ATB 310, ATB 320) can be used for upper jaw model transfer.

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Model making

Magnet-Splitcast-Former

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ACCESSORIES FOR SAM MODEL TECHNOLOGY

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MAGNET SPLITCAST FORMER COMBINATION KIT

includes all four splitcast former yellow S (\varnothing 70 mm), red M (\varnothing 80 mm), black reciprocal M (\varnothing 80 mm) and green XL (\varnothing 110 mm), 4 rubber rings, 1 magnet remover, 6 magnets and 6 magnet cups, 6 adhesion plates and 1 UG-sep spray

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Adhesion plates with grooved retention

(50 pieces) for magnetic splicast former, turned and galvanized, Ø 25 mm, thickness: 3 mm

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Magnet holding cups

(50 pieces) for MSF magnets, made of turned solid material and galvanized, Ø 25 mm, thickness: 7,3 mm

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Magnets for MSF

(20 pieces) with MSF magnet remover with wing clamping, Ø 20 mm, thickness: 6 mm

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MSF mini student kit

includes six magnets and magnetic holding cups as well as six adhesion plates

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Magnet splitcast former kit normal, red

includes form plate normal, red, size 80 mm, rubber ring, magnet remover, six magnets and magnetic holding cups, six adhesion plates

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MAGNET SPLITCAST FORMER KIT RECIPROCAL, BLACK

includes reciprocal form plate, black, size 80 mm, rubber ring, magnet remover, six magnets and magnetic holding cups, six adhesion plates

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MAGNET SPLITCAST FORMER KIT SMALL, YELLOW

includes form plate small, yellow, size 70 mm, rubber ring, magnet remover, six magnets and magnetic holding cups, six adhesion plates

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MAGNET SPLITCAST FORMER KIT XL, GREEN

includes form plate XL, green, size 110 mm, rubber ring, magnet remover, six magnets and magnetic holding cups, six adhesion plates

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BOXING AID DEVICE KIT XL, GREEN

used for boxing denture impressions without performing the usual wax boxing procedure, includes two form plates XL green, magnet remover, rubber ring, modelling clay, six magnets and magnet holding cups, six adhesion plates

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DISPOSPLIT STARTER KIT

(6x pieces each) quick matrix holders with rubber ring, magnets and magnetic holding cups and adhesion plates (yellow gold) and one magnet remover

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DISPOSPLIT LABORATORY PACK, WHITE

(25 pieces each) quick socket plates includes magnets, magnetic holding cups and adhesion plates (yellow gold)

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PIN DRILL MACHINE

Horizontal pin drilling machine with a view of the drill tip and optimum positioning of the drill hole according to the saw cut drawings, with drilling depth gauge and hexagon key.

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DOWEL PIN DRILL BIT 1.6 mm

(3 pieces) for conical or AXIOPIN parallel dowel pins, 3,0 mm shaft

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CONICAL PIN FOR CERAMICS

(24 pieces), reusable, for duplication in ceramics parallel dowel pin system

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PINS AND SLEEVES AXIOPIN STARTER KIT

(200 pieces each) includes pins and sleeves and one drill bit

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The need for control of the gypsum or investment in the practice or laboratory is due to the differing batches from manufacturer, local water quality and the individual dosage and processing methods. The SAM expansion meter EMI is designed to study the expansion of gypsum and investments, the accuracy of the mechanical

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gauge is 0.01 mm. The expansion is therefore easy to convert into percent, as the measuring cup has a length of 100 mm ($0.01 \text{ mm} \triangleq 0.1\%$). To prevent the test materials from sticking to the measuring cup, the cup should be lined with rubber dam (0.1 - 0.2 mm) and talcum powder.

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EXPANSION MEASURING INSTRUMENT, ANALOG
with mechanical dial gauge, precision ($0.01 \text{ mm} \leq 0.1\%$)

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EXPANSION MEASURING INSTRUMENT, DIGITAL
with LCD digital gauge, precision ($0.01 \text{ mm} \leq 0.1\%$)

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Used to measure the expansion behavior of gypsum from different batches and mixing procedures.

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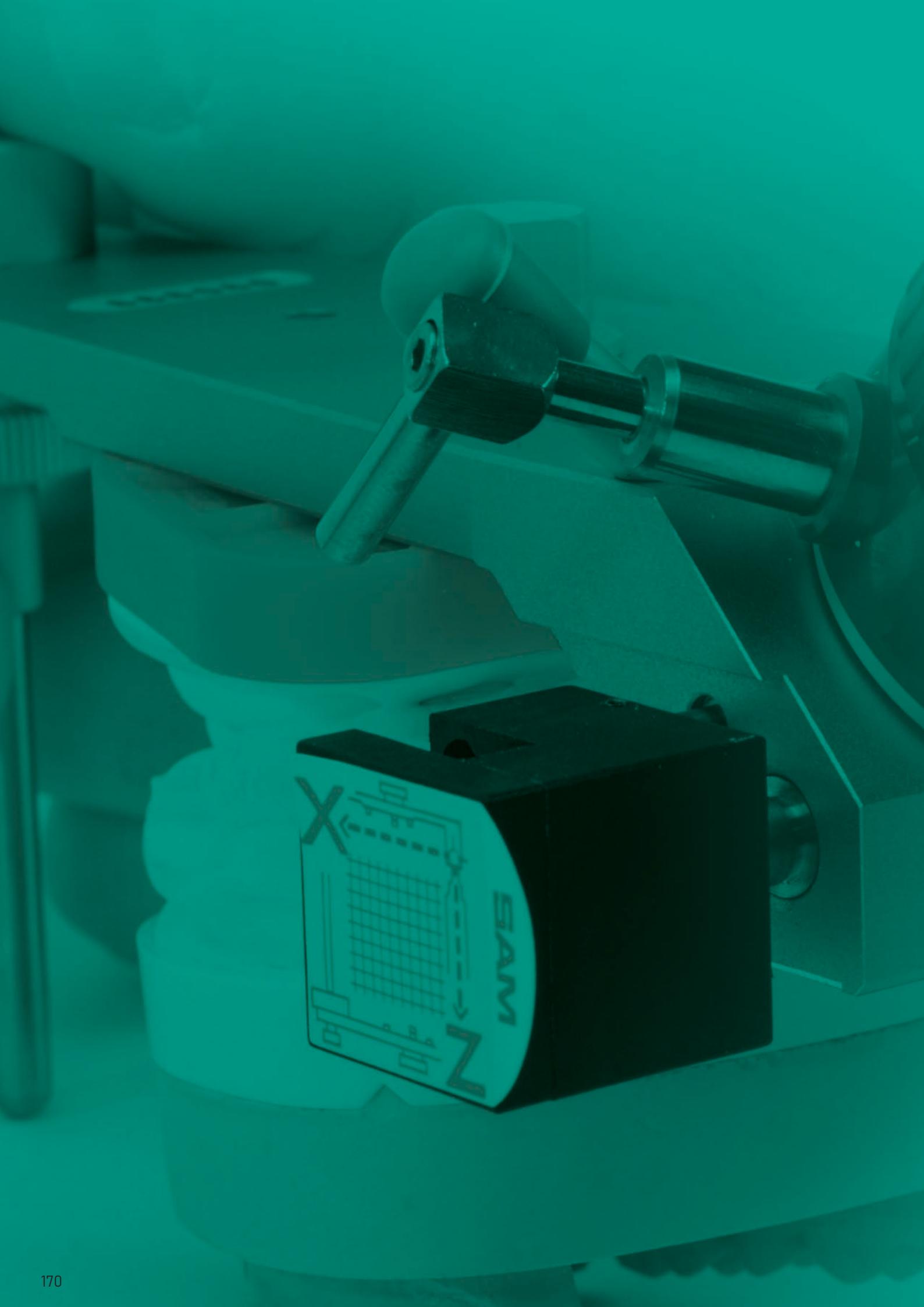
EXPANSION MEASURING INSTRUMENT, DIGITAL PC KIT
with LCD digital gauge, precision ($0.01 \text{ mm} \leq 0.1\%$) and USB output

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For automatic documentation of the expansion behaviour of gypsum over several hours. Measurement recording by time interval or expansion steps. Easy further processing in a spreadsheet program for graphical representation.

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MPI - FOR MEASURING THE CONDYLAR DISPLACEMENT IN THE ARTICULATOR

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REGISTER LABELS / REPORT BLOCK FOR MPI
(15 sheets) six labels each and report block DIN A5 25 sheets each

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used for comparing centric records, changes in mandibular cast position and monitoring splint adjustments, etc., includes universal incisal pin, 15 sheets à six labels with record sheet (MPI 110), and calibration tool

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MPI - THE SPECIAL ARTICULATOR FOR THREE-DIMENSIONAL MEASUREMENT

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for MPS magnetic mounting plate system

for scientific analysis, three dimensional measuring in the mandibular area, tolerance 0,01 mm, supplied with master check and a carrying case

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MPV - THE SPECIAL ARTICULATOR FOR TEMPOROMANDIBULAR JOINT THERAPY

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Special articulator based on the SAM 3, with cross table guides instead of condylar housings, for precise linear displacement of condylar positions, e.g. for the fabrication of bite blocks.

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for MPS magnetic mounting plate system

Special articulator for the fabrication of bite blocks, based on the SAM 3 articulator, with cross table guides instead of condylar housings, with precise adjustment option thanks to vernier display, lateral adjustment for linear displacement of condylar positions, raised by 15 mm.

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for MPS magnetic mounting plate system

Special articulator on the basis of a SAM 3 with cross table guides instead of condylar housings for precise linear relocation of the condylepositions, increased by 15 mm e.g. in the production of occlusion aids.

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for MPS magnetic mounting plate system

Special articulator for the fabrication of bite blocks, based on the SAM 3 articulator, with cross table guides instead of condylar housings, with precise adjustment option thanks to Vernier display, lateral adjustment for linear displacement of condylar positions, raised by 15 mm.

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Lateral adjustment set for MPV III - Standard
*Displacement transversal to 3 mm,
subsequent fitting on request*

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Ohne Lateralverstellung
Without lateral adjustment

Mandibular-Positions-Variator III Variationen

Mandibular-Positions-Variator III Variations

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Our experts recommend the Model III - Plus and MKG for more versatile use.

	REF-Nr.	Modellbefestigungssystem (Montageplatte)		Bauhöhe		Verstellmöglichkeit	
		Schraub / für ASP	MPS	Norm	Norm + 15mm	2D (X- & Z-Achse)	3D (X-, Y- & Z-Achse)
STANDARD	REF MPV 500	○		○		○	
	REF MPV 515M		○	○		○	
MKG	REF MPV 560	○			○	○	
	REF MPV 575M		○		○	○	
PLUS	REF MPV 550	○		○			○
	REF MPV 565M		○	○			○
MKG PLUS	REF MPV 580	○			○		○
	REF MPV 595M		○		○		○

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REF MPI 316

Mandibulator-Positions-
Indikator, MPS
*Mandibular positions
indicator, mps*

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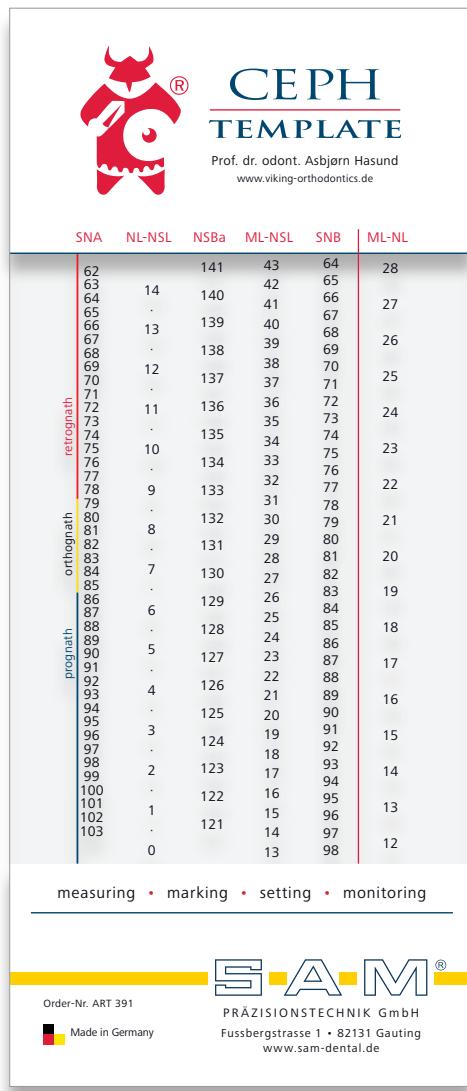
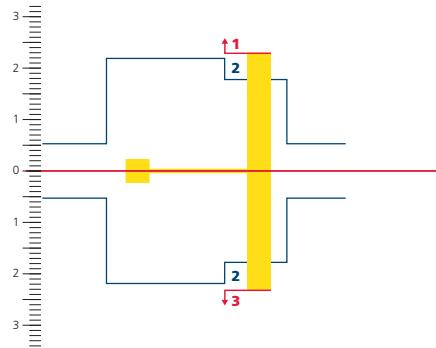
The Ceph Template was developed as a tool for the practical implementation of Hasund analysis. The CEPH template is easy to handle. Current data are visualized graphically and thus can be interpreted quickly and precisely.

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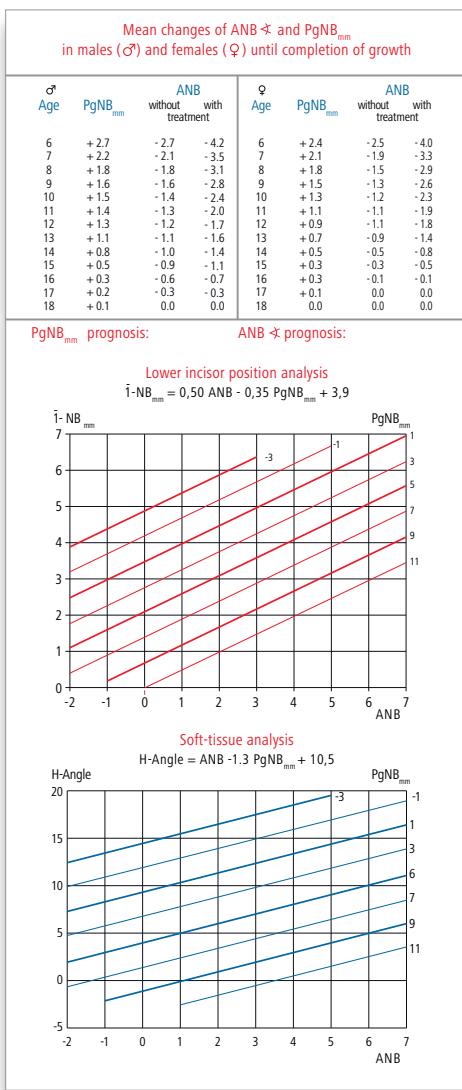
CEPH TEMPLATE front:

- harmony box with basal cardinal values, range of facial type classification (retro-, ortho-, prognath)
- movable grid with tolerance limits (TG)
- superharmonious line (SHL)
- yellow marker on SHL (NL-NSL) and yellow bar: range of the vertical basal classification (1,2,3,) on ML-NSL
- measurement scale

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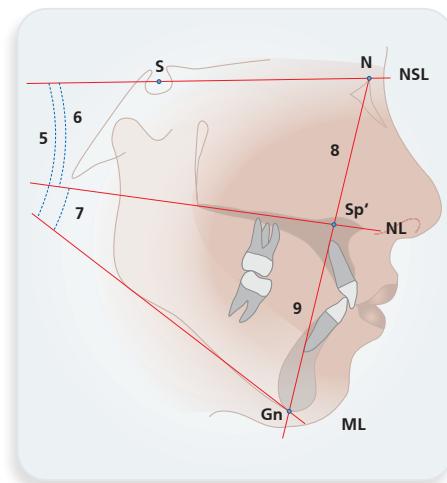
CEPH template rear:

- a) Table with mean values to support the prognosis of ANB-angle and prominence of the chin at the end of treatment
- b) Nomogram for the lower incisor position analysis
- c) Nomogram for the soft-tissue analysis

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CEPH TEMPLATE
inclusive instruction manual with description of the HASUND-analysis

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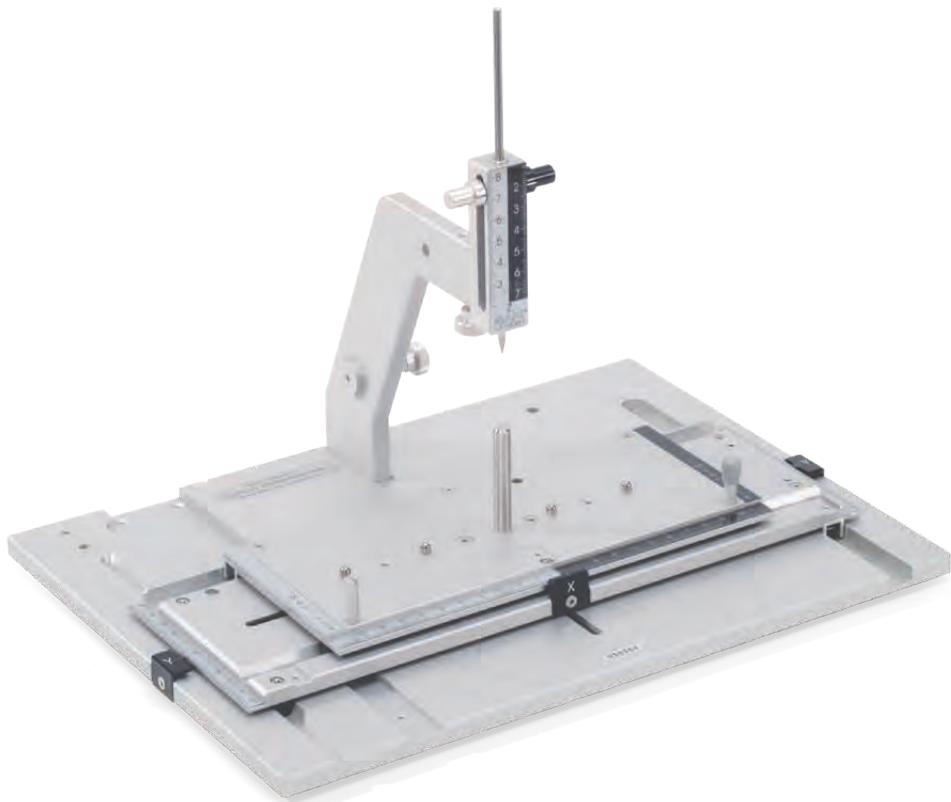
RMI - 3D DETECTION OF ANTAGONISTIC TOOTH CONTACTS

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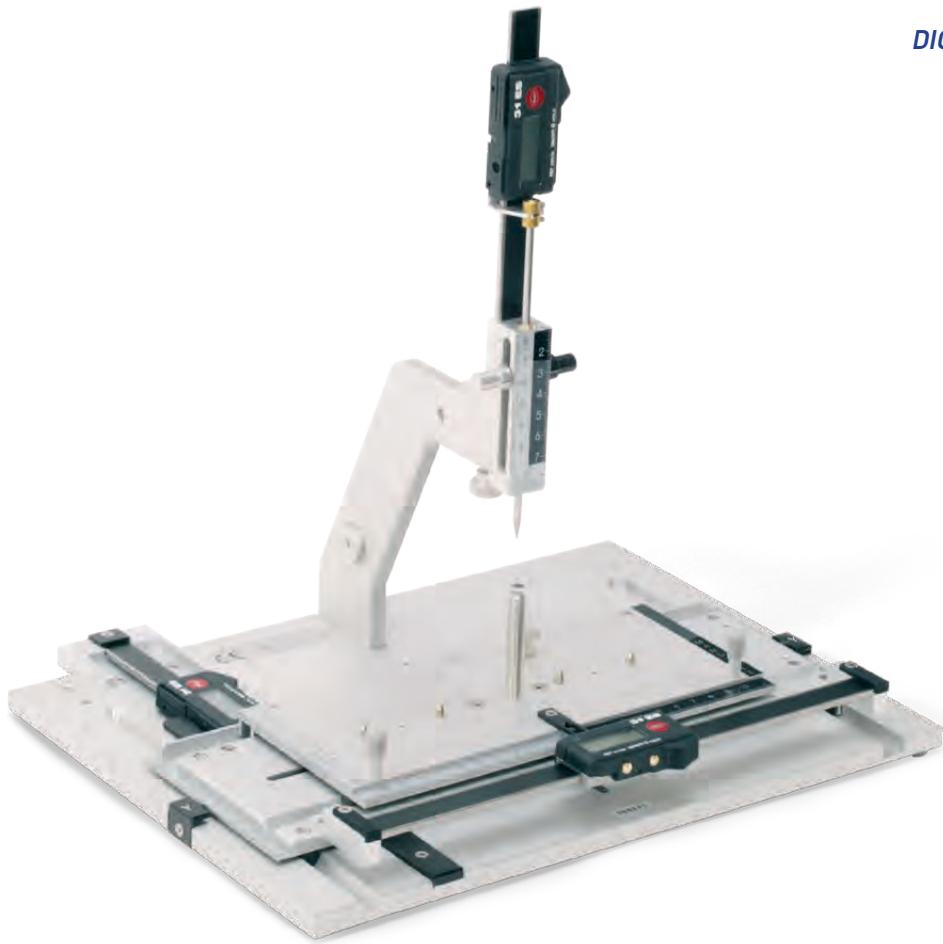
The SAM RMI 500/550 enables simultaneous three-dimensional measurement of mounted maxillary and mandibular models. For comparison of corresponding points in the occlusion or on the alveolar ridge, optimization of the position of implant abutments and documentation. The 3D measuring instrument is available in a mechanical version (RMI500), as well as in an electronic version (RMI550).

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The SAM 3 H offers even more space during model assembly (ART 560 / ART 575M) and for the model repositioning instrument (MRI 300); Content is at page 196.

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MRI plastic sleeve insert kit

includes sleeve insert tool (MRI 212), drill bit (MRI 211), and 200 plastic sleeve inserts (MRI 210)

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MODEL REPOSITIONING INSTRUMENT

with MRI sleeve kit II (MRI 220K)

The MRI was developed in cooperation with Prof. Dr. med. dent. Rainer Schwestka-Polly (University of Hanover) and is used for surgical planning in maxillofacial surgery for three-dimensional repositioning of the upper or lower jaw model.

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are registered trademarks of SAM® Präzisionstechnik GmbH
SAM Präzisionstechnik GmbH