

# Instruction For Use

## Ortho-Jel® Adhesive

Tray adhesives are recommended to assure reliable adhesion of impression material to the impression tray and to avoid uncontrollable deformation during impression taking.

### Indications

Ortho-Jel® Adhesive secures safe adhesion between addition curing or condensation curing silicones and metal - or methacrylate trays.

**Note :** The adhesive does not always secure sufficient adhesion to trays made of other plastic materials or individual trays made of base plates. A polyether adhesive must be used for polyether impression materials.

### Working Instructions

Apply a thin layer of Ortho-Jel® Adhesive to the tray using the applicator brush and allow to dry at least **3 minutes**. Then load the tray and take the impression..

### Cleaning from the tray

Ortho-Jel® Adhesive can be removed from the tray with acetone.

### Warning

Ortho-Jel® Adhesive should only be used in well ventilated rooms. In case of contact with skin, wash off with water.

### Storage

Do not store above 25 °C (78 °F). Do not use after expiry date.

### Warranty

RMO Europe warrants this product will be free from defects in material and manufacture. RMO Europe makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and RMO Europe's sole obligation shall be repair or replacement of the product.

### Limitation of Liability

Except where prohibited by law, RMO Europe will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**Keep away from children.**

**Only for use by dentists and dental technicians.**