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INSTRUCTIONS FOR USE

1. Keep the water between 40 to 45°C.
2. Adjust the water temperature according to the orthodontic forces to be applied on the teeth. Example: for sliding mechanics with light forces with thin wires, raise the temperature to 45°C. When working with heavy forces, such as Cervical Headgear, use lower temperature, around 40°C.
3. Caution, when the temperature reaches 50°C all anatomic wax structure will start to collapse.
4. In order to obtain the best movement, leave the typodont between 10 to 25 minutes under the water, this way the warm temperature can reach the most internal parts of the wax.
5. Do not manipulate the typodont immediately after you remove from warm water, instead keep it under cold water for 5 minutes.
6. The portion related to the teeth roots are made with thin metal alloy with a small retention on the apex to facilitate the teeth movements. This the reason why that, after the teeth movement on warm water, the teeth can present mobility, but they will not fall off the wax socket. After any movement step we recommend the student to fulfill the gaps between the mobile teeth and the socket with melt wax (with the supplied wax bar), before move to a next movement step.
7. The interior of a closed car under the sun can reach 60°C. Therefore, never leave the typodont inside a parked car under the sun.