## ACCESSOIRES LIBERTY BIELLE® FIX™

# Liberty Bielle® Fix™

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RÉFÉRENCE	DESCRIPTION	UNITÉ DE VENTE
ECM3035	<b>Kit de démarrage Liberty Bielle® Fix™*</b>  Contient : <ul style="list-style-type: none"> <li>• 1 ECM3030S - 1 Liberty Bielle® Fix™ 32mm</li> <li>• 1 ECM3032S - 1 Liberty Bielle® Fix™ 22mm</li> <li>• 1 ECM3034S - 1 Liberty Bielle® Fix™ 20mm</li> <li>• 1 ECM3071 - Tournevis Liberty Bielle®</li> <li>• 2 cales de propulsion 1,5mm</li> <li>• 2 Vis Arc Hexagonale</li> </ul> <b>*Existe en version de distalisation et/ou clipsable</b>	1
ECM3030S	Liberty Bielle® Fix™ 32mm	1 cas
ECM3032S	Liberty Bielle® Fix™ 22mm	1 cas
ECM3034S	Liberty Bielle® Fix™ 20mm	1 cas
ECM3047	Liberty Bielle® Fix™ Maxillaire 32mm	2
ECM3048	Liberty Bielle® Mandibulaire 32mm	2
ECM3036	Liberty Bielle® Fix™ Maxillaire 22mm	2
ECM3038	Liberty Bielle® Fix™ Maxillaire 20mm	2
ECM3031	Liberty Bielle® Fix™ Pivot	10
ECM3067	Liberty Bielle® Fix™ Pivot de distalisation (Fil jusqu'à 0.22x.028)	10
ECM3071	Tournevis Liberty Bielle®	1
ECM3042	Cales de propulsion 1.5mm	10
ECM3046	Vis Liberty Bielle®	10
J01508	Vis arc hexagonale	5

### PRÉCAUTION :

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### CAUTION :


This appliance contains nickel and should not be used for individuals with known nickelsensitivity.

### CLEANING :

The Liberty Bielle is delivered clean and non-sterile. However, after adjusting the device and before inserting it into the mouth, we recommend its ultrasonic cleaning.

# Liberty Bielle® Fix™

## Instructions for use

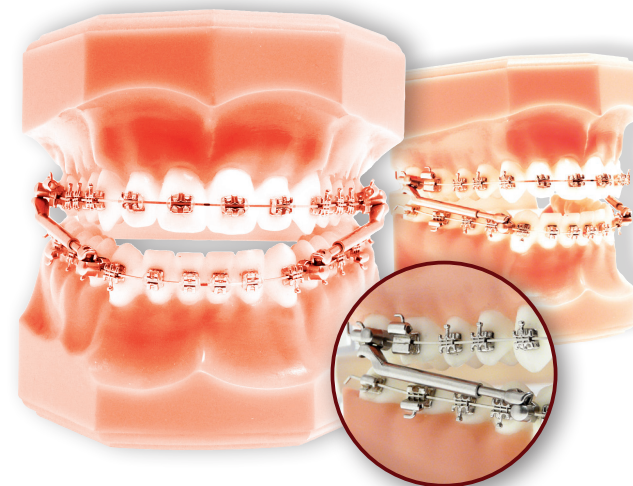### THIS MEDICAL DEVICE IS FOR SINGLE USE :

Screw is self-locking, if the Liberty Bielle screw is de-mounted it must be replaced by a new one, in order to assure the locking feature.

If re-used cross contamination can occur.

Notice LibertyBielle\_Fixe\_ Rev2.3 - 02/10/17 - EN


## Mounting Procedure

# Liberty Bielle® Fix™

One of the goals in the development of the Liberty Bielle® Fix™ was the mounting comfort. Liberty Bielle® Fix™ Starter includes 3 appliances of 3 lengths : 32 mm for positioning between first & second molar and the lower cuspid & bicuspid ; 22mm & 20mm for positioning between the bicuspid & the first molar and the lower cuspid & bicuspid. Select the length best suited to your case and check if any adjustment is necessary :

**1** Insert the upper support on the archwire between the first and the second molar or between the first molar and the second bicuspid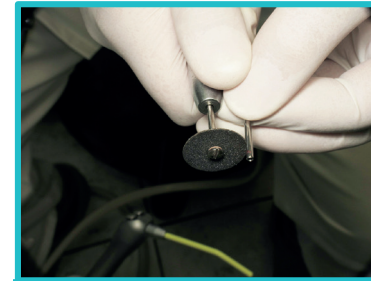

**4** Tighten the upper arm on the archwire using the hex wrench.

**2** If needed, mark the length to the mesiodistal space between the lower cuspid and first bicuspid.

**5** Adjust the lower arm at the proper length and screw the lower support between the lower cuspid and the first bicuspid.

**3** If needed, cut the upper arm at the marked length.

### Advice:

The lower arm should not be longer than 2 mm distal of the upper arm in closed position in order to avoid any discomfort to the patient.

Liberty Bielle® Fix™ can be used on .018 x .025 to .022 x .028 arches. Considering the light forces involved, Liberty Bielle® Fix™ can be fixed on NiTi arches.