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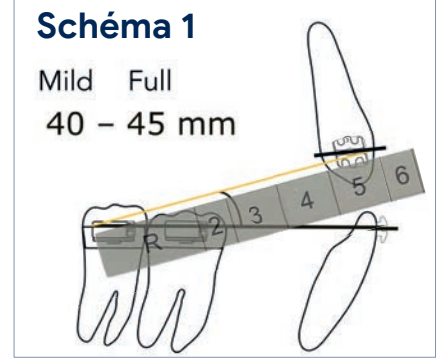
ECi0390

Schéma 1

Mild Full
40 - 45 mm

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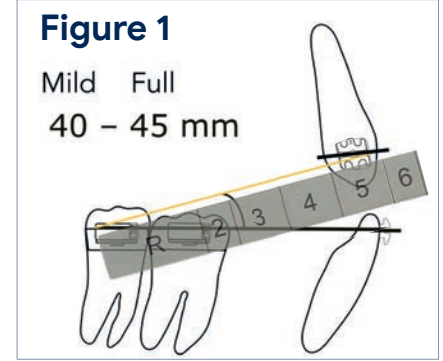
Figure 1

Mild Full
40 – 45 mm

The Elastic Selector Gauge is a tool that allows the clinician to easily select the intraoral elastics for the correction of cross-bite, class II, class III and midline. A recent study¹ tested the different types and sizes of the elastics currently in production, and showed clearly that inventory can be reduced, while also using only 5 types of elastics.

In order to simplify the selection of the elastic for orthodontic purpose, the thicker bands (5 oz.) were chosen, consequently eliminating all the others. As the size of the elastics is expressed in sixteenths of inches, a new numbering is proposed: 1/8"=2 3/16"=3 1/4"=4 5/16"=5 3/8"=6

Elastics Selector Gauge RMO selection chart (5oz.)*	Gauge Number	2	3	4	5	6
	Required diameter <small>1/1 scale</small>					
		1/8" • 0.32cm	3/16" • 0.48cm	1/4" • 0.64cm	5/16" • 0.79cm	3/8" • 0.95cm
Order Number	 jump rope J-01140	 skateboard J-01141	 unicycle J-01142	 bicycle J-01143	 jogging J-01144	

This instrument can be autoclaved and is easy to use:

- with the tool, simply measure the distance between the points of the application of the elastic and read the number;
- the number indicated on the instrument correspond to the elastic to be used for the correction (figure 1-5).

Designed by Dr Sergio Sambataro

For further information,
scan the QR code.

¹Sambataro S, Bocchieri S, Bafumi L, Fiorillo L, Cervino G, Cicciù M. Elastics Selector Gauge as Orthodontics Device Applied to Inter-Maxillary Traction during Malocclusion Correction. Journal of Functional Morphology and Kinesiology. 2019; 4(3):63. <https://doi.org/10.3390/jfmk4030063>

Figures

Figure 2

a. In this growing patient, sectional mechanics is used to correct class II. The elastic will be applied from the lower right first molar to the upper right canine; elastic #4 is suggested.

b. Point of the application of the tool on the lower molar.

Figure 3

a. In this adult patient, sectional mechanics is used to correct class II. The elastic will be applied from the lower right second molar to the upper right canine; elastic #4 is suggested.

b. Same patient on the left side that show a different condition, so elastic #5 is suggested.

Figure 4

a. In this adult patient, straight wire is used to correct mild class III. The elastic will be applied from the upper right first molar to the lower right canine; elastic #4 is suggested.

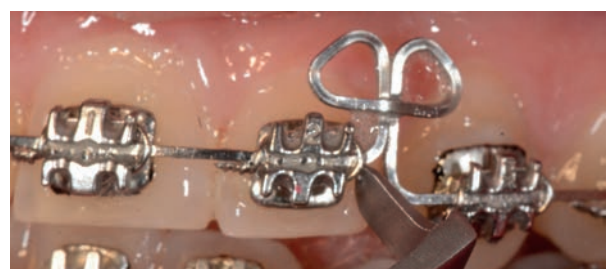
b. Same patient on the left side; elastic #4 is suggested.

Figure 5

a. In this case, elastic #4, applied from the upper left lateral incisor to the lower right canine, is suggested to correct midline.

b. Point of the application of the tool on the upper left lateral incisor.

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Elastics Selector Gauge RMO selection chart (5oz.)*	Gauge Number	2	3	4	5	6
	Required diameter <small>1/1 scale</small>					
		1/8" • 0.32cm	3/16" • 0.48cm	1/4" • 0.64cm	5/16" • 0.79cm	3/8" • 0.95cm
Order Number						
	jump rope J-01140	skateboard J-01141	unicycle J-01142	bicycle J-01143	jogging J-01144	

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Elastic Selector Gauge RMO selection chart (5oz.)*	Gauge Number	2	3	4	5	6
	Required diameter <small>1/1 scale</small>					
		1/8" • 0.32cm	3/16" • 0.48cm	1/4" • 0.64cm	5/16" • 0.79cm	3/8" • 0.95cm
Order Number						
	jump rope J-01140	skateboard J-01141	unicycle J-01142	bicycle J-01143	jogging J-01144	

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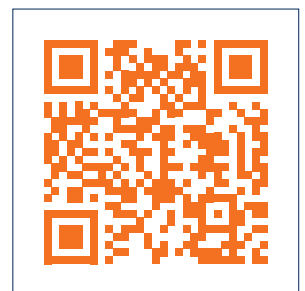
¹Sambataro S, Bocchieri S, Bafumi L, Fiorillo L, Cervino G, Ciccù M. Elastics Selector Gauge as Orthodontics Device Applied to Inter-Maxillary Traction during Malocclusion Correction. Journal of Functional Morphology and Kinesiology. 2019; 4(3):63. <https://doi.org/10.3390/jfmk4030063>

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