# Fiche de port

→ Nom : .....

→ Prénom : .....

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→ Name : .....

→ Last name : .....

To be effective, your appliance must be worn as often as possible and never less than 4 hours during the day and 8 hours at night. The more the appliance is in place, the quicker the desired results will be achieved.

Please fill in the number of hours worn per days.

	1 <sup>st</sup> week			2 <sup>nd</sup> week			3 <sup>rd</sup> week		
	Day	Night	Total	Day	Night	Total	Day	Night	Total
Monday									
Tuesday									
Wednesday									
Thursday									
Friday									
Saturday									
Sunday									

	4 <sup>th</sup> week			5 <sup>th</sup> week			6 <sup>th</sup> week		
	Day	Night	Total	Day	Night	Total	Day	Night	Total
Monday									
Tuesday									
Wednesday									
Thursday									
Friday									
Saturday									
Sunday									

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## Your Doctor

The Multi-Family® System, Functional Education® Appliances is an integrated system of appliances that allow the orthodontist to choose the ideal appliance according to the age and the malocclusion of the patient.

**Multi-S®**  
Multi Start

**Multi-T2®**  
Multi Trainer

**Multi-P2® & P2 Wide®**  
Multi Possibility



**Multi-DB® & DB Wide®**  
For Deep Bite

**Multi-TB®**  
Multi Trainer for brackets

ECXPORT\_V8\_EN\_2022

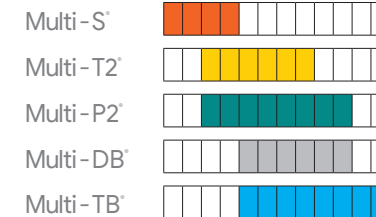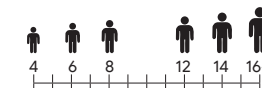### Multi-S® Multi Start

Order Number : ECXS

- Children ages 4-8 years old
- Reduce muscular forces on the teeth
- Promotes proper anterior tooth alignment
- One size fits all

# Multi-Family®

### Multi-T2® Multi Trainer

Order Number : ECXT2


- Children ages 6-12 years old
- Eruption guide for both incisors and cuspids
- Reduce problems associated with: swallowing, thumb sucking, tongue thrusting, labial dysfunction, respiratory problems
- One size fits all

### Multi-P2® & Multi-DB® (Wide)

Multi Possibility

Order Number : XP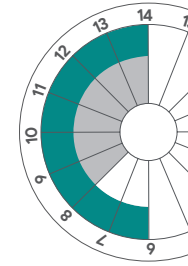

#### Multi P2

- Children ages 6-14 years old
- Eruption guides for incisors, cuspids and bicuspid
- Propulsion guide to free mandibular growth
- Available in "low volume", "high volume", short and long molar region
- From size 35 : Improved material finishing/ Laser marking/ Size accuracy

#### Multi DB

- Including Multi P2
- Children ages 8-14 years old
- Incisal reinforcement for deep bite

#### Multi P2 Wide & Multi DB Wide


- Large appliances for ovoid arch shapes

### Multi-TB®

Multi Trainer for Brackets

Order Number : ECXTB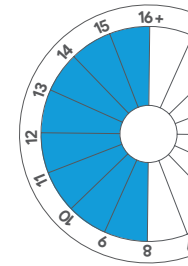

- Children ages 8 years and older equipped with fix appliances.
- It affects the mandibular unlocking, reshaping of the arch, the tongue placement in the palet, the middle correction. increases the efficiency of fix appliances.
- Used while in Fixed Appliance treatment with brackets.

## How to use Multi-family® appliances

### In the daytime

- while reading,
- while at the computer,
- while watching television,
- while doing homework.

### At night

Your appliance must be worn all night. In the early days of treatment it is not uncommon for the appliance to dislodge during the night time.

Wearing the appliance regularly during the day will quickly resolve this problem. **Remember, a disciplined approach in the use of the appliance is the key to successful treatment.**

### Practical advices

- When not in use, rinse and clean the appliance with a toothbrush. It can be disinfected in boiling water. Store the disinfected appliance in its container.
- Some initial discomfort is common at the beginning of treatment. This is a clear indication that the appliance is working. It is only temporary and will disappear after a few days.

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