SAM®

# AXIOQUICK®

DER ANATOMISCHE TRANSFERBOGEN  
*THE ANATOMIC FACEBOW*

CE

 Made in Germany

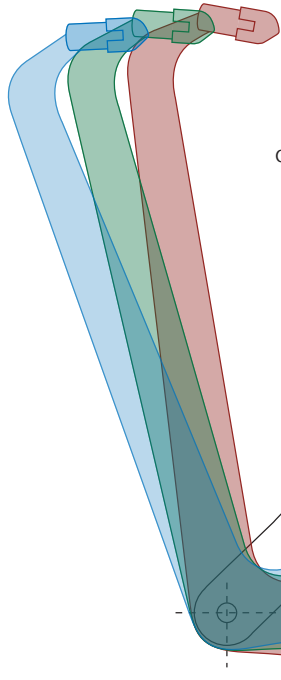
# Das AXIOQUICK Transfersystem

## The AXIOQUICK Transfer system

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*The AXIOQUICK Esthetic Cranial Related Anatomic Face Bow Transfer System records and transfers the maxillary teeth to cranium relationship and also the condyle rotation center position that is used for functional mandibular recordings and centric relation jaw records reference position.*

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
*The cranial related Frankfort horizontal (FH) plane is the transverse reference for the patient and the articulator. The reference points of the Frankfort horizontal plane (porion and orbital) are the same as the axis-orbital plane.*

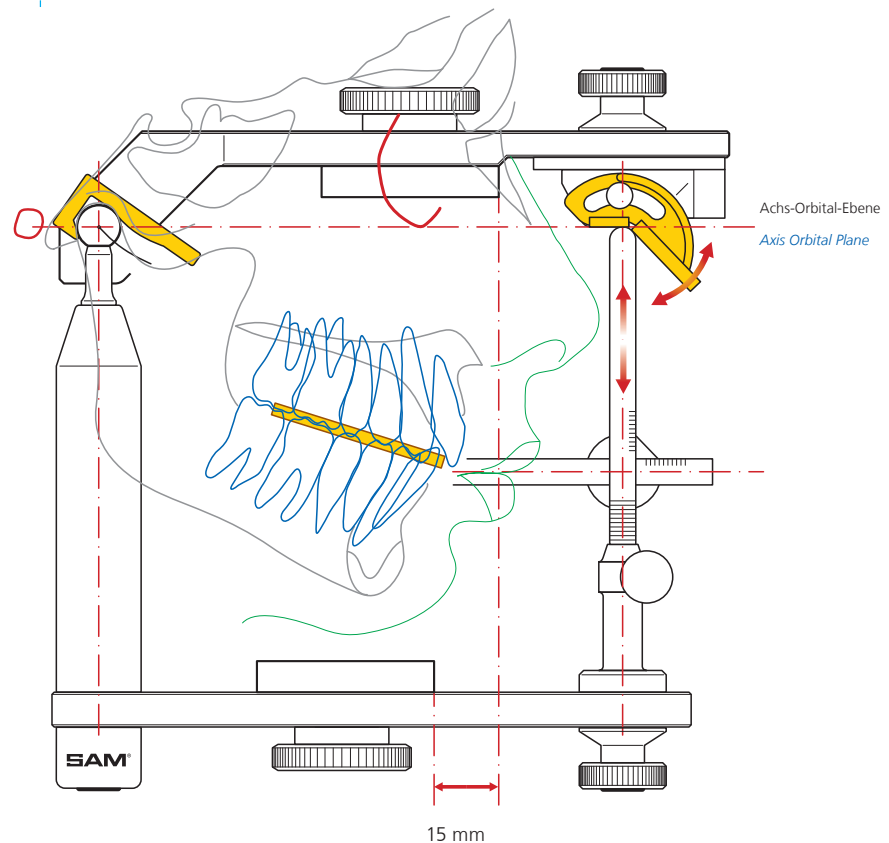*The caliper type of mechanism of the SAM® facebow provide for the optimal positioning of the transfer fork.*

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## *The FACEBOW for Esthetics and Function*

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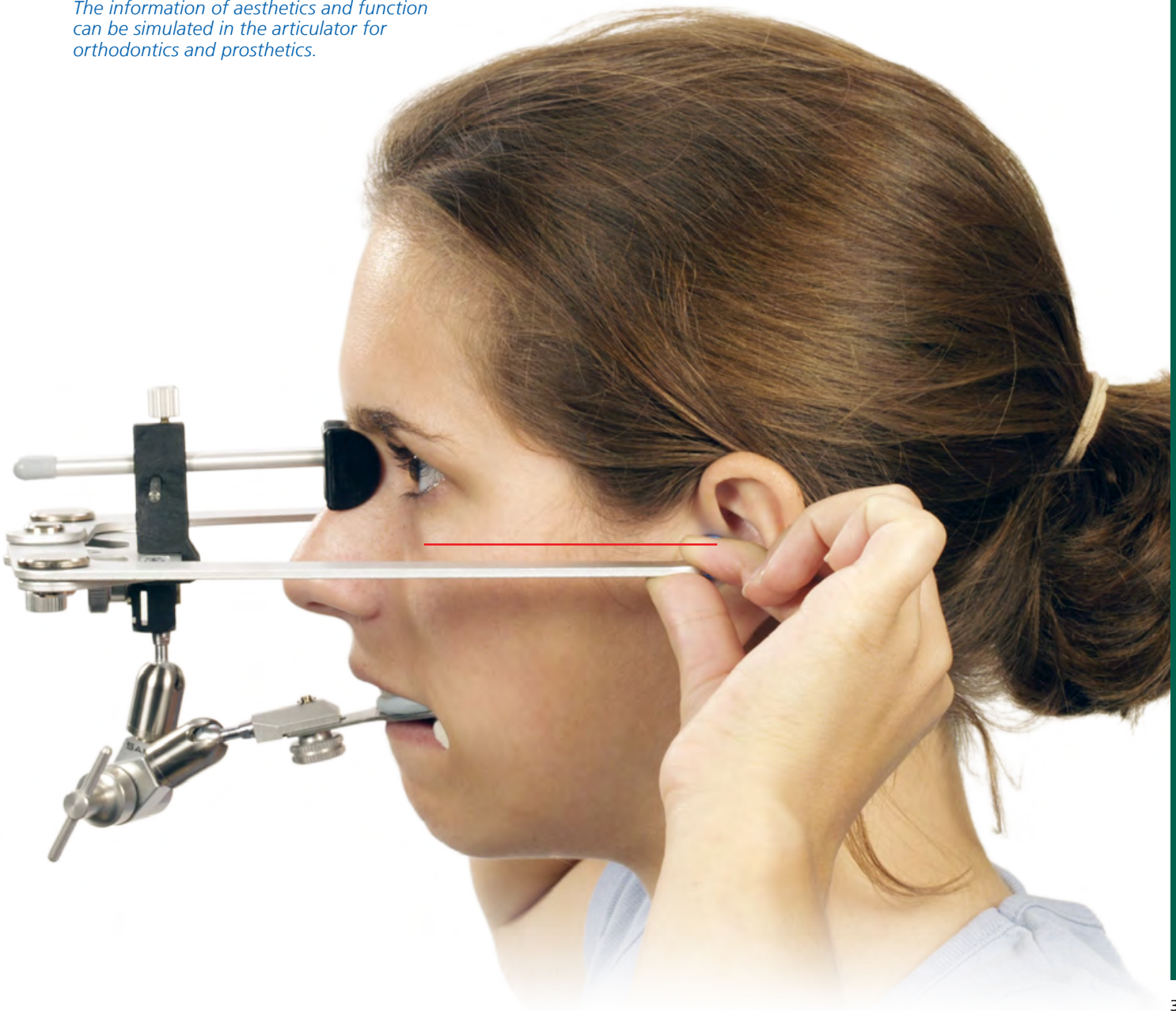
*The special geometry and mechanics of the SAM® facebow provides an automatic balancing on the skull. The Anatomic Porion oriented and cranial related attaching technique assures that the skull and maxillary teeth esthetics are correctly transferred to the articulator. The outcome of the on both-sided correctly to the Porion attached facebow is the together with the orbital reference plane the „Frankfurt plane“.*

*The patented nasion relator with the interpupillary line leveler helps to assure the parallel relationship between the interpupillary line and the inter anatomic porion line in the face bow earpieces.*

*The information of aesthetics and function can be simulated in the articulator for orthodontics and prosthetics.*

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*The blue hygienic earpiece caps (ATB240) are set to the black earpieces of the face bow.*

*The notch is pointing outwards, the flat side facing inward*

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*The nasion relator (ATB394) is properly positioned on the face bow (ATB303) when it is fully inserted between the pins on the underside of the face bow.*

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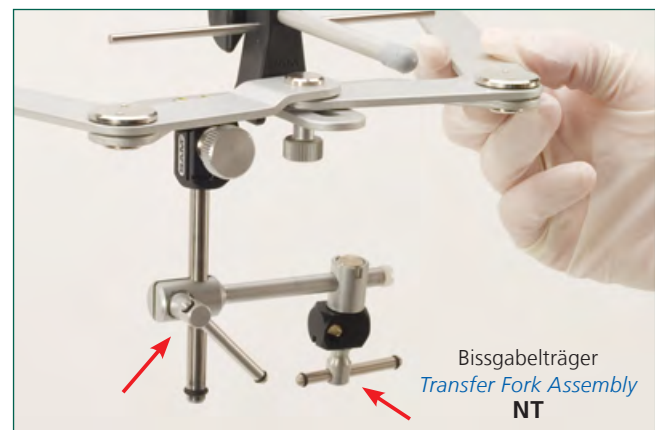
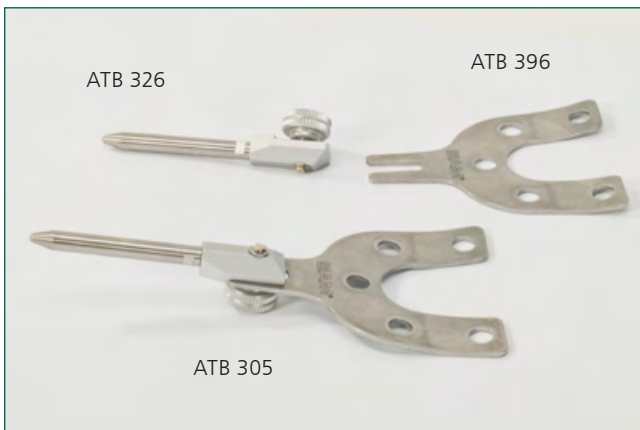
*The transfer fork assembly is pushed onto the sliding attachment at the bottom of the AXIOQUICK® face bow until it is fully seated. The thumbscrew is loosened lightly and then 1/2 turn more. By tightening the thumbscrew, the attachment connection is secured.*


*The other two bar clamps are loosened 1/2 turn to allow sliding. When in the correct position, Tighten lightly and*

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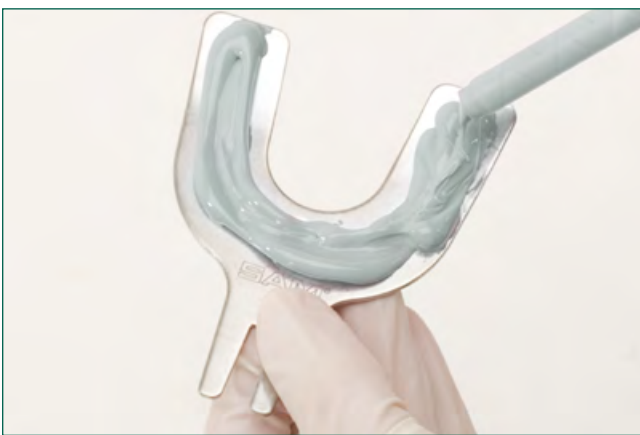*The new transferfork assembly NT is in two parts and can be assembled prior to coating with a recording material, which facilitates the sagittal alignment.*

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*A hard type of silicone can be used for the face bow transfer. Apply a layer over the holes under the curved side of the fork and then a thin layer on the top.*

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*The examiner holds the transferfork in contact with the upper teeth and without pressure.*

*The anterior midpoint of the fork and upper teeth midline should match.*

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*After hardening, the transferfork is taken out of the mouth and all the impressions and molded deep fissures and interdental spaces are eliminated. It will remain only the impressions of the cusps.*

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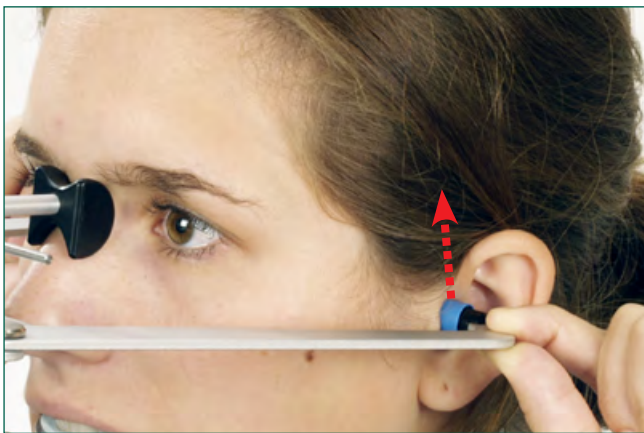
*The transferfork is re-inserted and supported with cotton rolls in the area of the premolars. The patient closes his mandible jaw and fixes the transfer fork with the mandible teeth.*


**The backrest is lowered more than 45°**

*The face bow is held with thumb and index finger on the corner of the earpiece by the patient.*

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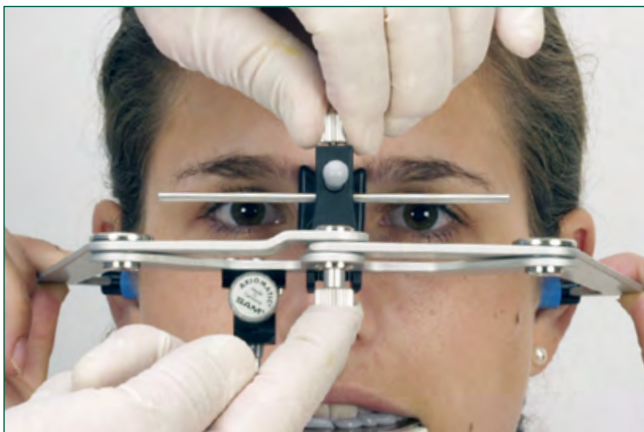
*The patient inserts the earpieces into the ear canal until hearing lightly blocked. Patient is instructed to hum so that there is now equal high pitched sounds on both sides. To bring the face bow in the exact position, the patient himself raises the earpieces straight upwards and then inwards into a small superior depression just above anatomic porion. The face bow is held firmly and the interpupillary line is rechecked and adjusted.*

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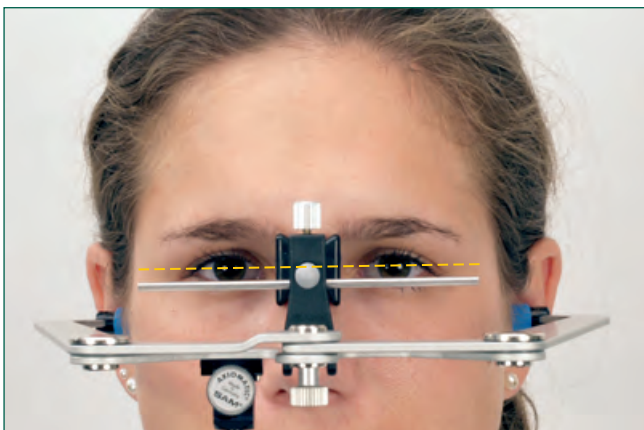*When the face bow is in the correct position, the patient can fully hear again. The nasion relator is brought into contact with the nasion with light contact. With the upper thumbscrew the nasion relator is fixed and the facebow is kept in place.*

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*The interpupillary line leveler is used to control the parallelism of the interpupillary line and facebow. The facebow is now esthetically correct and aligned to the Frankfort horizontal.*

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## Connecting the transfer fork assembly

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*When the toggle of the transfer fork assembly is half open, the guide component can be easily put on the transfer fork.*

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*The clamps of the two-toggle transfer fork assembly NT system are open 1/2 turn, the clamps slide easily from top to bottom and at the correct height to handle the transfer fork.*

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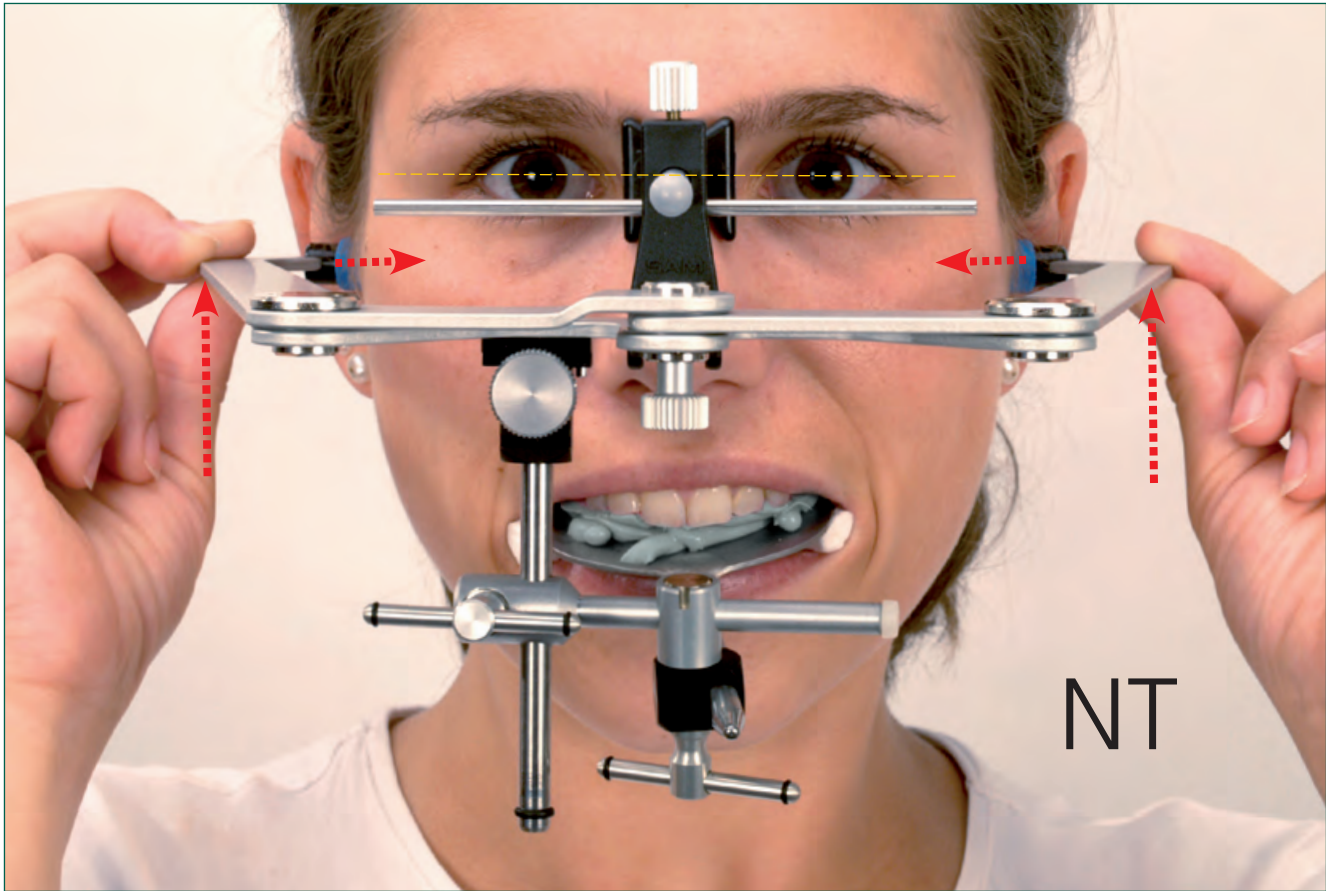
*With one hand, the transfer fork assembly is stabilised, with the other hand, the screw is tightened.*

*With the two-toggle transfer fork assembly NT the NT clamp is fixed first and then the vertical rod clamp last.*

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
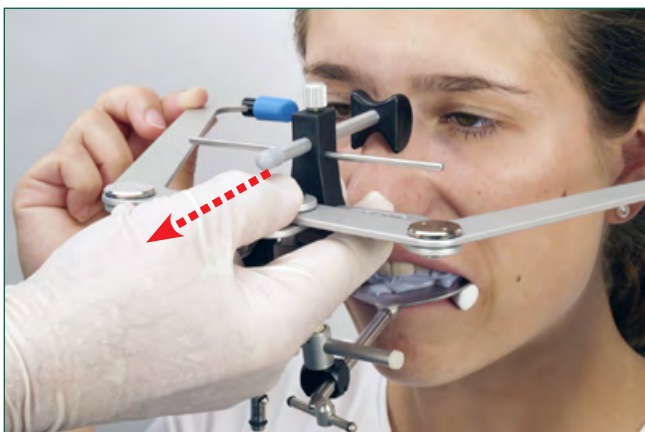
*The patient opens his mouth and removes the earpieces from the ear canal.*

*The right hand of the practitioner supports the middle part of the opened face bow.*

*For upper cast mounting with SAM® transfer stand (ATB 398) the transfer fork assembly is removed from the face bow sliding guide.*

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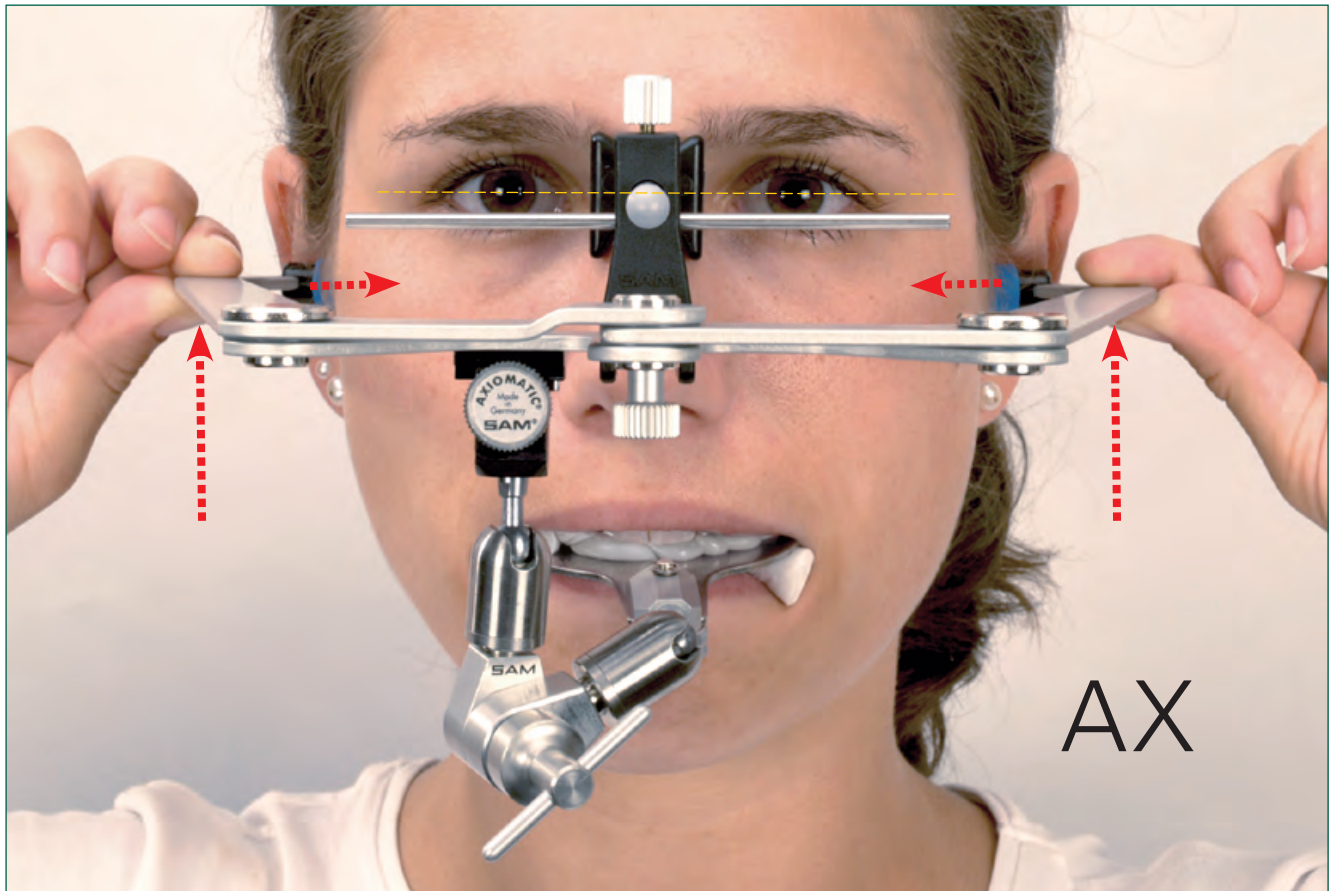*The patient opens his mouth and removes the earpieces from the ear canal.*

*The right hand of the practitioner supports the middle part of the opened facebow.*

*For upper cast mounting with SAM® transfer stand (ATB 398) the transfer fork assembly is removed from the sliding guide.*

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*The optimal solution for Mounting the upper cast is using the transfer stand AX (ATB 398).*

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*A magnetic block (ATB 338 and 339 for screw type) instead of the mounting plate accommodates for the later set up transfer fork support a big and secure support area.*

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*The transfer-fork assembly is inserted in the guide on the upper part of the transfer stand (black) until it stops.*

*The thumbscrew on the transfer fork assembly is tightened.*

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*The telescopic transfer fork support (ATB 336) is slid under the transfer fork..*

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*The articulator is turned over and the thumb screw on the transfer fork support loosened. The discs of the transfer fork slides down until they apply the transfer fork without pressure. By screwing the thumb screw the transfer fork support is fixed. The articulator can be reversed again.*

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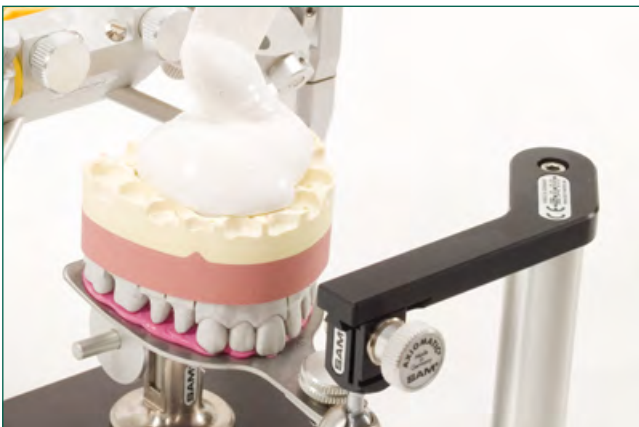
*The upper cast is seated into the impression on the transfer fork. A preliminary check is made to assure enough space for mounting gypsum. A simple rule for space is about 15 ml of water for each finger width of space between the mounting plate and underside of the cast.*

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
*Special mounting gypsums are available with minimum expansions that can be mixed to a creamy consistency. Gypsum is applied to mounted plate first. Dental should be slightly moist prior to adding mounting gypsum to the under surface of the dental cast.*

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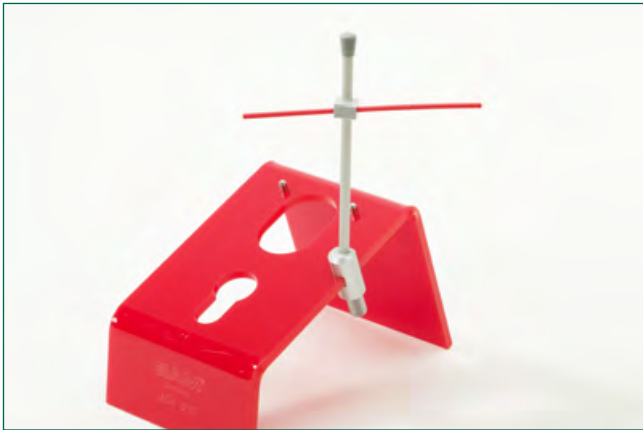## *Mounting with centric registration*

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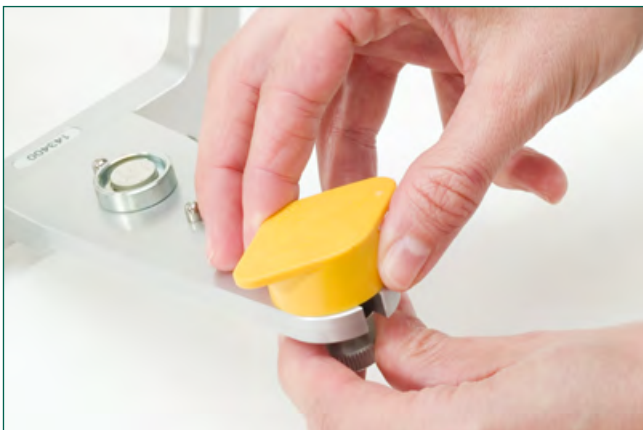
*The mounting stand 560 MOH is used for the mandibular cast mounting and is compatible with all SAM® articulators and laboratory cast mounting instruments.*

*Before inserting the articulator in the mounting stand the incisal pin is inserted into the upper part of articulator.*

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*The thickness of the occlusal registration in the range of premolars is determined with a measuring forceps. Use twice value to increase the height of the incisal pin.*

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*The incisal table can be placed in the lower part of articulator to avoid an obstruction during the closing rotation.*

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*It is strongly recommended to fix centred the models with the intermediate registration with the red flexible plastic rod.*

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*Prior to adding creamy gypsum to the lower mounting plate be sure to check for proper clearance. If OK, moisten under surface of lower cast before closing the lower member of the articulator.*

*The articulator is closed until the contact of the incisal pin with the incisal table.*

# Schädelbasis, Frankfurter Horizontale, Interpupillarlinie

## *Anatomic Porion, Frankfurt Horizontal, Interpupillary line*

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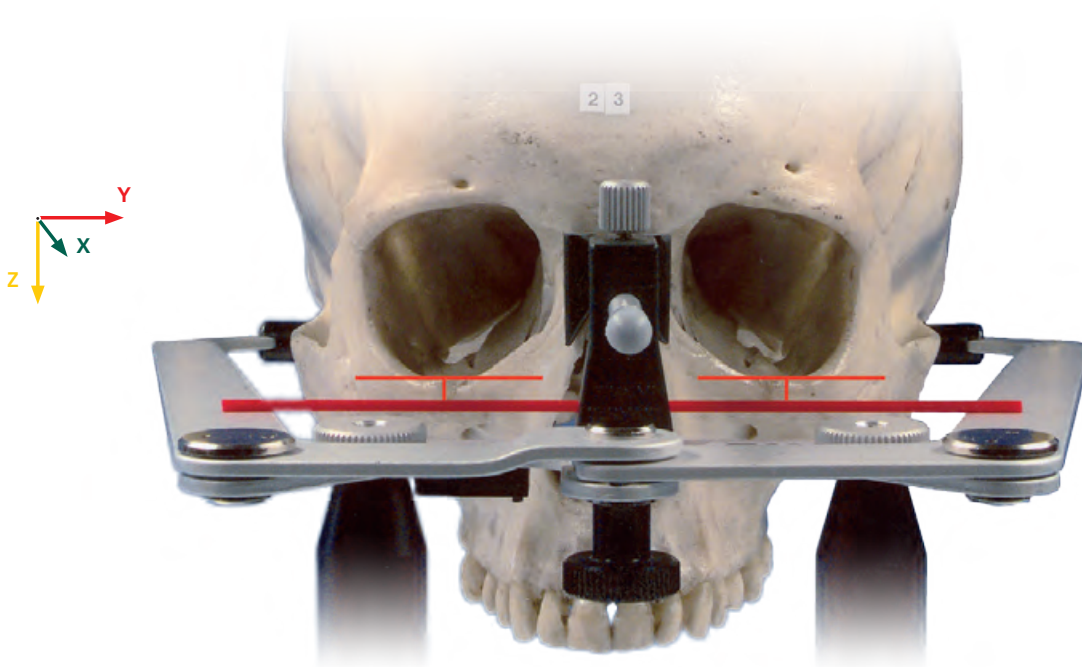
*The AXIOQUICK face bow was attached to 122 skulls and digitally photographed. Using modified earpieces, it was possible to identify midsagittal and horizontal planes and to show that the horizontal plane of the face bow and the lower limit of the orbits we always sufficiently parallel to the horizontal plane of the facebow. Another similar study was done with 44 skulls and it showed the horizontal plane was sufficiently parallel to a projected interpupillary plane. The survey using Photoshop 6.0 revealed side differences of the skull in 70% of cases, as well as patients of  $\leq 1$  mm and lateral deviations of  $< 2$  mm. This study shows the AXIOQUICK face bow is a clinically acceptable method for transferring the cranial relationship of the upper jaw to an articulator such that they will be virtually the same.*

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*Both in the group of historical skull, as well as in the group of patients, approximately 70% of the other side difference  $\leq 1$  mm. In the larger areas of deviation is observed, however, a higher variability in the skulls. In the group of patients were all side differences  $\leq 2$  mm in the group of the skull were 8%  $> 2$  mm. The difference in the scatter of measurements is confirmed by the coefficient of variation.*

Bergstrom G.

On the reproduction of dental articulation by means of articulators, a kinematic investigation. Acta Odontol Scan 1950;9 (suppl 4):125-141

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

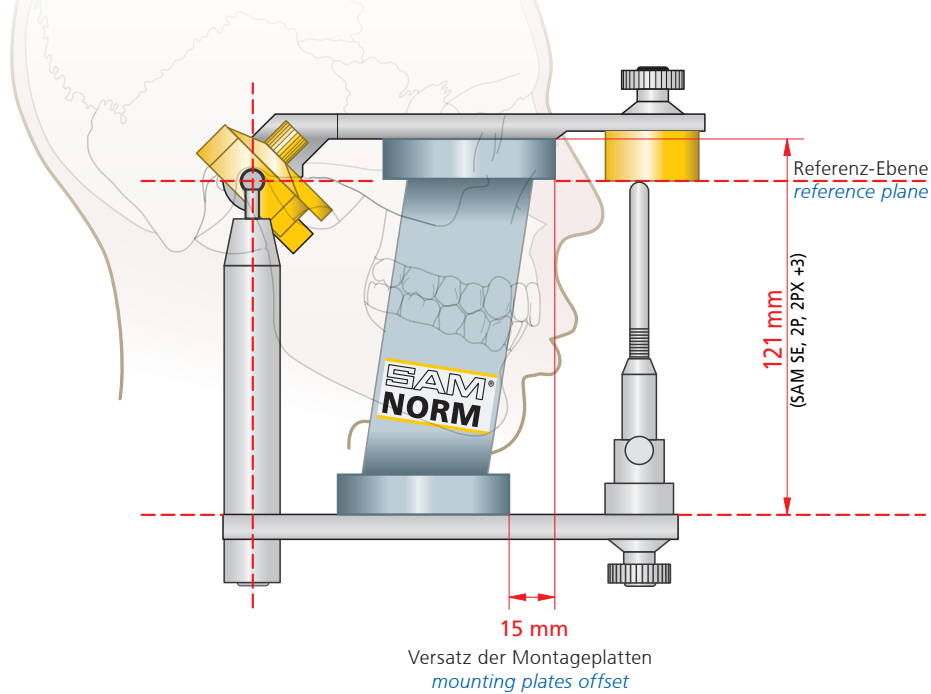
*The SAM® Norm, in combination with the anatomic facebow, assures the correct positioning of the upper cast in the articulator. The aesthetics and function are accurately simulated for Orthodontics and Prosthetics.*

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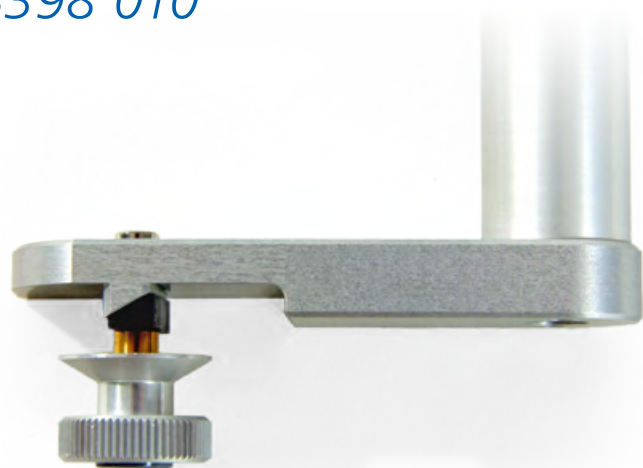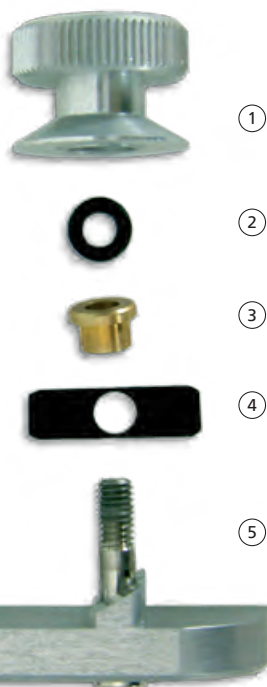
*Please assemble part number 1-4, in order shown on to fixed screw (5) of lower part assembly.*

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*When being used, **DO NOT** unscrew thumb nut **COMPLETELY**.*

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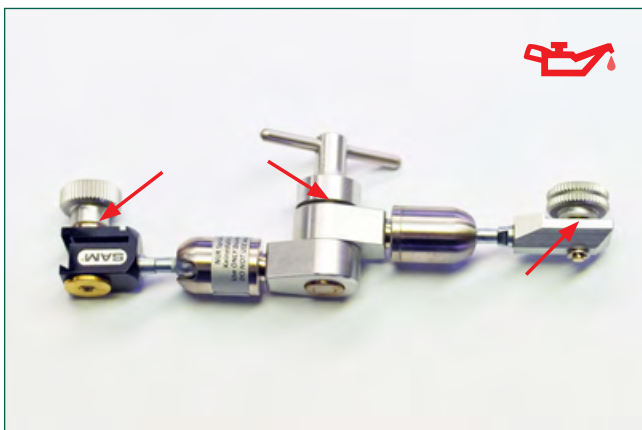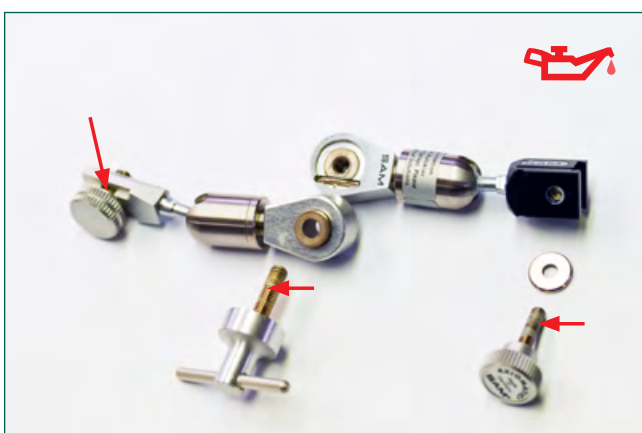
*The internal and external threads of the transfer fork assembly should be periodically lubricated with dental hand piece oils or similar type of lubricants*

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*Open the toggle and fixing screw of the transfer fork assembly AX.*

*Place drop of oil only on the thread poles of the transfer fork assembly.*

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*Pay attention with the reassembly to the fact that the protruding edge of the joint has a precise place in the indentation and fitting should require no force.*

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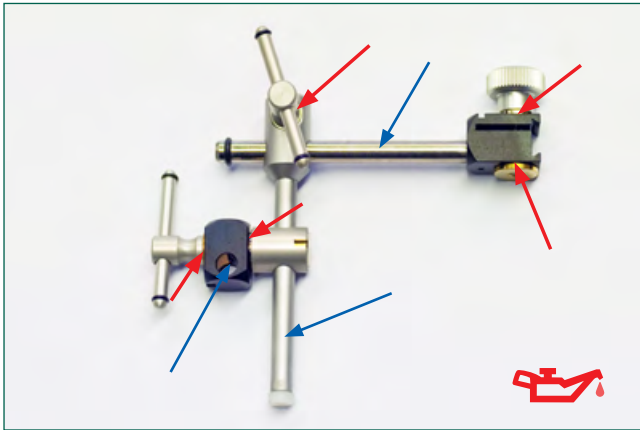
### Please:

*Never tighten thumbscrews more firmly than needed.  
The screws threads will be damaged.*

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
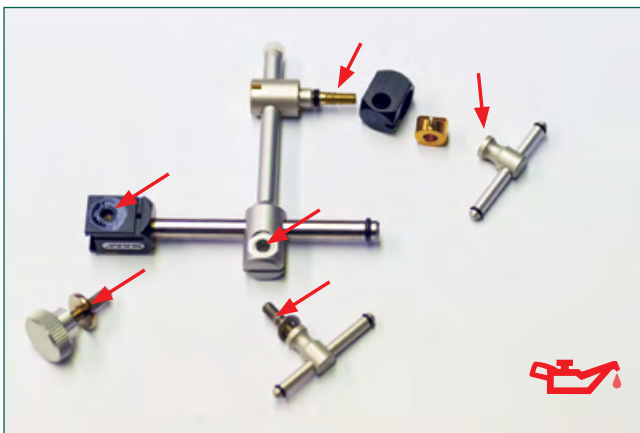
*The blue marked rods and holes for the transfer fork must be absolutely grease-free. They should be degreased periodically with alcohol to preserve the proper friction effect for clamping.*

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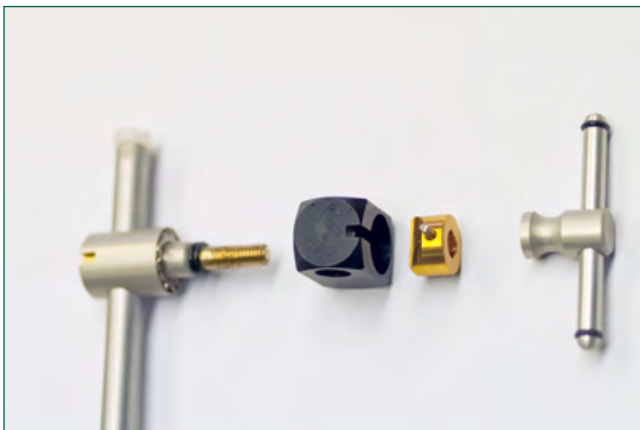*The internal and external threads of the transfer fork assembly should periodically be lubricated with dental hand piece oils or similar types of oils.*

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*The diagram on the left shows the proper assembly of the parts in the clamping mechanism. Notice that the cylinder pin completely should fit in the notch. Only then the transfer fork assembly may be screwed together again.*

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### Please note:

The black clamping mechanism does work unless there is a transfer fork handle in the slot.

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## Cleaning, disinfection and sterilization

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*The blue hygiene protection caps are designed only for the single use and should be treated as such.*

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*The transfer fork is classified as a „semicritical medicine product“ (according to MPG) and must be sterilized as such. For this the steam sterilization is recommended in the fractionated vacuum procedure for 134 degrees centigrade for 10 minutes in a device after German Institute for Standardization DIN EN ISO 17665-1.*

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*It is highly recommended to use type of cloth wipes for cleaning the SAM® face bow and transfer fork assemblies.*

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
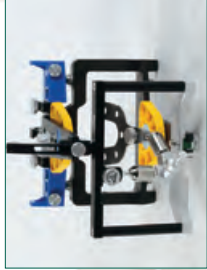
*Facebow, transfer fork assembly, adaptors and nasion relator should NEVER be played in chemical solutions or ultrasound.*

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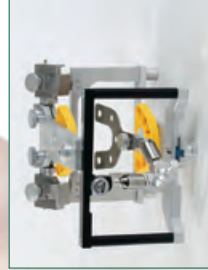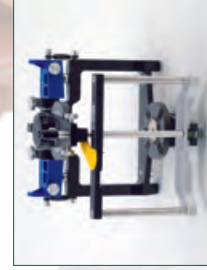
*With the SAM® anatomic facebow and the special transfer stands, the upper jaw model can also be transferred to most current articulators that are not made by SAM®. Using the the AX transfer stand, the AXIOMATIC® transfer fork assembly or any other SAM® transfer fork assemblies with a sliding guide (ATB 310, ATB 320, ATB 305) can be attached to the transfer stands for the transfer of the upper cast.*

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Fussbergstrasse 1 82131 Gauting Germany

Tel: +49 (89) 800 654 - 0 • Fax: +49 (89) 800 654 - 32

info@sam-dental.de • www.sam-dental.de

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SAM® Präzisionstechnik GmbH