

Recommendation for Cleaning and Sterilization of the Aluminum Impression Tray:


1. Remove alginate residue from the tray using a toothbrush or plastic (not metal) spatula and water.
2. Place in neutral (not acidic or basic) solution of alginate tray cleaner, following the instructions from the manufacturer of the tray cleaner. Dedicate a fresh solution only to aluminum trays. Use only aluminum trays in that solution; do not also clean stainless steel instruments or metal appliances at the same time. Do not use the same cleaning solution that you previously used for other metal instruments or appliances.
3. Remove trays from solution, rinse, and dry thoroughly.
4. Sterilize using a dry heat sterilizer, following the procedures from the manufacturer of the sterilizer.

RMO[®]
rocky mountain[®] orthodontics
www.rmortho.com



RMO, Inc.
650 West Colfax Avenue
Denver, Colorado 80204 USA
To order call 303-592-8200
Toll Free 800-525-6375

CE **REP**
European Representative
RMO Europe
Rue Geier de Kayserberg
67400 Illkirch, France

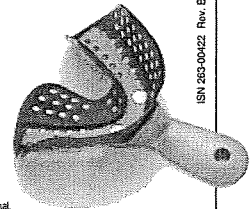

ISO 263-00422 Rev. B Made in Japan

U.S. federal law restricts this device to sale by or on the order of a dental professional.