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MIGMA 200
The original mixer for
bubble-free impression
compounds.

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NOUVEAU PRIX

MIGMA 200
MIKRONA

MIKRONA
The Swiss Dental Technology

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The base for precise impressions.

MIGMA 200 guarantees a homogeneous, bubble-free impression compound and thus creates the base for precise impressions and a perfect prosthetic work.

MIGMA 200 mixes alginate and other impression materials in a few seconds.

MIGMA 200 is easy to use, compactly built and is available as a table-top model (1) as well as a built-in model (2).

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100 VAC - 230 VAC, 50/60 Hz, 250 W

Standard equipment (3)2 large mixing bowls with lids
1 spatula**Options for the mixing of Alginate (4)**

Water flask

Options for the mixing of Silicone, Polyether compounds as well as adhesives (5)Adapter to small mixing bowl
Small mixing bowl
Rapid applicator
Adapter to rapid applicator
Small stirring rod**Dimension table-top model (1)**

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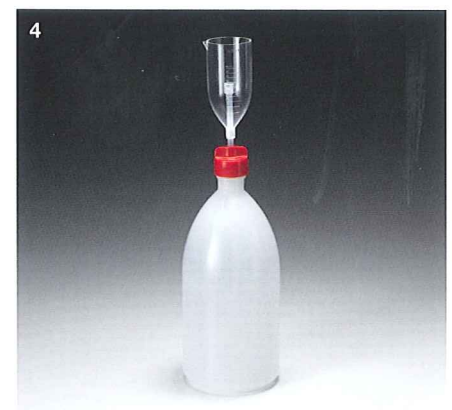
The compound for precise impressions, instantly ready.

- 1 Chose the mixing time on the scale.
- 2 Dose the alginate as suggested by the producer and fill into the mixing bowl.
- 3 Add the correct amount of water either with the producer's measuring cup or with the optional available water flask.
- 4 Close the lid of the mixing bowl with a twist.
- 5 Shake the bowl once or twice.
- 6 Put the mixing bowl into the machine and close the lid of the machine.
- 7 Press START. The machine stops automatically when the chosen time is up.
- 8 The alginate can now easily be removed from the bowl, using the spatula.
- 9 Within seconds a homogenous, bubble-free impression compound.

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MIKRONA TECHNOLOGIE AG