Patienteninformation

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Patient information

Your dentist has given you one or more BRUX CHECKER® foils which were moulded over your maxillary cast. A biocompatible colour coat has been applied to one side of this foil, which will be ground on the contact points between upper and lower jaw. By means of the grinding pattern which will be thus received, your dentist can evaluate potential disorders of your stomatognathic system such as grinding (bruxism).

In order to receive a significant result, please note the following points:

- The foil will be worn during sleep.
- Please press the foil carefully onto your upper jaw, beginning at the frontal teeth and continuing to the molars.
- In the morning, please proceed reversely. Please remove the foil with the finger tips beginning at the molars and then to the front teeth.
- Rinse the foil under lukewarm water and let it dry. Then put the foil again over the plaster model, in order to avoid deformation.
- Each foil is for use during one night only.
- Please avoid alcohol and medication like sleeping pills, as this may lead to a non-typical grinding pattern.
- After return of the model and the foil, your dentist will talk with you about the following measures to take.

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Laboratory information

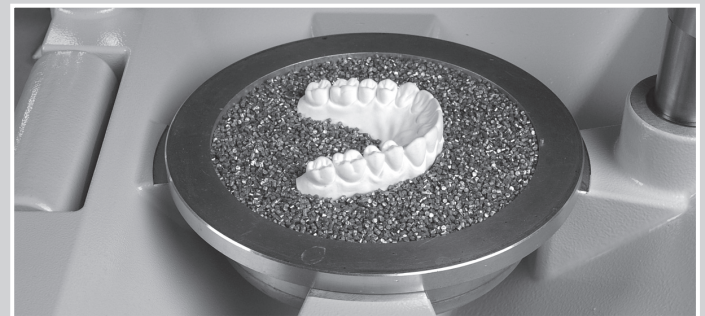
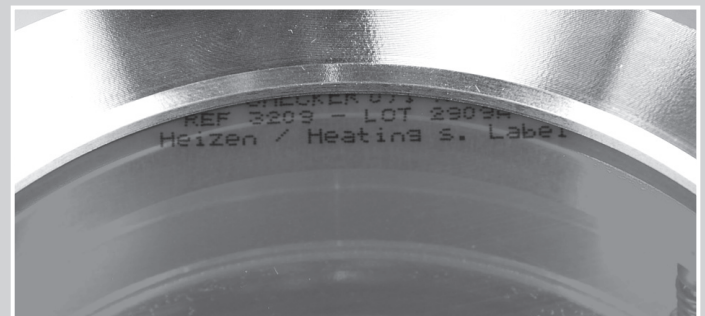
SCHEU-DENTAL pressure moulding machines:

Please place the red BRUX CHECKER® foil (without white separating foil) on the O-ring of the adjustment ring with the imprint on the foil right reading visible and secure the foil by means of the locking ring.

Place the model in the pellets or on the model platform so that the dental arch can be easily moulded. You can achieve optimal results by embedding the model up to 5 mm below the gingival margin. It is important that the model should be placed with the incisors facing towards the locking handle.

Machines of other manufacturers:

Foils of 120 mm diameter carry the imprint „coated side“. Please pay attention that after pressure moulding the imprint on the model can be read properly. The right application can be tested by scratching.

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