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## Important directions and safety instructions

*SYMPROfluid Universal / Nicoclean are highly effective cleaning agents.\*  
Please note the following recommendations in the interests of safety and the maximum possible cleanliness:*

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### 1. Personal protection

- Disinfect object as prescribed by the hygiene regulations.
- Wear personal protective equipment.

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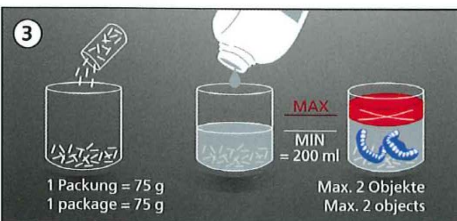
### 2. Positioning the unit

Place unit on an acid-resistant and non-slip surface.

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### 3. Filling the beaker

- Empty 75 g of pins into beaker.
- Fill up to MIN mark with *SYMPROfluid Universal* (~200 ml).
- Once the objects have been added to the beaker the level must not exceed the MAX mark.

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### 4. Removing objects

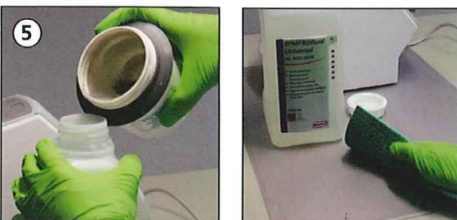
- Take out objects using tweezers.
- Rinse thoroughly with water.

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#### NB:

Disinfect objects as prescribed by the hygiene regulations before bringing them into contact with the patient.

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### 5. Changing cleaning fluids

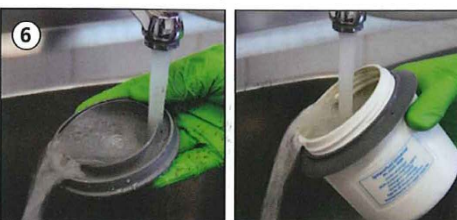
- Pour used cleaning fluid into a canister.
- Dispose of in compliance with official requirements.

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#### NB:

Clean up any *SYMPROfluid* spillages immediately using a damp cloth and then wipe over area of spillage again with a dry cloth.

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### 6. Rinsing beaker, lid and pins

- Rinse beaker, pins and lid thoroughly with water
- Dry grooved areas of beaker and lid with a paper towel. This prevents any residual fluid from dribbling out when the beaker is used again.

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### 7. Storing needles

- After thorough cleaning (rinse them 3 times), store pins in the beaker
- Fill up the beaker with water up to MIN-mark and close it with the lid

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Renfert GmbH