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FOR ACTIVATION OF THE 3D® LINGUAL ARCH OR 3D® SECTIONAL (WILSON®)

The 3D® Activator is a dynamic 3-dimensional force system, having directional movement possibilities for predictable forces. It is designed with five separate angles, and activation of these angles is with an i-140 RMO plier.


NOTE: Care must be taken to assure that you don't over-activate any angle. Excessive adjustments cause arch displacement, loss of control and gingival compression. 1.0mm or less will give you 100% force dissipation.

No heat treating is required. (Please see illustrations on front of instructions.)

The 3D® Wilson® lingual appliances are inserted in the 3D® lingual tubes, RMO order no. AO4314 with gingival hook, or RMO order no. AO4315 without gingival hook.

NOTE: Use the i-420 plier only on these 3D® Modular Orthodontics (Wilson®) appliances: 3D® Quad Helix, 3D® Palatal Appliance, 3D® Multi-action Palatal Appliance, D.Y.S. Appliance and the Clark LAD®.

It is recommended that an i-420 plier be used for all leveling, torquing, and rotation adjustments made to the 3D® Twin Posts. Engage the appliance using the outermost grooves of the plier. Bends should be made using thumb or finger pressure while holding the appliance as shown.

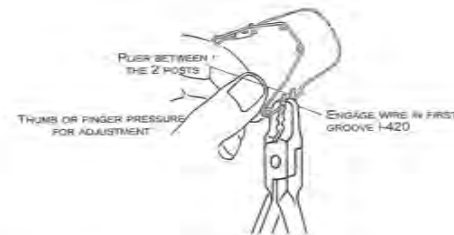CAUTION: As with all chairside treatment, care should be exercised to avoid the possibility of a patient aspirating this 3D® Wilson® appliance. Always transport the appliance using a well-serrated How plier. If needed, use dental floss tied between the twin vertical posts. Remove floss after plug-in.

The friction-fit between the 3D® lingual tube and the twin vertical posts may require adjustment to ensure secure retention of the appliance. This is particularly true if the appliance has been subjected to multiple insertions/removals during fitting or activation. Always monitor the nature of the friction characteristics at final insertion and follow the steps below if less than adequate friction is detected. A firm and stable engagement of the posts in the lingual tube confirm ideal friction.

For 3D® Wilson® appliances with wire-formed posts:

Carefully install a slight mesially distal inward cant of the posts using a 3-jaw plier as shown.


For 3D® Wilson® appliances with solid posts:

Carefully install a slight buccal-lingual cant of the posts as shown. Hold each post with a separate How plier.

If space and soft tissue permit, use a How plier between the gingival of the lingual tube and the post section of the appliance to engage the posts. Otherwise a firm, gingivally directed force using a band director will achieve complete engagement. In either case, if less than adequate friction is detected during final insertion in the mouth, the following adjustments can be made to reestablish the ideal friction-fit.

WELDING INSTRUCTIONS FOR 3D® LINGUAL TUBES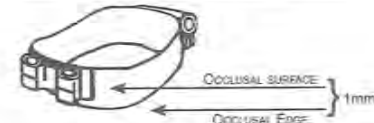

Weld tube to band so that the occlusal surface of the tube is 1mm from the occlusal edge of the band.